


Adjustable Angle Footplate with Pivot Tube - AT5543 (Before 6/1/03)

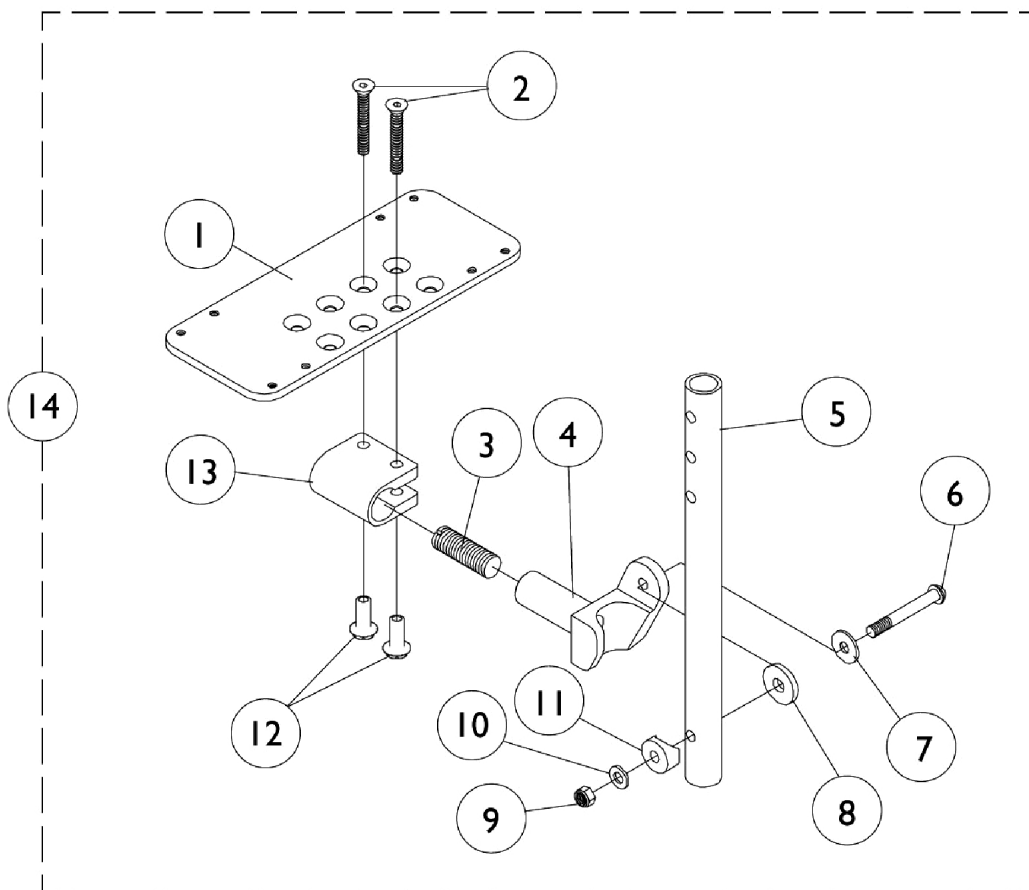

Adjustable Angle Footplate with Pivot Tube - AT5543 (Before 6/1/03)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1060425-NLA	NLA - Footplate, Adjustable Angle, Small, Black (3-1/4" W x 8" L)	1	1
1		1060426-NLA	NLA - Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
1		1060427-NLA	NLA - Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
1		1062569-NLA	NLA - Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
2	A	1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
3		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
4		1052102	Hinge, Footplate - Right	1	1
4		1052103	Hinge, Footplate - Left	1	1
5		1120384-NLA	NLA - Tube, Pivot/Slide, Straight - Alum. (9-7/8" L)	1	1
5		1120385	Tube, Pivot/Slide, Straight - Black (11-7/8" L)	1	1
6		7930321-NLA	NLA Package, Screw, Socket Button Head (1/4-20 x 1-7/8")	25	1
7		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
8		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	1
9		7930333	Locknut (1/4-20)	25	1
10		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
11		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
12	A	1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
13	A	1058517	Clamp, Footplate	1	1
14		See Chart	#14 90 Degree and 70 Degree MFX	1	1
14		See Chart	#14 70 Degree Tapered	1	1
14		See Chart	#14 60 Degree, 60 Degree MFX, 70 Degree Non-Tapered, & ELR's	1	1

NOTE: A - Due to a hardware change, for chairs built Before 8/99 items 2, 12, and 13 must be ordered together.

Item #14 Adjustable Angle Footplate w/ Pivot Tube

60 Degree, 60 Degree MFX, 70 Degree Non-Tapered, & ELR's

Straight Slide Tube (9-7/8")	Right	Left
10-11" Wide (3-1/4" Footplate)	1060428-NLA	1060429-NLA
12-13" Wide (4-1/4" Footplate)	1060430	1060431
14-15" Wide (5-1/4" Footplate)	1060432	1060433
16" Wide (6-1/4" Footplate)	1062572	1062573
ALPT Straight Slide Tube (11-7/8") Aluminum	Right	Left
10-11" Wide (3-1/4" Footplate)	1062574-NLA	1062575-NLA
12-13" Wide (4-1/4" Footplate)	1062576-NLA	1062577-NLA
14-15" Wide (5-1/4" Footplate)	1060440-NLA	1060441
16" Wide (6-1/4" Footplate)	1062578	1062579

Adjustable Angle Footplate with Pivot Tube - AT5543 (Before 6/1/03)

Item #14 Adjustable Angle Footplate w/ Pivot Tube

70 Degree Tapered

Straight Pivot Tube (9-7/8")	Right	Left
16" Wide (4-1/4" Footplate)	1060430	1060431
ALPT Straight Pivot Tube (11-7/8")	Right	Left
16" Wide (4-1/4" Footplate)	1062576-NLA	1062577-NLA

Item #14 Adjustable Angle Footplate w/ Pivot Tube

90 Degree and 70 Degree MFX

No Pivot Tube Bolts Through Support	Right	Left
10-11" Wide (3-1/4" Footplate)	1060434-NLA	1060435-NLA
12-13" Wide (4-1/4" Footplate)	1060436-NLA	1060437
14-15" Wide (5-1/4" Footplate)	1060438	1060439
16" Wide (6-1/4" Footplate)	1062570	1062571