

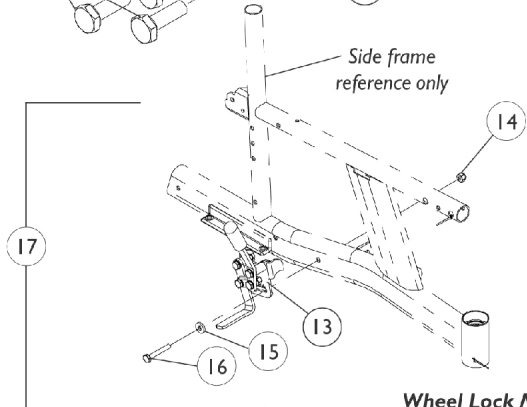


Accessories


Wheel Locks After 2/14/03

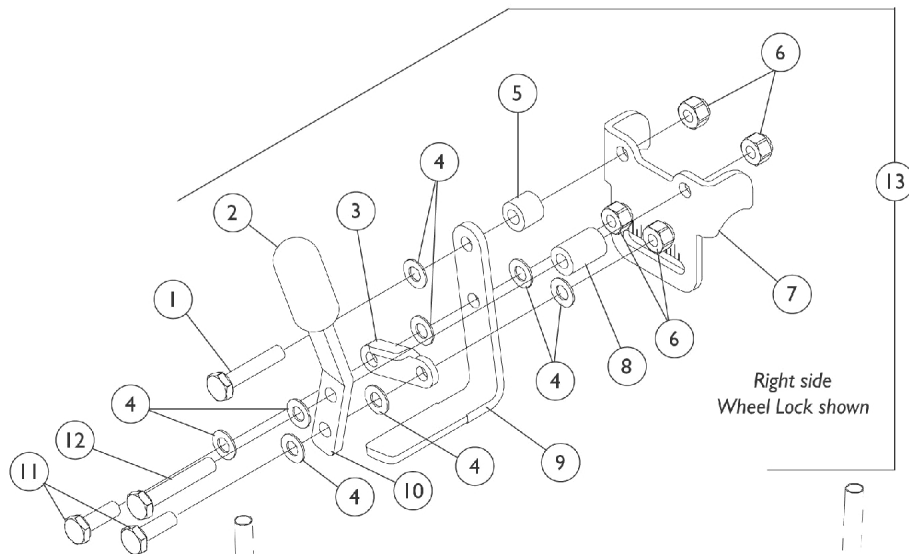


Wheel Lock Mounting Hardware
(Adult)

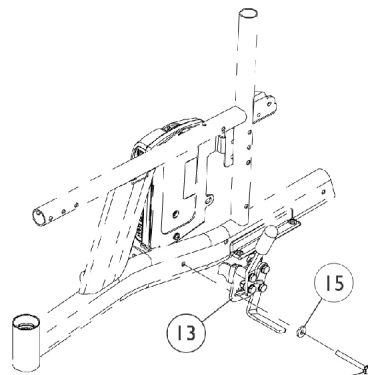

NOTE: Assemble left wheel lock to side frame using the 1/4-20 x 2-1/4" hex head cap screw from the controller mounting hardware.

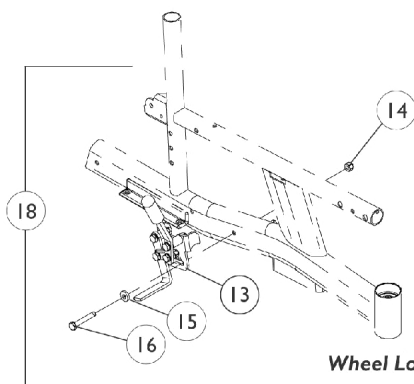

Wheel Lock Mounting Hardware
(Hemi)

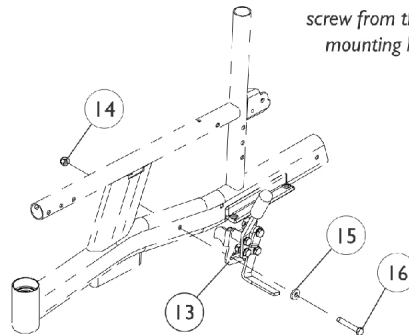

Accessories

Wheel Locks After 2/14/03

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
2		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
3		1018733-NLA	NLA - Link, Stop, Black	1	1
4		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	8
5		1106326-NLA	NLA - Spacer Nylon (17/64 x 1/2 x 7/16")	1	1
6		7930333	Locknut (1/4-20)	25	4
7		1057912-NLA	NLA - Bracket, Black	1	1
8		1106325-NLA	NLA - Spacer Nylon (17/64 x 1/2 x 3/4")	1	1
9		1021405-NLA	NLA - Shoe, Wheel Lock	1	1
10		1028414	Handle, Wheel Lock, Black - Right/ Left	1	1
11		7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	2
12		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	1
13	A	1106300	Wheel Lock Assembly without Attaching Hardware Right	1	1
13	A	1106301	Wheel Lock Assembly without Attaching Hardware Left	1	1
14	C	1025736	Locknut (1/4-20)	1	1
15		1018739	Washer (1/4 x 5/8 x 3/32")	1	2
16	C	1037610	Screw, Hex Head (1/4-20 x 1-5/8")	1	1
17	B	1115552-NLA	NLA - Wheel Lock with Attaching Hardware, Adult	2	1
18	B,C	1115553-NLA	NLA - Wheel Lock with Attaching Hardware, Hemi	2	1
	D,E	1116397-NLA	NLA - Wheel Lock Attaching Hardware	2	1
NOTE: A - Includes items 1-12 B - Includes items 13-16 C - Hemi frame mounting hardware includes quantity of 2 each for items 14 and 16 D - Not Shown E - Includes 2 each of items 14, 15 and 16					