
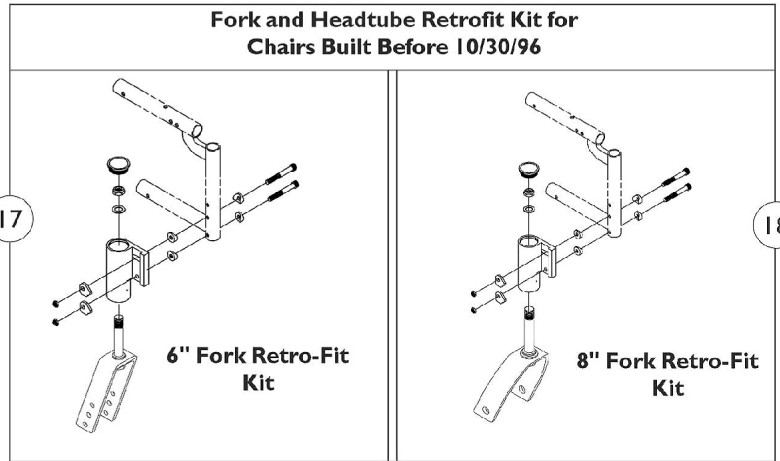
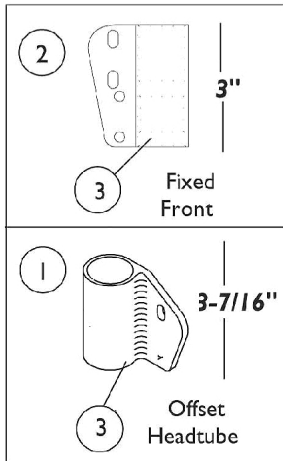
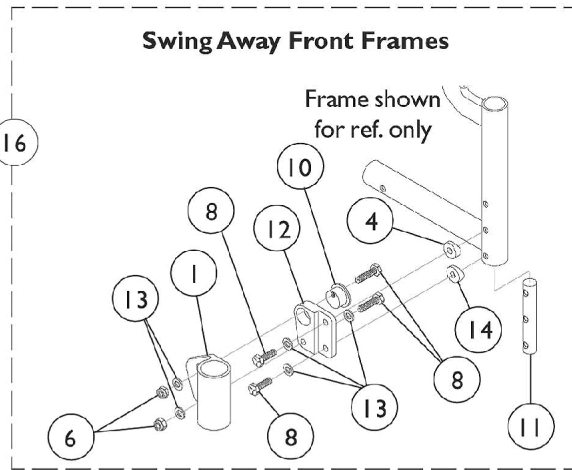


Caster Headtubes and Hardware

All MVP/Style and Xtra/Jr.
(Built Before 4/03)

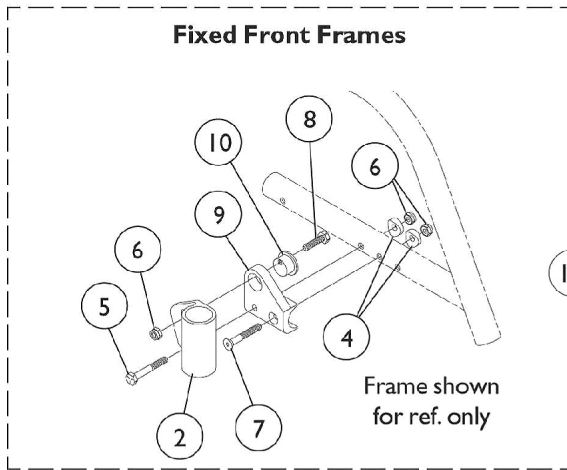

Caster Headtubes and Hardware
All MVP/Style and
Xtra/Jr.
(Built Before 4/03)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1040764-NLA	NLA - Headtube, Caster, Black - Right	1	1
1	A	1040765-NLA	NLA - Headtube, Caster, Black - Left	1	1
2		1076369-NLA	NLA - Headtube, Caster, Black - Right	1	1
2		1076370-NLA	NLA - Headtube, Caster, Black - Left	1	1
3		40117X021	Bearing (1/2" I.D. x 1-1/8" O.D.)	1	2
4		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
5		1055349	Screw, Hex Head (1/4-28 x 2")	1	4
6		1055366	Locknut (1/4-28)	1	6
7		1055332-NLA	NLA - Screw, Socket Flat Head (1/4 -28 x 1-3/4")	1	1
8	B	1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	2
8	C	1055296-NLA	NLA - Screw, Hex Head (1/4-28 x 1")	1	2
9		1040759-NLA	NLA - Bracket, Headtube Adjustment, Black - Right	1	1
9		1040760-NLA	NLA - Bracket, Headtube Adjustment, Black - Left	1	1
10		1040766	Cam, Headtube Adjustment	1	2
11		1056675-NLA	NLA - Nut, Barrel (1/4-28)	1	2
12		1040773	Bracket, Headtube Adjustment, Black - Right	1	1
12		1040774	Bracket, Headtube Adjustment, Black - Left	1	1
13		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
14		1009861	Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
15	D	1046147-NLA	NLA - Kit, Caster Headtubes w/ Hardware (Pair)	1	1
16	E	1042505-NLA	NLA - Headtube Assembly w/ Mounting Hardware, Black (Pair)	1	1
17	F	1065633-NLA	NLA - Kit, Fork Retro-Fit (6") (Pair)	1	1
18	F	1065634-NLA	NLA - Kit, Fork Retro-Fit (8") (Pair)	1	1

NOTE: A - Order Fork and Headtube Retrofit Kits for all Caster Headtubes on Chairs built Before 10/30/96.
B - Chairs with Transport Option
C - Chairs without Transport Option
D - Fixed Front Frame Chairs. Includes items 2, 4-10.
E - Swingaway Front Frame Chairs. Includes items 1,4,6,8, and 10-14.
F - Includes Forks, Headtubes, and All Attaching Hardware.