


Seat Frame Support and Mounting Hardware

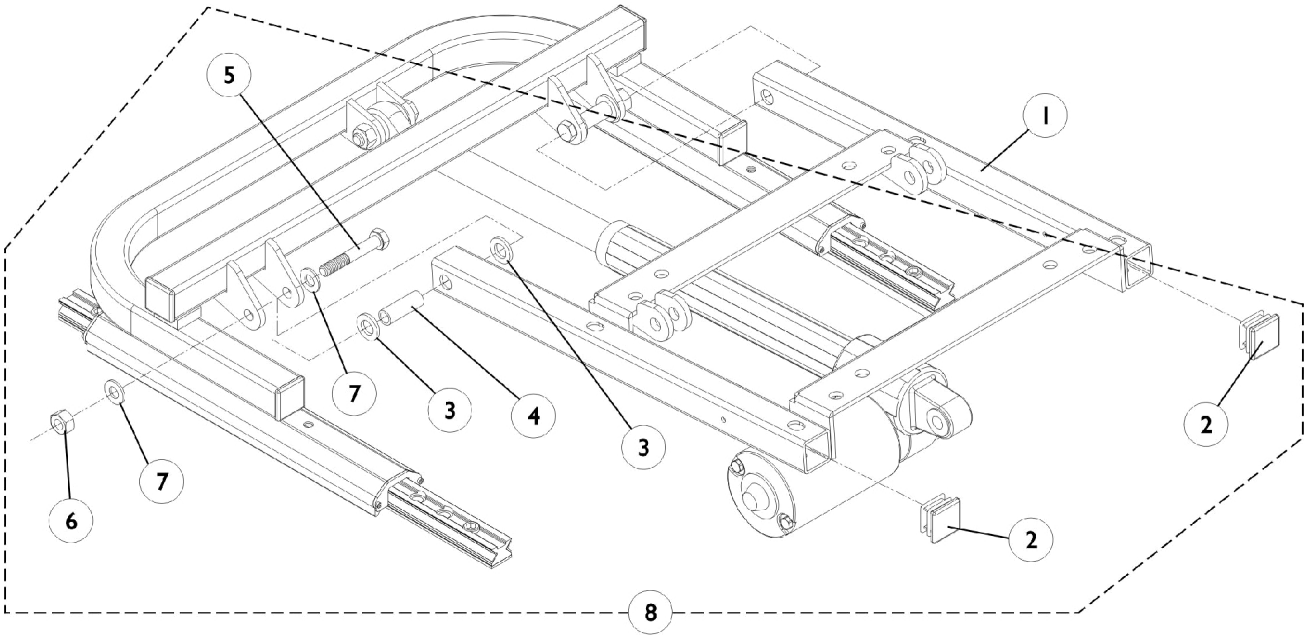

Seat Frame Support and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1087626-NLA	NLA - Support, Seat Frame	1	1
2		1071237	Plug Button, Square, Black (1")	1	2
3		0055800-NLA	NLA - Washer, Nylon (7/16 x 3/4 x 3/32")	1	4
4		0060300-NLA	NLA - Bushing, Bronze (5/16 x 7/16 x 1-1/4")	1	2
5		1070412-NLA	NLA - Screw, Hex Head (5/16-18 x 2-1/4")	1	2
6		1026158	Locknut (5/16-18)	1	2
7		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	4
8	A	1106540-NLA	NLA - Hardware, Seat Frame Support Mounting	1	1

NOTE: A - Includes items 2-7