


# Frame Hardware

All MVP/Style and Xtra/Jr.  
(Built Before 4/03)

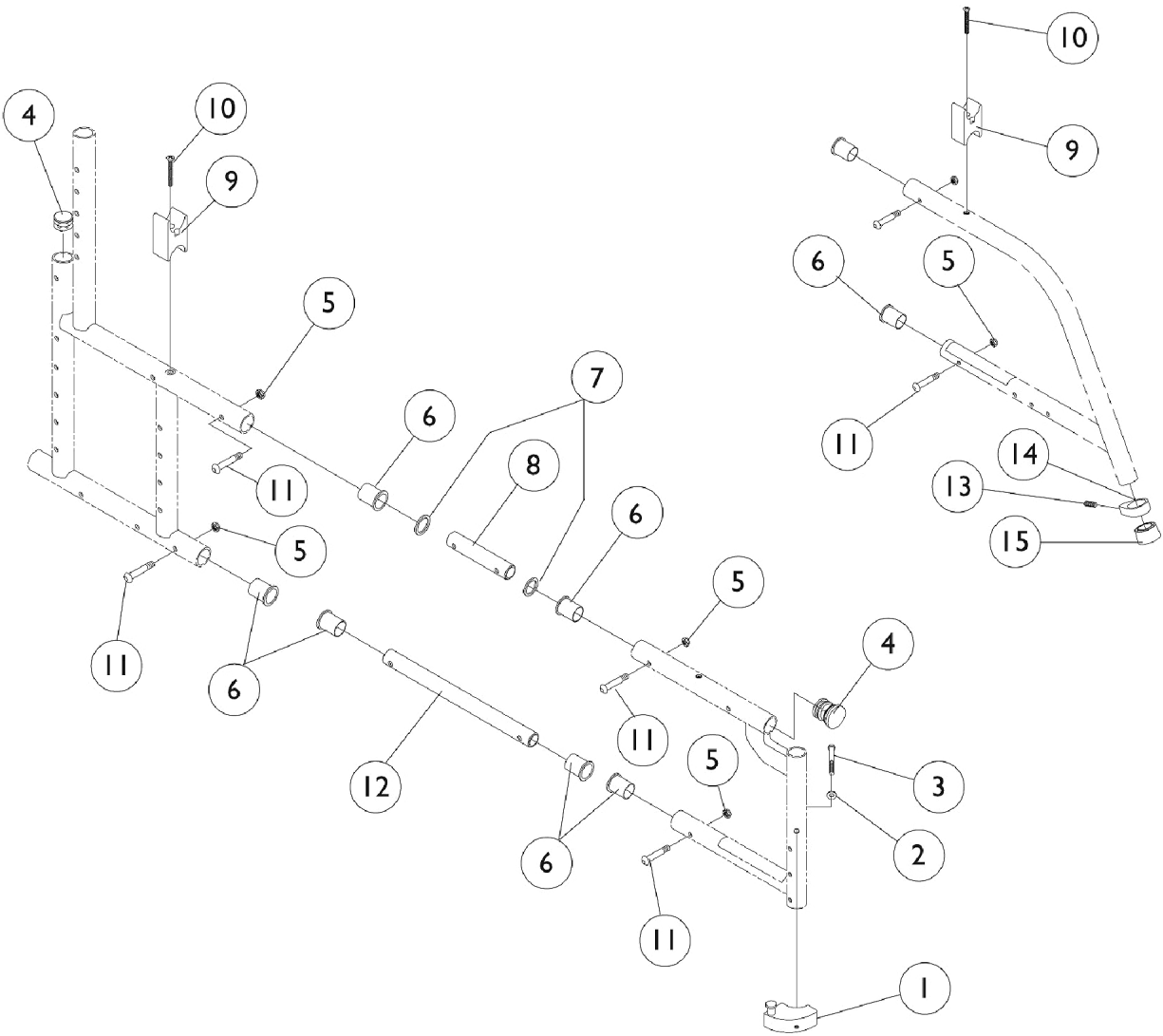

**Frame Hardware**  
**All MVP/Style and Xtra/Jr.**  
**(Built Before 4/03)**

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1054571	Kit, Sector Block - Right	1	1
1	A	1054572	Kit, Sector Block - Left	1	1
2		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
3		1052957	Screw, Hex Head w/ Patch (1/4-20 x 2")	1	1
4		1056328	Plug Button, Black (1")	1	2
5		1065362	Locknut (1/4-28)	1	8
6		1024786	Bushing, Pivot	1	6
7		1026888	Washer, Nylon (3/4 x 1 x 1/32")	1	2
8		1024787-NLA	NLA - Tube, Insert, Upper	1	1
9		1007541	Clamp, Seat	1	2
10		1025703-NLA	NLA - Screw, Phillips Oval Head (#10-32 x 1")	1	2
11		1055309	Screw, Socket Button Head (1/4-28 x 1-1/4")	1	4
12		1024788	Tube, Insert, Lower	1	1
13		1026911-NLA	NLA - Screw, Socket Set Cone Point (1/4-20 x 1/2")	1	1
14		1024749	Clamp	1	1
15		15680X013	Anti-Rattle (1")	1	1

NOTE: A - Includes items 2 and 3.