

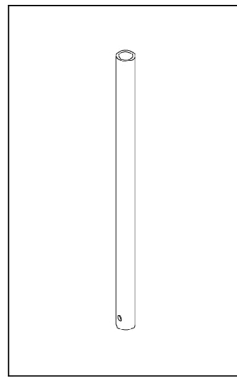



Canes - Back

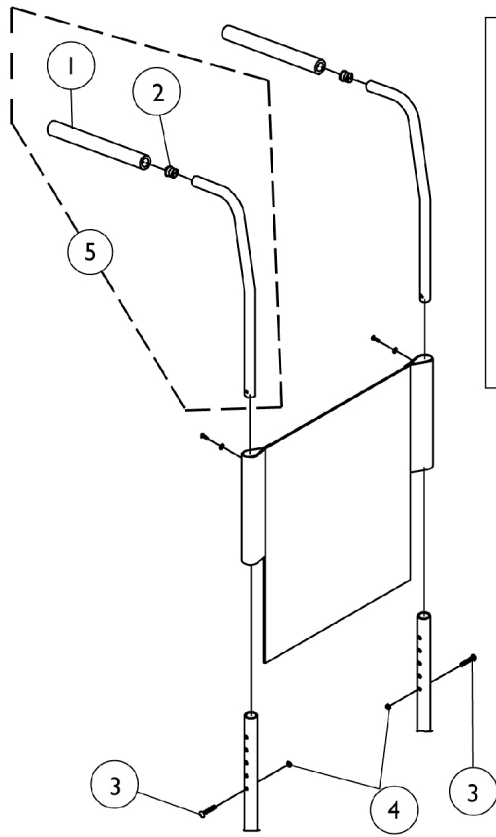

Sportster Cane

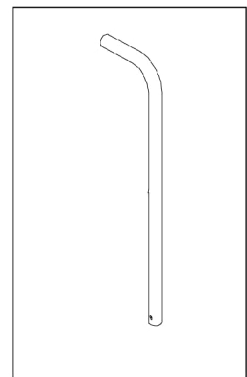

Straight Cane with Push Handle

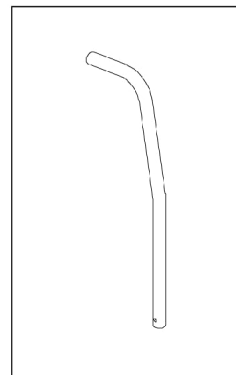

10° Bend Cane with Push Handle

Canes - Back

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1032246	Grip, Cane	1	1
2		1027097	Plug Button, Black (3/4")	1	1
3		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
4		1025195	Locknut (1/4-20)	1	1
5		See Chart	#5 Back Canes w/ Grip - Jr.	1	1
5		See Chart	#5 Back Canes w/ Grip - MVP/Style built After 7/4/00	1	1
5		See Chart	#5 Back Canes w/ Grip - Xtra and (MVP/Style Before 7/5/00)	1	1

#5 Back Canes w/ Grip

Xtra and (MVP/Style Before 7/5/00)

Back Cane Description	Part Number
Sportster Cane, 10-14" Back Height	1028785-NLA
Straight Cane w/ Push Handle, 12-16" Back Height	1027850
Straight Cane with Push Handle, 14-18" Back Height	1044031
10 Degree Bend Cane w/ Push Handle, 14-18" Back Height	1088495
10 Degree Bend Cane w/ Push Handle, 20" Back (Non-Adjustable)	1088493-NLA

#5 Back Canes w/ Grip

MVP/Style built After 7/4/00

Back Cane Description	Part Number
Sportster Cane, 10-14" Back Height	1094588
Straight Cane w/ Push Handle, 12-16" Back Height	1094925
Straight Cane w/ Push Handle, 14-18" Back Height	1094926
10 Degree Bend Cane w/ Push Handle, 14-18" Back Height	1094586
10 Degree Bend Cane w/ Push Handle, 20" Back (Non-Adjustable)	1094587

Canes - Back

#5 Back Canes w/ Grip

Jr.

Back Cane Description	Part Number
10 Degree Bend w/ Push Handle, 10-14" & 14-18" Back Height	1052789-NLA
Straight Cane w/o Grip, 20" High	1059249