


# Two Point Arm Attaching Hardware

Front Arm Sockets

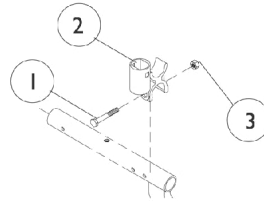

<b>Xtra, Jr.</b>	<b>MVP, Style Chairs before 7/5/00</b>	<b>MVP, Style Chairs after 7/4/00</b>
<p data-bbox="233 1100 464 1136"><b>20 Conventional</b></p>	<p data-bbox="618 1100 849 1136"><b>22 Conventional</b></p>	<p data-bbox="1040 1100 1271 1136"><b>24 Conventional</b></p>
<p data-bbox="258 1646 472 1682"><b>21 Space Saver</b></p>	<p data-bbox="631 1646 846 1682"><b>23 Space Saver</b></p>	<p data-bbox="1040 1646 1255 1682"><b>25 Space Saver</b></p>

## Two Point Arm Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1055298	Screw, Hex Head (1/4-28 x 1-5/8")	1	6
2		1036485	Socket, Arm	1	2
3		1065362	Locknut (1/4-28)	1	8
4		1055351-NLA	NLA - Screw, Hex Head (1/4-28 x 2-1/2")	1	2
5		1015856	Socket, Armrest	1	2
6		1068015-NLA	NLA - Support, Post, Arm	1	2
7	F	1114604-NLA	NLA - Bracket, Arm Mounting, Black	1	2
8		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
9		1015426	Spacer, Coved Cam - Right	1	1
9		1015427	Spacer, Coved Cam - Left	1	2
10		1028553-NLA	NLA - Spacer (1/4 x 3/8 x 3/4")	1	2
11		1055309	Screw, Socket Button Head (1/4-28 x 1-1/4")	1	2
12		1025765	Locknut (1/4-28)	1	2
13		1036473-NLA	NLA - Bracket, Armrest, Inside Mounting, Black - Right (14" Deep)	1	1
13		1036472-NLA	NLA - Bracket, Armrest, Inside Mounting, Black - Left (14" D)	1	1
13		1036474	Bracket, Armrest, Outside Mounting, Black - Right (14" D)	1	1
13		1036475-NLA	NLA - Bracket, Armrest, Outside Mounting, Black - Left (14" Deep)	1	1
13		1036469-NLA	NLA - Bracket, Armrest, Inside Mounting, Black - Right (15 - 18" Deep )	1	1
13		1036468-NLA	NLA - Bracket, Armrest, Inside Mounting, Black - Left (15 - 18" Deep)	1	1
13		1036470-NLA	NLA - Bracket, Armrest, Outside Mounting, Black - Right (15 - 18" Deep)	1	1
13		1036471-NLA	NLA - Bracket, Armrest, Outside Mounting, Black - Left (15 - 18" Deep)	1	1
14		1066261	Bracket, Armrest, Black	1	1
15		1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	3
16		1036435	Washer, Nylon (1/4 x 7/8 x 1/32")	1	2
17		1087219	Spacer (1/4 x 3/8 x 1")	1	2
18		1026900	Washer (1/4 x 23/32 x 1/16")	1	4
19		1083099	Screw, Socket Button Head (1/4-28 x 1-1/2")	1	2
20	A,E	1069807	Hardware, Arm Mounting, Conventional Arm (Pair)	2	1
21	A,E	1036796	Hardware, Arm Mounting, Space Saver Arm (Pair)	2	1
22	B,E	1079517-NLA	NLA - Hardware, Arm Mounting, Conv. Arm (Pair)	2	1
22	C,E	1079519-NLA	NLA - Hardware, Arm Mounting, Conv. Arm (Pair)	2	1
23	B,E	1079516-NLA	NLA - Hardware, Arm Mounting, S.S. Arm (Pair)	2	1
23	C,E	1079518-NLA	NLA - Hardware, Arm Mounting, S.S. Arm (Pair)	2	1
24	D,E	1101043-NLA	NLA - Hardware, Arm Mounting, Conv. Arm (Pair)	2	1
25	D,E	1094944-NLA	NLA - Hardware, Arm Mounting, S.S. Arm (Pair)	2	1

## Two Point Arm Attaching Hardware

NOTE: A - Xtra, Jr.  
B - MVP, Style Before 7/5/00 (14" Deep)  
C - MVP, Style Before 7/5/00 (15-18" Deep)  
D - MVP, Style After 7/4/00 (14-18" Deep)  
E - Includes Front Arm Sockets w/ Hardware (Items 1-3)  
F - Part No Longer Available individually. See Part # 1069807 for replacement.