


Fixed Height Arms

After 11/92

Fixed Height arms are only available in desk length

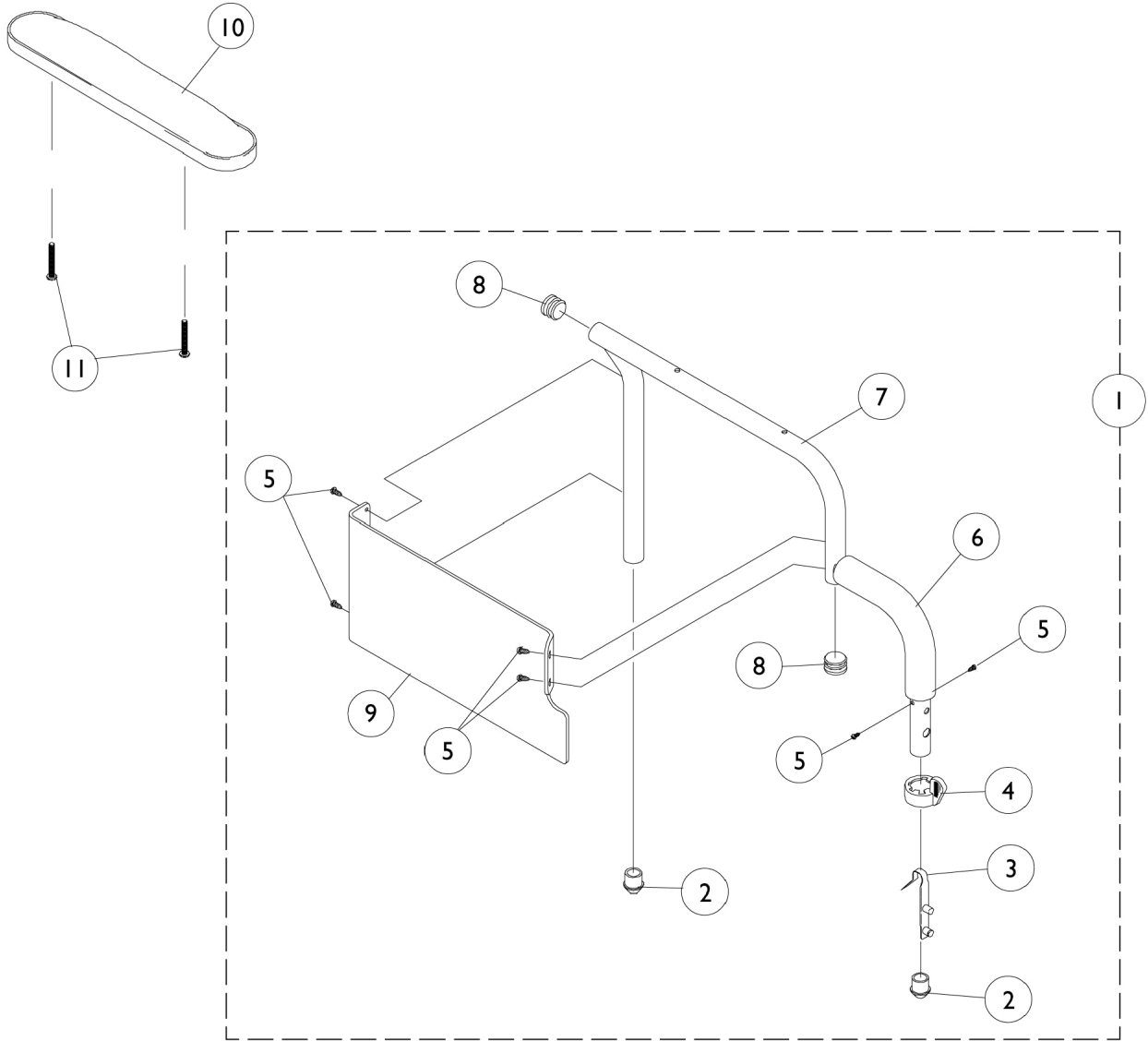

Fixed Height Arms

After 11/92

Fixed Height arms are only available in desk length

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E,I	8881044965-NL A	NLA - Arm, Space-Saver, Desk Length - Right	1	1
1	E,I	8881044966-NL A	NLA - Arm, Space-Saver, Desk Length - Left	1	1
1	F,I,J	8881056986	Arm Assembly, Space-Saver Desk Length w/o Arm Pad, Color - Right	1	1
1	F,I,J	8881056987	Arm Assembly, Space-Saver Desk Length w/o Arm Pad, Color - Left	1	1
1	B,G	8881044815-NL A	NLA - Arm Assembly, Conventional Desk Length w/o Arm Pad, Color - Right	1	1
1	B,G	8881044816-NL A	NLA - Arm Assembly, Conventional Desk Length w/o Arm Pad, Color - Left	1	1
2		1036484	Plug, Armrest, Black (3/4") (18GA)	1	2
3		1018225	Spring, Torsion with Button Lock	1	2
4		1015450	Handle, Quad Release	1	2
5		1020022	Screw, Phillips Pan Head Tap (#6-20 x 3/8")	1	6
6		1037719	Grip, Hand, Foam (6 1/2")	1	1
7		8881038589-NL A	NLA - Tube, Arm, Space-Saver, Desk Length, Color - Right	1	1
7		8881038590-NL A	NLA - Tube, Arm, Space-Saver, Desk Length, Color - Left	1	1
7		8881038587-NL A	NLA - Tube, Arm, Conventional, Desk Length, Color - Right	1	1
7		8881038588-NL A	NLA - Tube, Arm, Conventional, Desk Length, Color - Left	1	1
8		1027097	Plug Button, Black (3/4")	1	2
9	C,H	1117365	Kit, Clothing Guard, Conventional, Desk Length w/ Screws and Decals (Pair)	2	1
9	D,H	1123307	Kit, Clothing Guard, Space Saver, Desk Length w/ Screws and Decals (Pair)	2	1
10	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
11		40183X002	Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2

NOTE: A - Specify U67 upholstery color when ordering.
 B - Due to Clothing Guard change, it is recommend (for chairs built Before 10/1/02) when only one Arm is requested the Clothing Guard Kit (p/n 1117365) be ordered also so that the arms will match.
 C - Conventional Arms
 D - Space-Saver Arms
 E - Xtra
 F - MVP and Style
 G - MVP, Style, and Xtra
 H - Clothing Guards are not sold individually. Kit includes Clothing Guards, Screws, and Decals.
 I - Due to Clothing Guard change, it is recommend (for chairs built Before 11/25/03) when only one Arm is requested the Clothing Guard Kit (p/n 1123307) be ordered also so that the arms will match.
 J - Must order the Hand Grip separately