
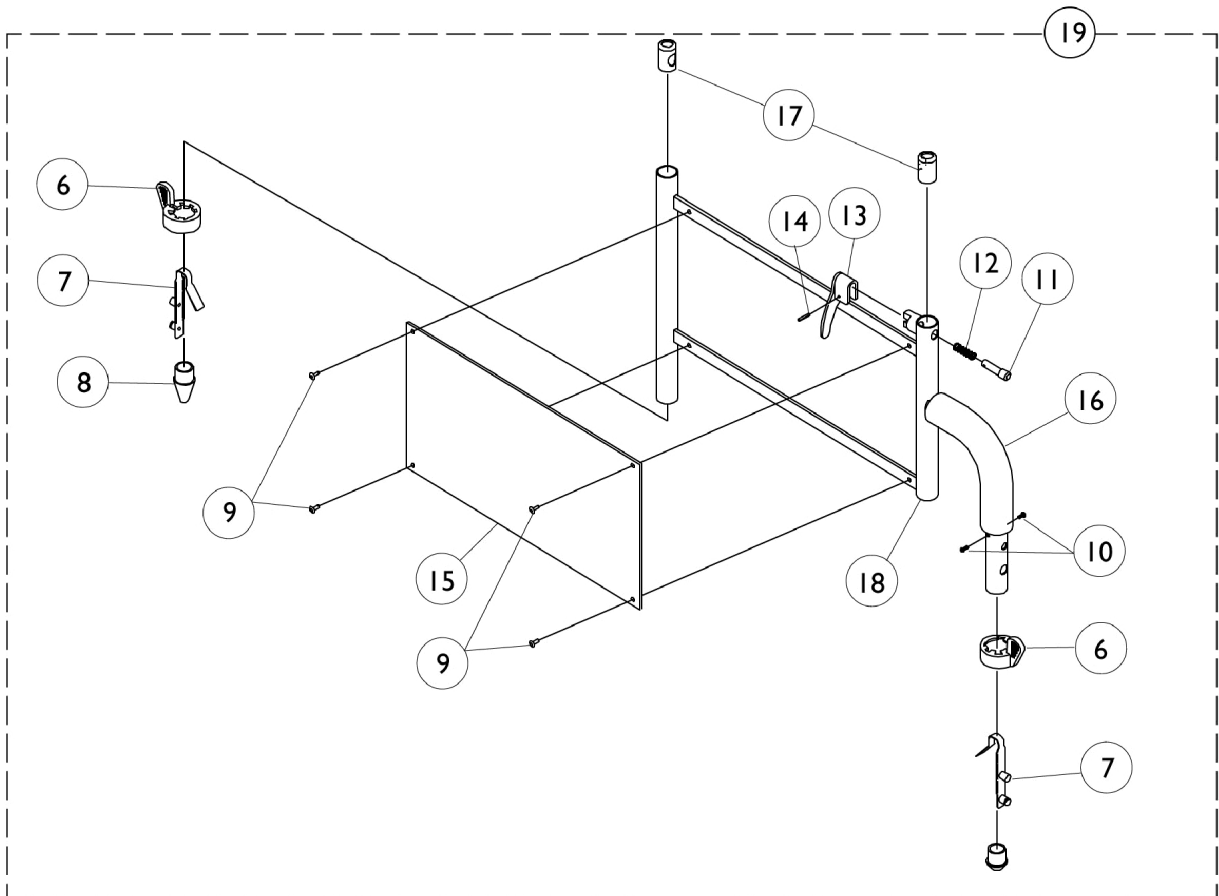


Adjustable Height Arms

After 11/92

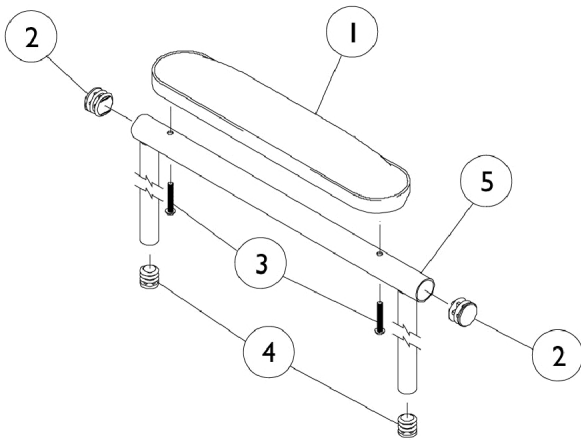

Adjustable Height Arms After 11/92

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
1	A	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1
2		1027095	Plug Button, Black (7/8")	1	2
3		1111714	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	1
4		1035614	Plug Button, Black (5/8")	1	2
5		1038470	Arm Tube Assembly, Conventional w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
5		1068576	Arm Tube Assembly, Conventional w/ Plug Buttons, Full Length, Black - Right/Left	1	1
5		1038462	Armrest Assembly, Space Saver Desk Length, Black - Right	1	1
			Tube, Armrest, Desk, Black - Right	1	1
			Plug Button, Black (7/8")	1	2
			Plug Button, Black (5/8")	1	3
5		1038463	Armrest Assembly, Space Saver Desk Length, Black - Left	1	1
			Tube, Armrest, Desk, Black - Left	1	1
			Plug Button, Black (7/8")	1	2
			Plug Button, Black (5/8")	1	3
5		1038466	Armrest Assembly, Space Saver Full Length, Black - Right	1	1
			Tube, Arm, Black - Right	1	1
			Plug Button, Black (7/8")	1	2
			Plug Button, Black (5/8")	1	3
5		1038467	Armrest Assembly, Space Saver Full Length, Black - Left	1	1
			Tube, Arm, Black - Left	1	1
			Plug Button, Black (7/8")	1	2
			Plug Button, Black (5/8")	1	3
6		1015450	Handle, Quad Release	1	2
7		1018225	Spring, Torsion with Button Lock	1	2
8		1020060	Plug, Arm	1	1
9		1028645	Rivet	1	4
10		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	2
11		47471X068	Pin, Locking	1	1
12		47471X074	Spring, Arm Lock	1	1
13		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
14		1079151-NLA	NLA - Pin, Roll (5/64 x 5/8")	1	1
15		1064129	Guard, Clothing	1	1
16		1037719	Grip, Hand, Foam (6 1/2")	1	1
17		1036150	Anti-Rattle	1	1
18		1032303-NLA	NLA - Arm Base Assembly, w/o Hardware, Black	1	1
19		1036783	Arm Base Assembly, Conventional Adjustable Height - Right	1	1
19		1036784	Arm Base Assembly, Conventional Adjustable Height - Left	1	1
19	B,D	1037779-NLA	NLA - Arm Base, Space-Saver, Black - Right	1	1

Adjustable Height Arms After 11/92

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
19	B,D	1037780-NLA	NLA - Arm Base, Space-Saver, Black - Left	1	1
19	C	1048323-NLA	NLA - Arm Base Assembly, Space Saver - Right	1	1
19	C	1048324-NLA	NLA - Arm Base Assembly, Space Saver - Left	1	1
NOTE: A - Specify U67 upholstery color when ordering. B - MVP, Xtra chairs C - Style chairs D - Arm Bases are for 18-21" deep chairs. For 22" deep chairs, arm bases must be ordered through specials.					