
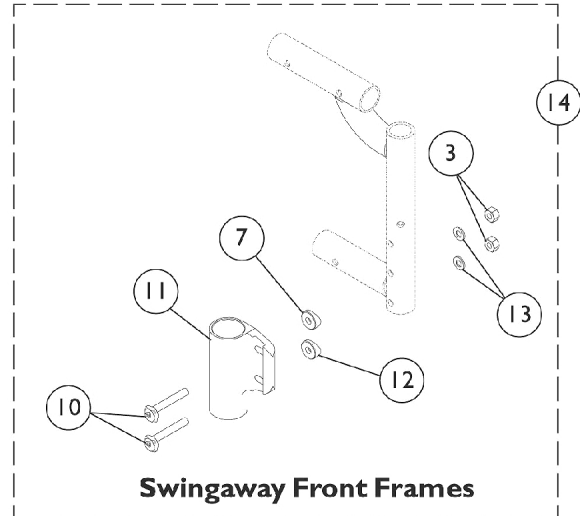


Caster Headtubes and Hardware

(Chairs Built After 3/03)

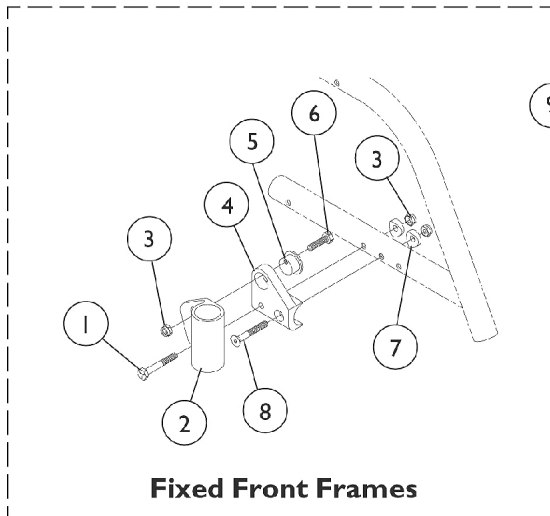

Caster Headtubes and Hardware (Chairs Built After 3/03)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1055349	Screw, Hex Head (1/4-28 x 2")	1	2
2		1076369-NLA	NLA - Headtube, Caster, Black - Right	1	1
2		1076370-NLA	NLA - Headtube, Caster, Black - Left	1	1
3		1055366	Locknut (1/4-28)	1	4
4		1040759-NLA	NLA - Bracket, Headtube Adjustment, Black - Right	1	1
4		1040760-NLA	NLA - Bracket, Headtube Adjustment, Black - Left	1	1
5		1040766	Cam, Headtube Adjustment	1	2
6	A	1055296-NLA	NLA - Screw, Hex Head (1/4-28 x 1")	1	2
6	B	1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	2
7		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
8		1055332-NLA	NLA - Screw, Socket Flat Head (1/4 -28 x 1-3/4")	1	2
9	C	1046147-NLA	NLA - Kit, Caster Headtubes w/ Hardware (Pair)	2	1
10		1113501	Screw, Eccentric (1/4-28 x 1-11/16")	1	4
11		1113576	Headtube, Caster, Black - Right	1	1
11		1113577	Headtube, Caster, Black - Left	1	1
	E	40117X021	Bearing (1/2" I.D. x 1-1/8" O.D.)	1	2
12		1009861	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
13		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
14	D	1114704-NLA	NLA - Kit, Caster Headtubes w/ Hardware (Pair)	2	1
NOTE: A - Chairs without Transport Option B - Chairs with Transport Option C - Fixed Front Frames. Includes items 1-8. D - Swingaway Front Frames. Includes items 3,7, & 10-13. E - Not Shown - Caster Headtube Bearings					