
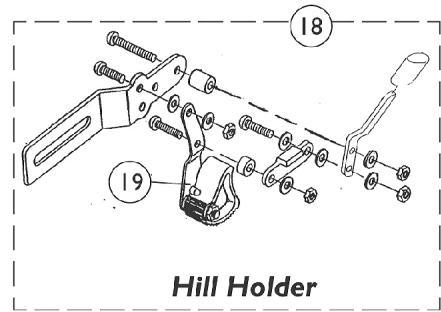


# High Mount Wheel Locks

## Pull to Lock - Style

### Hill Holder - MVP / Xtra / Jr. / Style

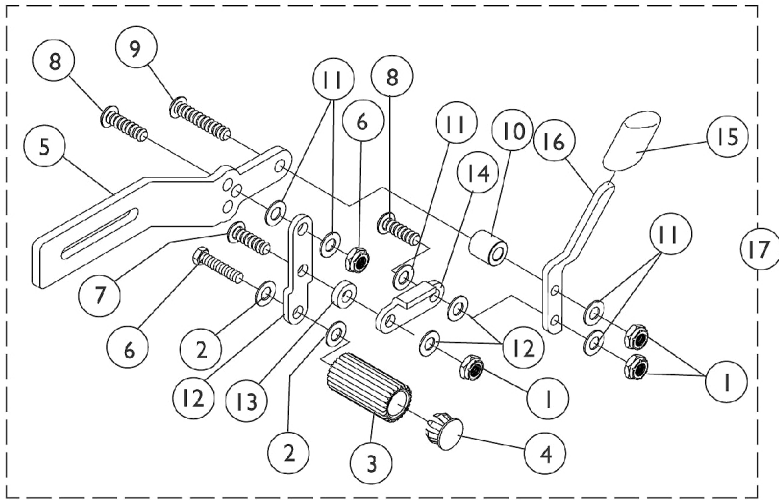

# High Mount Wheel Locks

## Pull to Lock - Style

### Hill Holder - MVP / Xtra / Jr. / Style

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1025765	Locknut (1/4-28)	1	2
2		1026108	Lockwasher (1/4 x 15/32 x 1/64")	1	2
3		1020361-NLA	NLA - Shoe, Wheel Lock	1	1
4		1025761-NLA	NLA - Plug Button, Black (5/8")	1	1
6		1025708-NLA	NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
7		1055305-NLA	NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	2
8		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	1
9		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
10		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
11		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
12		1014372	Link, Off-Set Shoe, Black	1	1
13		1026104-NLA	NLA - Spacer	1	1
14		1018733-NLA	NLA - Link, Stop, Black	1	1
14		1018727-NLA	NLA - Link, Stop, Black	1	1
15		1006490	Tip, Plastic, Black	1	1
16		1038344-NLA	NLA - Handle, Wheel Lock, Black - Right	1	1
16		1038345-NLA	NLA - Handle, Wheel Lock, Black - Left	1	1
16		1031780-NLA	NLA - Handle, Pull/ Lock, Black - Right	1	1
16		1031781-NLA	NLA - Handle, Pull/ Lock, Black - Left	1	1
17	A	1026715-NLA	NLA - Wheel Lock Assembly, Pull to Lock - Right	1	1
17	A	1026716-NLA	NLA - Wheel Lock Assembly, Pull to Lock - Left	1	1
18	B	1026712	Hill Holder Wheel Lock Assembly - Right	1	1
			Bracket, Black	1	1
			Screw, Socket Button Head (1/4-28 x 5/8")	1	2
			NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
			Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
			NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
			Locknut (1/4-28)	1	4
			NLA - Link, Stop, Black	1	1
			NLA - Handle, Wheel Lock, Black - Right	1	1
			Tip, Plastic, Black	1	1
			NLA - Spacer	1	1
19			Hill Holder Assembly - Right	1	1
			NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	1
18	B	1026713	Hill Holder Wheel Lock Assembly - Left	1	1
			Bracket, Black	1	1
			Screw, Socket Button Head (1/4-28 x 5/8")	1	2
			NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
			Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
			NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
			Locknut (1/4-28)	1	4
			Tip, Plastic, Black	1	1
			NLA - Handle, Wheel Lock, Black - Left	1	1
			NLA - Link, Stop, Black	1	1

# High Mount Wheel Locks

## Pull to Lock - Style

### Hill Holder - MVP / Xtra / Jr. / Style

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
19	C	1026352	NLA - Spacer	1	1
			NLA - Hill Holder Assembly - Left	1	1
			NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	1
			Kit, Wheel Lock Hardware	2	1
			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	2
			Washer (1/4 x 5/8 x 3/32")	1	2
			NLA - Clamp, Half, Threaded	1	2
			NLA - Clamp, Half, Non-Threaded	1	2
			Washer, Nylon (1/4 x 7/16 x 5/64")	1	2
			Screw, Socket Head (1/4-20 x 1-1/4")	1	2
NOTE: A - Includes items 1 - 16. Item #5 is no longer available. B - Includes item 19. C - Not shown.					