


# Legrest Support Push Rod Assembly

## AT5544P and Genius

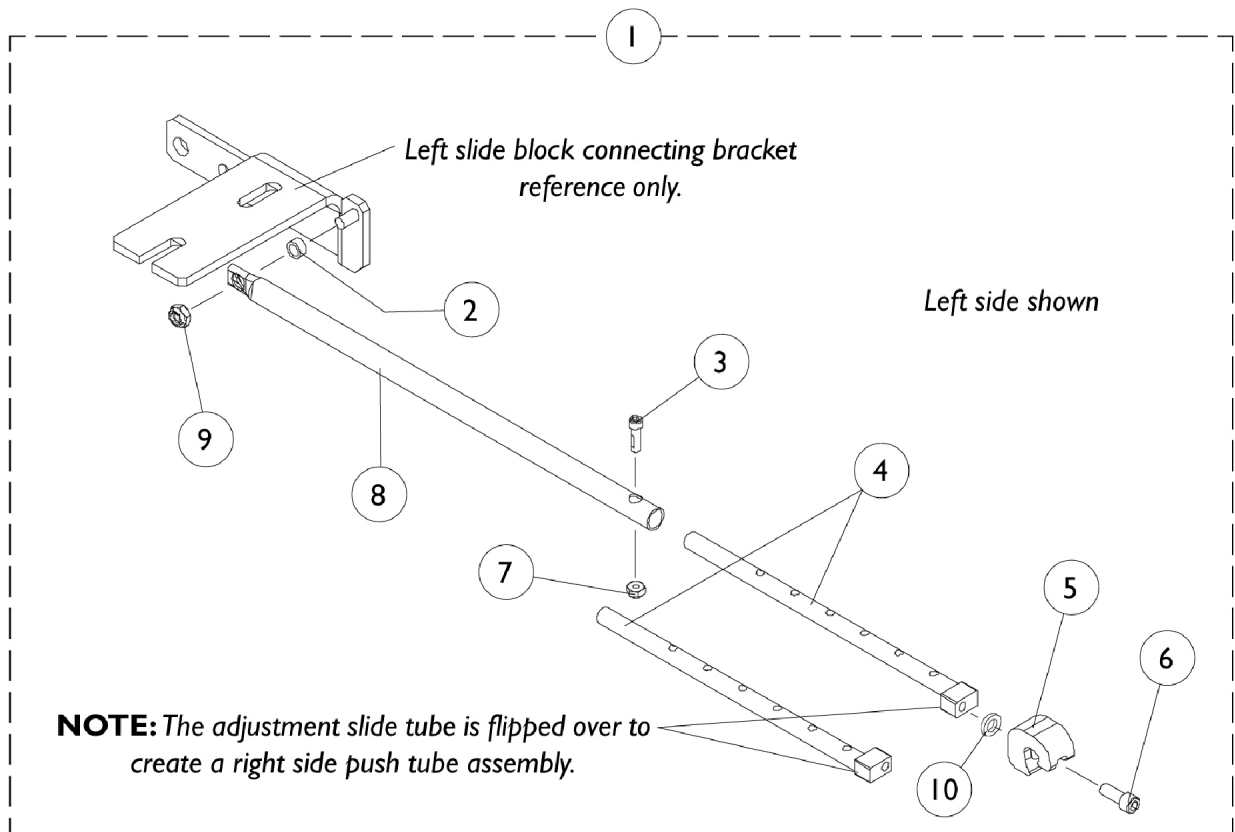

## Legrest Support Push Rod Assembly AT5544P and Genius

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1086966-NLA	NLA - Legrest Support Push Rod Assembly, Black - Right (16 - 19" D)	1	1
1		1086967-NLA	NLA - Legrest Support Push Rod Assembly, Black - Left (16 - 19" D)	1	1
1		1086968-NLA	NLA - Legrest Support Push Rod Assembly, Black - Right (20 - 22" D)	1	1
1		1086969-NLA	NLA - Legrest Support Push Rod Assembly, Black - Left (20 - 22" D)	1	1
2		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	1
3		1091871-NLA	NLA - Screw, Socket Head (#8-32 x 5/8")	1	1
4		1086958-NLA	NLA - Tube, Adjustable Slide, Black	1	1
5		1082922-NLA	NLA - Receptacle, Legrest	1	1
6		1007219	Screw, Socket Head (1/4-20 x 5/8")	1	1
7		1056098	Locknut (#8-32)	1	1
8		1086962-NLA	NLA - Rod, Push, Legrest Support, Black (16 - 19" D)	1	1
8		1086963-NLA	NLA - Rod, Push, Legrest Support, Black (20 - 22" D)	1	1
9		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
10		1013971	Washer, Nylon (1/4 x 7/16 x 5/64")	1	1