


Armrest Assembly

Recline Only (2GR) and Tilt/Recline (2GTR)

Back Cane Shown for
Reference Only

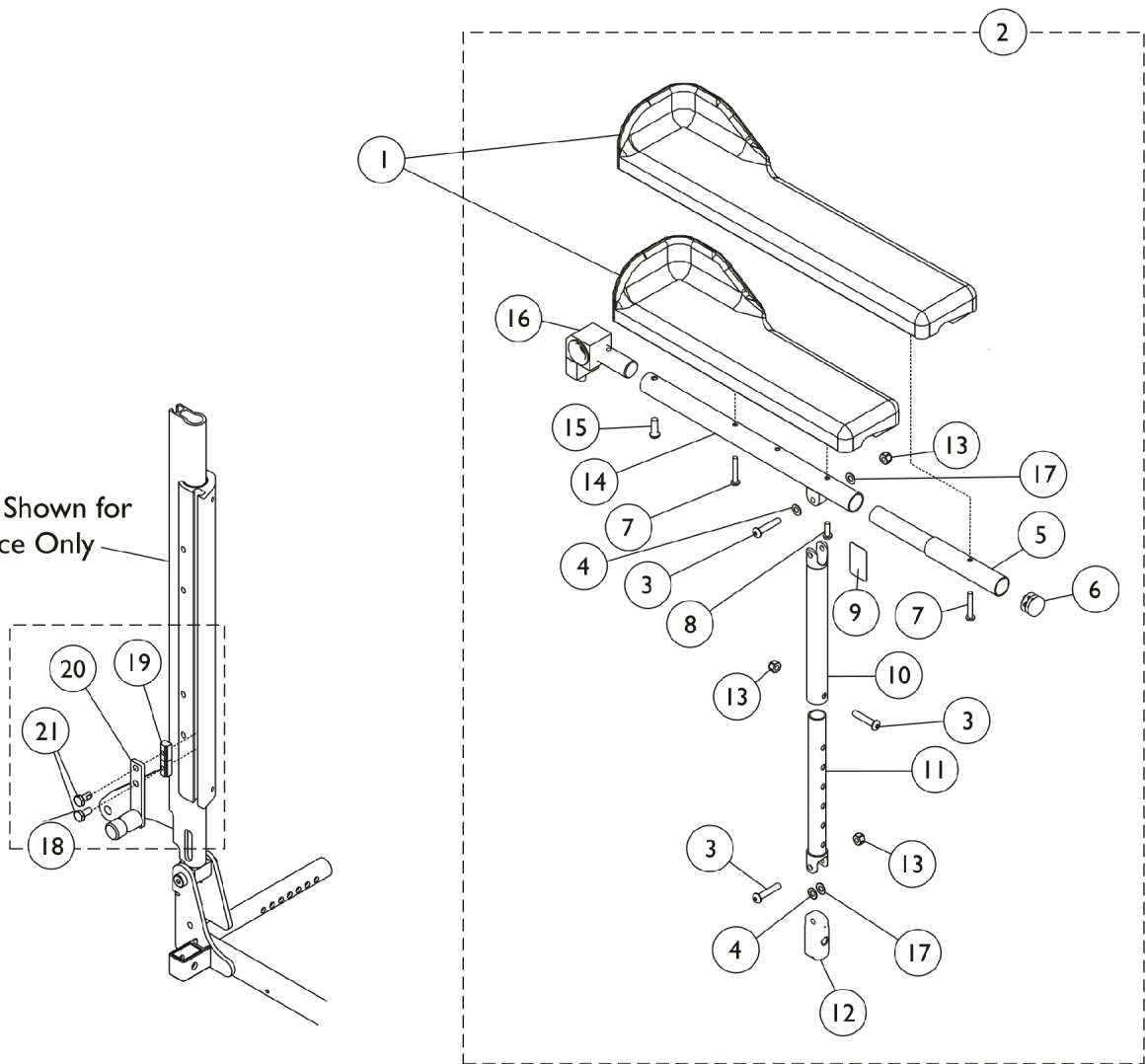

Armrest Assembly

Recline Only (2GR) and Tilt/Recline (2GTR)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1082910	Pad, Arm w/ Elbow Stop, Desk Length, Urethane, Black - Right (13" L x 3" W)	1	1
1		1082911	Pad, Arm w/ Elbow Stop, Desk Length, Urethane, Black - Left (13" L x 3" W)	1	1
1		1090190	Kit, Arm Pad w/ Screws and Elbow Stop, Desk Length, Urethane, Black (13" L x 3" W) (Pair)	2	1
1		1082912	Pad, Arm w/ Elbow Stop, Full Length, Urethane, Black - Right (17" L x 3" W)	1	1
1		1082913	Pad, Arm w/ Elbow Stop, Full Length, Urethane, Black - Left (17" L x 3" W)	1	1
1		1090191	Kit, Arm Pad w/ Screws and Elbow Stop, Full Length, Urethane, Black (17" L x 3" W) (Pair)	2	1
2	A,G	1086992	Arm Assembly w/o Arm Pad, Desk Length, Black - Right	1	1
2	A,G	1086993	Arm Assembly w/o Arm Pad, Desk Length, Black - Left	1	1
2	B,G	1087012	Arm Assembly w/o Arm Pad, Full Length, Black - Right	1	1
2	B,G	1087013	Arm Assembly w/o Arm Pad, Full Length, Black - Left	1	1
3		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	3
4		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	2
5		0028202	Tube, Extension, Full Length	1	1
6		1027095	Plug Button, Black (7/8")	1	1
7		1107155	Screw, Phillips Pan Head Tap (#10-32 x 1-1/4")	1	2
8	D	1028755	Screw, Phillips Pan Head (#10-32 x 5/8")	1	1
8	E	1107155	Screw, Phillips Pan Head Tap (#10-32 x 1-1/4")	1	1
9		1079203	Decal, Warning Pinch Point	1	1
10		1115392	Tube, Arm, Support, Black	1	1
11		1115393-NLA	NLA - Tube, Slide, Adjustable, Black	1	1
12		1082842	Plug, Arm, Black	1	1
13		7930333	Locknut (1/4-20)	25	3
14		1082927	Tube, Arm, Black	1	1
15		1091870	Screw, Socket Button Head w/ Patch (1/4-20 x 3/4")	1	1
16	H	1095437	Arm Pivot Assembly	1	1
17		1165484-NLA	NLA - Washer, Nylon (1/4 x 1/2 x 1/32")	1	2
18	C	1086990-NLA	NLA - Hardware, Arm Attaching	1	1
19		1082699	T-Nut (1/4-20)	1	2
20		1086932-NLA	NLA - Pivot, Back Arm	1	1
21		1078214	Screw, Hex Head w/ Patch (1/4-20 x 1/2")	1	2
	F	1119268-NLA	NLA - Kit, Slide Tube w/ Arm Support (Pair)	2	1

Armrest Assembly

Recline Only (2GR) and Tilt/Recline (2GTR)

NOTE: A - Includes Items 3,4,6, and 9-17
B - Includes Items 3-6 and 8-17
C - Includes Items 18-21
D - Used for full length arms only
E - Used for desk length arms only
F - Not Shown. Assembly includes 2 ea. of item numbers 9, 10 and 11
G - When placing an order for item 2, you must specify as a note "For Tarsys 2GTR seating" or "For Formula TRE seating" because item 16 Arm Pivot Assembly Pin faces down for the Tarsys 2GTR and faces up for the Formula TRE powered seating.
H - The Arm Pivot Assembly Pin faces down for the Tarsys 2GTR and faces up for the Formula TRE powered seating