
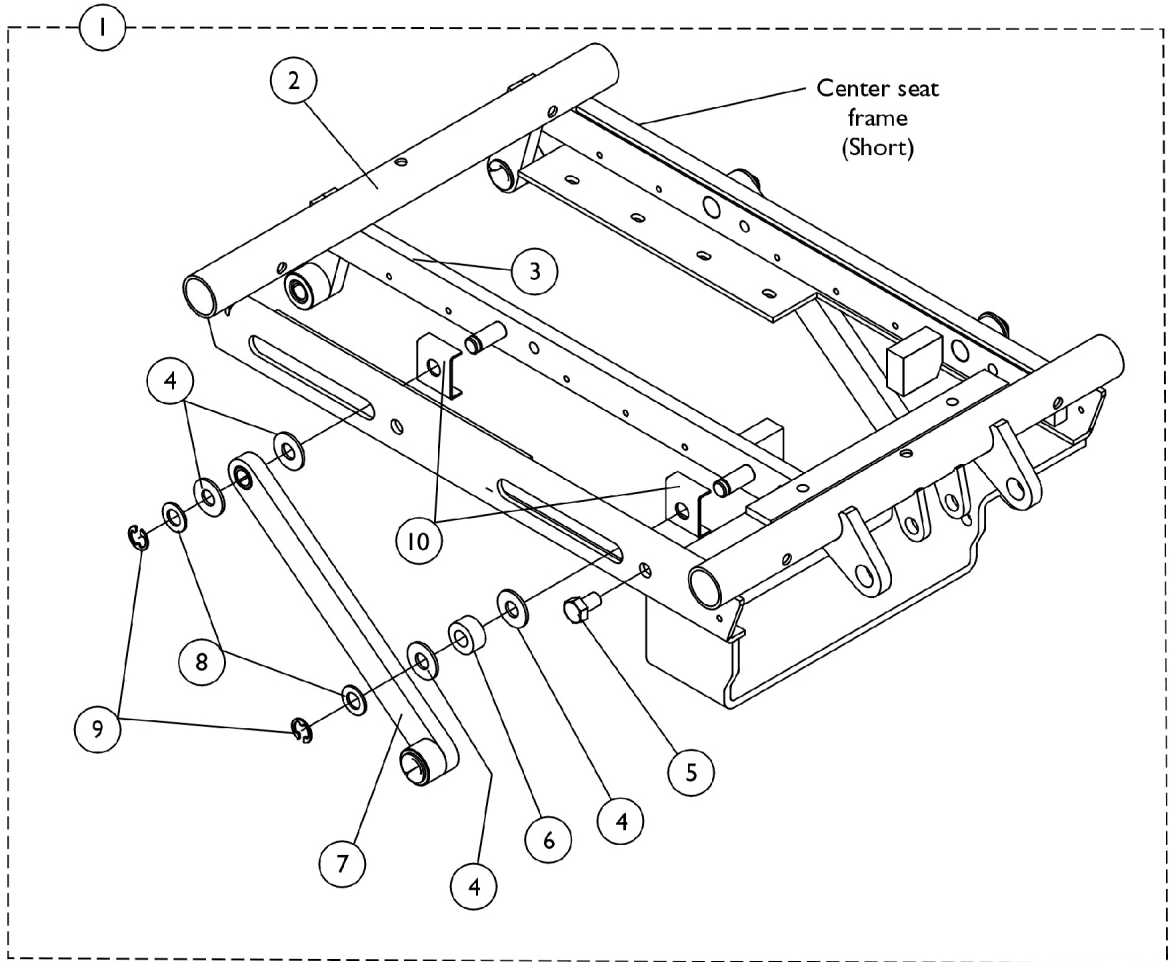


# Center Seat Frame Assembly

Tilt Only (2GT)

Center seat  
frame  
assembly  
(Long)

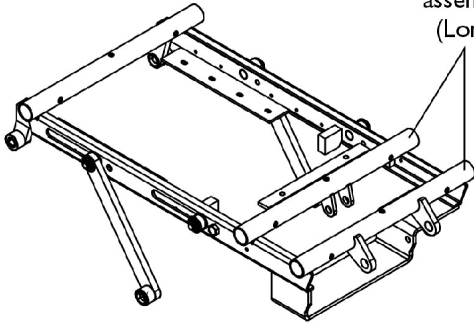

## Center Seat Frame Assembly Tilt Only (2GT)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1115859-NLA	NLA - Center Seat Frame Assembly, Short (16 - 19" D)	1	1
1	A	1115858-NLA	NLA - Center Seat Frame Assembly, Long (20 - 22" D)	1	1
2		1113613-NLA	NLA - Center Seat Frame, Short (16 - 19" D)	1	1
2	B	1113614-NLA	NLA - Center Seat Frame, Long (20 - 22" D)	1	1
3		1086590-NLA	NLA - Tilt Pivot Assembly, Right	1	1
3		1086591-NLA	NLA - Tilt Pivot Assembly, Left	1	1
4		1012371	Washer, Nylon (3/8 x 7/8 x 1/16")	1	8
5		1061922-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1/2")	1	2
6		1082876-NLA	NLA - Spacer (3/8 x 3/4 x 3/8")	1	2
7		1086595-NLA	NLA - Link, Tilt, Black	1	2
8		1065214	Washer, Flat (3/8 x 11/16 x 1/16")	1	4
9		1086930-NLA	NLA - Ring, Retaining (3/8")	1	4
10		1091502-NLA	NLA - Guide, Tilt Rail	1	4

NOTE: A - Includes Items 2-10  
B - No longer available individually. See Item 1, Part # 1115858 for replacement.