


Back Cane (7/8" Round-Optional) and Spreader Bar Assembly

Recline Only (2GR) and Tilt/Recline (2GTR)

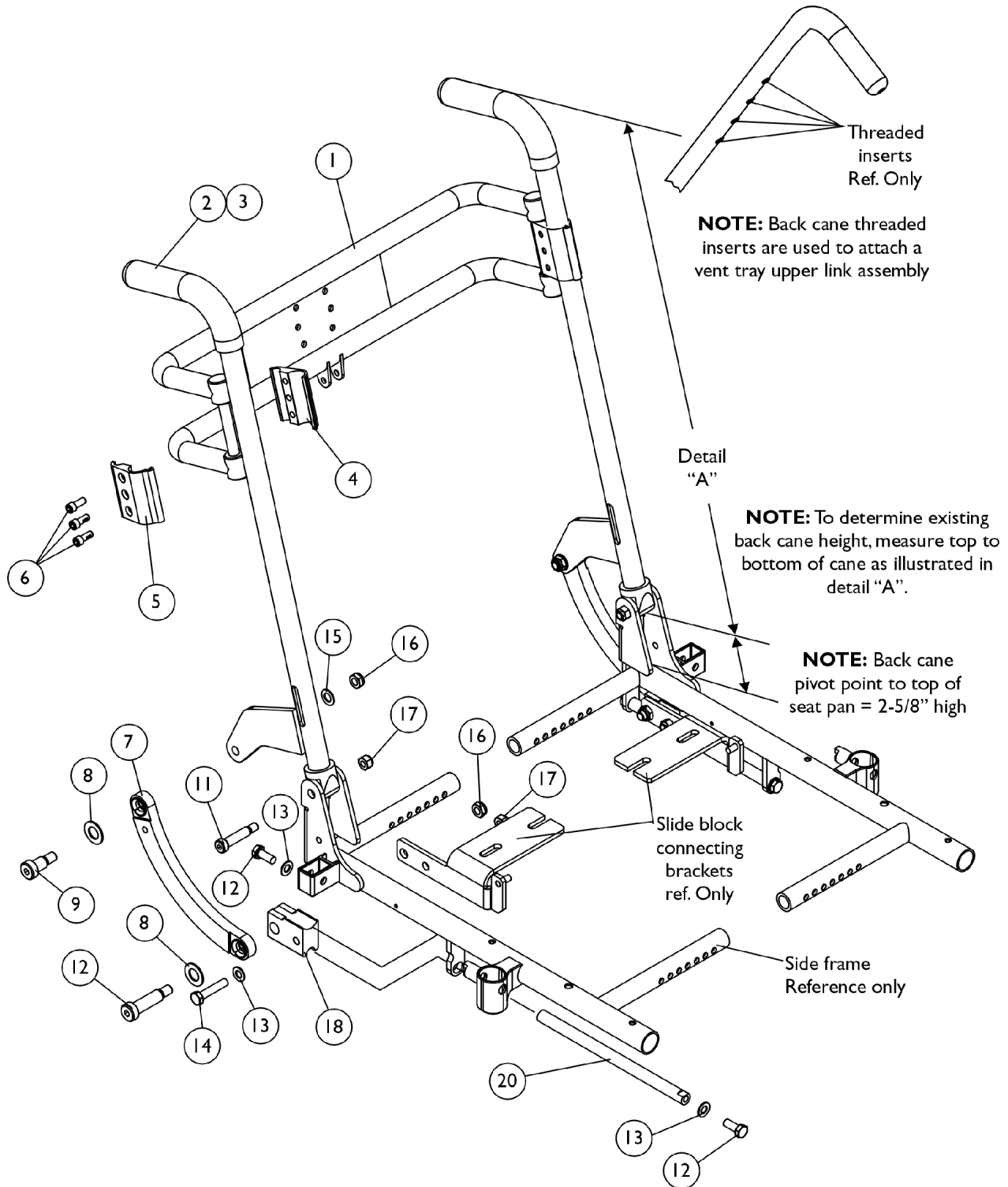

Back Cane (7/8" Round-Optional) and Spreader Bar Assembly

Recline Only (2GR) and Tilt/Recline (2GTR)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Spreader Bar	1	1
2		See Chart	#2 Back Canes	1	2
3		1086460	Grip, Foam, Hand (4-1/2" L)	1	2
4		0190722	Half Clamp, Un-Threaded Holes (7/8" Dia. Back Canes)	1	2
5		0190822	Half Clamp, Threaded Holes (7/8" Dia. Back Canes)	1	2
6		1007219	Screw, Socket Head (1/4-20 x 5/8")	1	6
7		1086587-NLA	NLA - Link, Curved, Black	1	2
8		1031673-NLA	NLA - Washer, Nylon (35/64 x 1 x 1/16")	1	4
9		1086323-NLA	NLA - Screw, Shoulder Hex Socket Head (1/2 x 5/8", 3/8-16)	1	2
10		1021851-NLA	NLA - Bolt, Socket Head Shoulder (1/2 x 1-1/2")	1	2
11		1062604	Screw, Shoulder Hex Socket (3/8 x 1-1/4", 5/16-18)	1	2
12		1039860	Screw, Hex Head w/ Patch (5/16-18 x 3/4")	1	4
13		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	6
14		1070398-NLA	NLA - Screw, Hex Head (5/16-18 x 1-1/2")	1	2
15		1065214	Washer, Flat (3/8 x 11/16 x 1/16")	1	2
16		40363X000	Hex Nut (3/8"-16)	1	4
17		1026158	Locknut (5/16-18)	1	4
18		1082725-NLA	NLA - Block, Slide	1	2

Item #1 Spreader Bar

16" Wide	17" Wide	18" Wide	19" Wide	20" Wide	21" Wide	22" Wide
0013602	0013702	0013802	0013902	0014002	0014102	0014202

Item #2 Back Cane with Grip

Back Height

Side	20-21" High	22-23" High	24-25" High	26" High
Right	1103008-NLA	1104726-NLA	1104728-NLA	1104730-NLA
Left	1103009-NLA	1104727-NLA	1104729-NLA	1104731-NLA