


Vent Tray Mounting Hardware For Round Back Canes

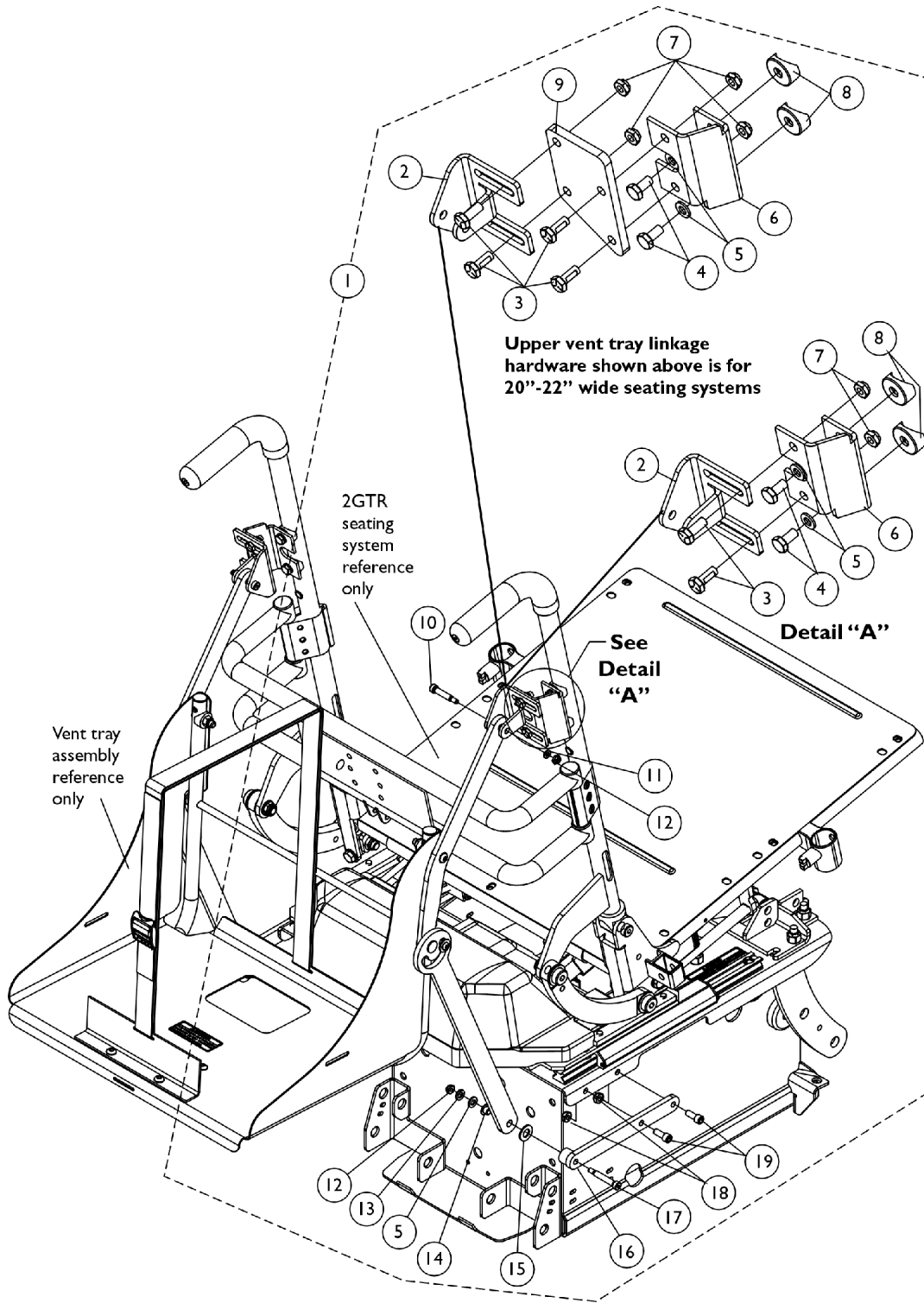

Vent Tray Mounting Hardware For Round Back Canes

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1121258-NLA	NLA - Hardware, Vent Tray Mounting for 16"-19" Wide Seats with Round Canes	2	1
1	B	1121259-NLA	NLA - Hardware, Vent Tray Mounting for 20"-22" Wide Seats with Round Canes	2	1
2		1097191-NLA	NLA - Bracket, Upper Link, Black	1	2
3	C	1121024	Screw, Hex Head w/ Patch (1/4-20 x 5/8")	1	4
4		1078214	Screw, Hex Head w/ Patch (1/4-20 x 1/2")	1	4
5		1026907	Washer (1/4 x 1/2 x 1/16")	1	6
6		1113638-NLA	NLA - Bracket, Vent-Connect Upper, Black - Right	1	1
6		1121097	Bracket, Vent-Connect Upper, Black - Left	1	1
7	C	1025195	Locknut (1/4-20)	1	4
8		1082908-NLA	NLA - Spacer, Coved (5/16 x 29/32 x 7/16")	1	4
9		1121021-NLA	NLA - Bracket, Seat Vent, Black (20 - 22" W)	1	2
10		1098289-NLA	NLA - Screw, Shoulder Hex Socket Head (1/4 x 1", #10-24)	1	2
11		1095360-NLA	NLA - Washer, Steel (7/8 x 3/8 x 1/16")	1	2
12		1026035-NLA	NLA - Locknut (#10-24)	1	4
13		1007453	Lockwasher, Split (1/4")	1	2
14		1097179-NLA	NLA - Bushing, Flanged	1	2
15		1049527	Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
16		1115815-NLA	NLA - Bracket, Vent Tray Attachment, Black	1	2
17		1000212	Screw, Shoulder Hex Socket (1/4 x 1-1/4", #10-24)	1	2
18		60275X000	Locknut (1/4-20)	1	4
19		1110468-NLA	NLA - Screw, Socket Head (1/4-20 x 5/8")	1	4

NOTE: A - Includes items 2-8 and 10-19
 B - Includes items 2-19
 C - Quantity of 8 are used for 20"-22" wide seats