
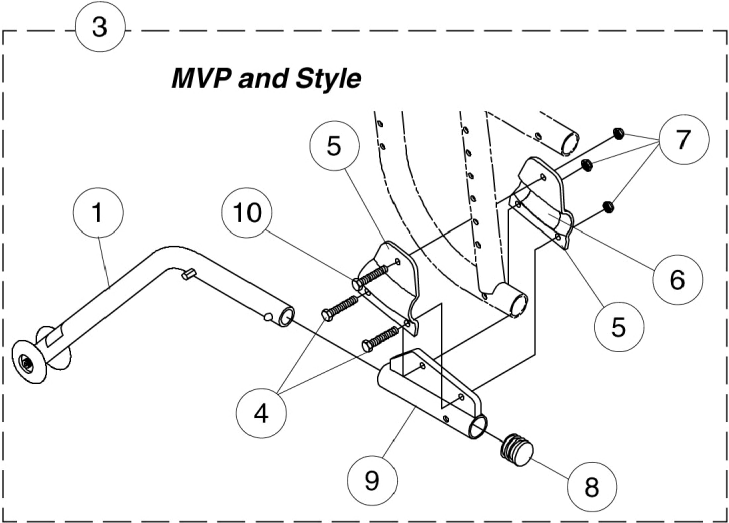


# Rear Anti-Tippers

*Xtra and Jr.*

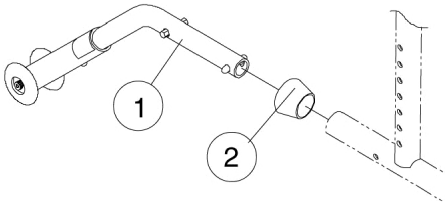

## Rear Anti-Tippers

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1050642-NLA	NLA - Anti-Tipper (Pair)	2	1
1	A,C	1050641	Anti-Tipper (Pair)	2	1
2		15680X013	Anti-Rattle (1")	1	1
3	D	1050643	Anti-Tipper (Pair)	2	1
4		1055296-NLA	NLA - Screw, Hex Head (1/4-28 x 1")	1	2
5		1028409-NLA	NLA - Clamp, Half	1	2
6		1029409	Strip, Protective	1	2
7		1065362	Locknut (1/4-28)	1	3
8		1056328	Plug Button, Black (1")	1	1
9		1026991-NLA	NLA - Bracket, Anti-Tipper, Black	1	1
10		1055346-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	1

NOTE: A - Anti-Rattle Collar not included.  
 B - Xtra  
 C - Jr.  
 D - MVP/Style