
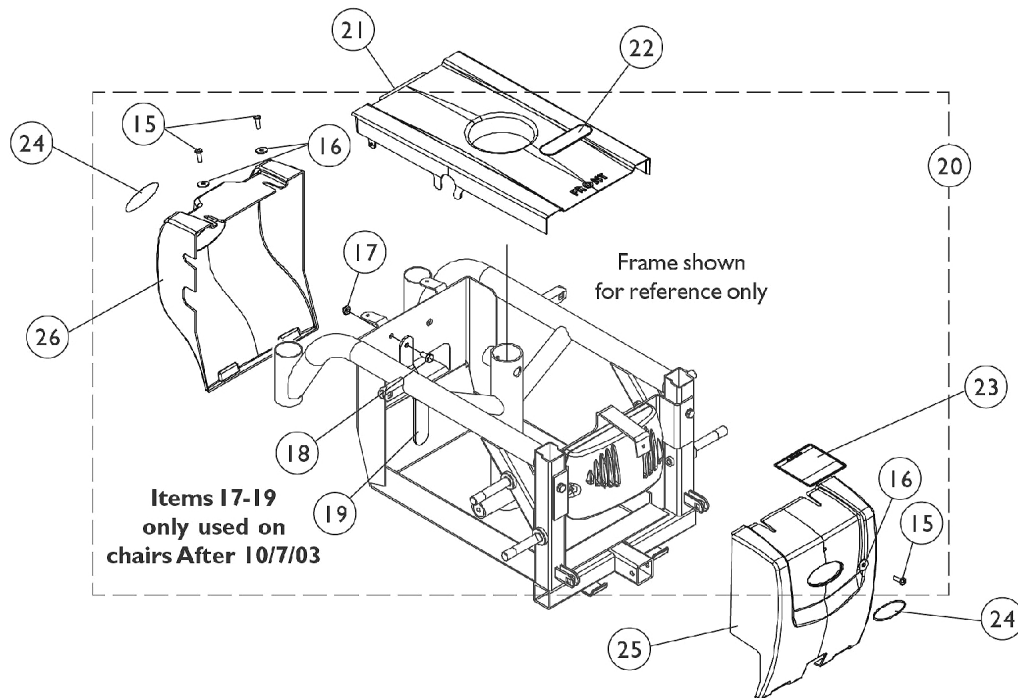


Shrouds, Battery Cover, and Attaching Hardware After 7/13/03

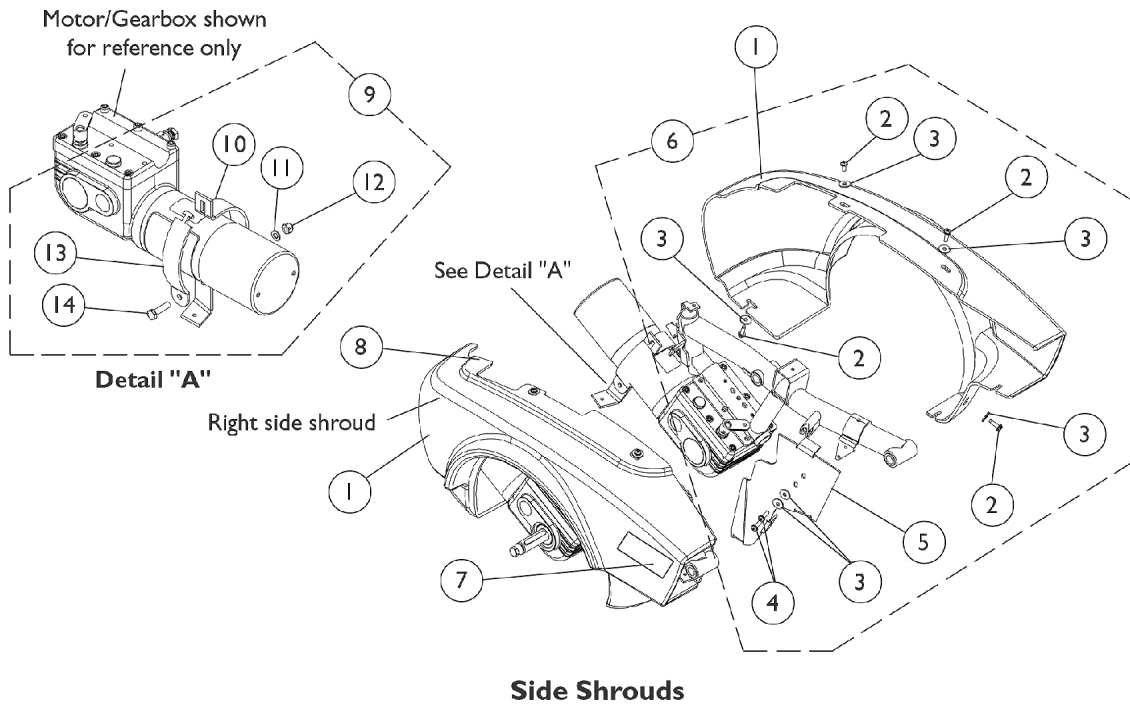

Shrouds, Battery Cover, and Attaching Hardware

After 7/13/03

#1 Dark Blue Pearl

ProntoM50 Right	ProntoM50 Left	ProntoM50 Pair	ProntoM51 Right	ProntoM51 Left	ProntoM51 Pair
1112813-NLA	1112814-NLA	1112819-NLA	1112815-NLA	1112816-NLA	1112821-NLA

#1 Candy Red Pearl

ProntoM50 Right	ProntoM50 Left	ProntoM50 Pair	ProntoM51 Right	ProntoM51 Left	ProntoM51 Pair
1112807-NLA	1112808-NLA	1112820-NLA	1112809-NLA	1112806-NLA	1112822-NLA

Shrouds, Battery Cover, and Attaching Hardware After 7/13/03

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	See Chart	#1 Side Shrouds Candy Red Pearl	1	1
1	E	See Chart	#1 Side Shrouds Dark Blue Pearl	1	1
2		1025451	Screw, Slotted Hex Washer Head (#10-32 x 5/8")	1	8
3		1070587	Washer, Flat (3/16 x 5/8 x 1/16")	1	12
4		1028755	Screw, Phillips Pan Head (#10-32 x 5/8")	1	4
5		1106366-NLA	NLA - Plate, Shroud -Right	1	1
5		1106367-NLA	NLA - Plate, Shroud -Left	1	1
6	B	1106343-NLA	NLA - Hardware, Shroud, Side (1 Pair)	2	1
7		1110604-NLA	NLA - Decal, Shroud - M50	1	1
7		1110605	Decal, Shroud - M51	1	1
8		1104797	Decal, Drive/Push	1	1
9	C	1108464-NLA	NLA - Kit, Shroud Clamp Hardware - Right	1	1
9	C	1108465-NLA	NLA - Kit, Shroud Clamp Hardware - Left	1	1
10		1109036-NLA	NLA - Bracket, Clamp Shroud Mounting, Black - Right	1	1
10		1109037-NLA	NLA - Bracket, Clamp Shroud Mounting, Black - Left	1	1
11		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
12		1025736	Locknut (1/4-20)	1	1
13		1109035-NLA	NLA - Clamp, Shroud	1	1
14		1025713-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	1
15		1028755	Screw, Phillips Pan Head (#10-32 x 5/8")	1	3
16		1070587	Washer, Flat (3/16 x 5/8 x 1/16")	1	3
17	A	1025195	NLA - Locknut (1/4-20)	1	1
18	A	1025752-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/2")	1	1
19	A	1122234-NLA	NLA - Bracket, Battery, Black	1	1
20	A,D	1119582-NLA	NLA - Hardware, Battery Shroud	1	1
21		1116388	Cover, Battery	1	1
22		1119281-NLA	NLA - Plate, Name, Patent Numbers	1	1
23		1122126-NLA	NLA - Label, Battery Warning	1	1
24		1095489	Decal, Invacare 3-D Logo (2-1/2" L)	1	2
25		1123236-NLA	NLA - Battery Shroud with Decals, Black - Front	1	1
26		1123241-NLA	NLA - Battery Shroud with Decal, Black - Rear	1	1

NOTE: A - Items 17-19 are only used on chairs built After 10/7/03.
B - Includes items 2-5
C - Includes items 10-14
D - Includes items 15-19
E - Includes decals, items 7 and 8