
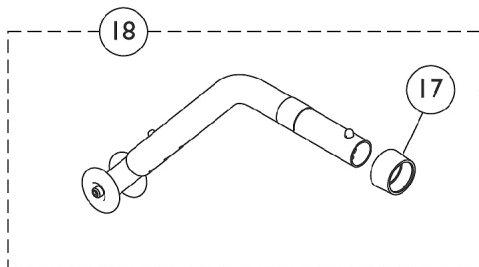


Frames, Frame Hardware, and Anti-Tips

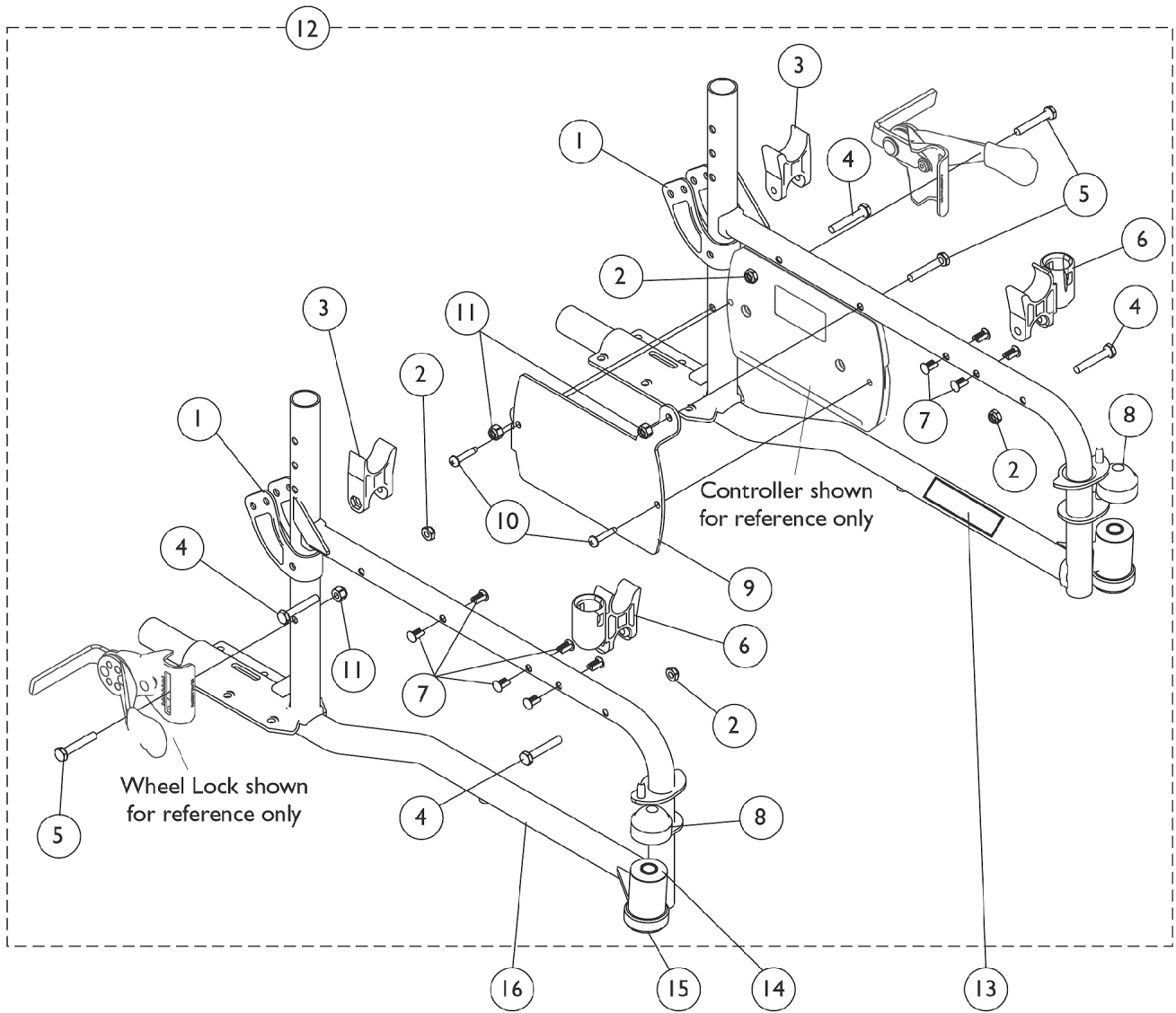

Frames, Frame Hardware, and Anti-Tips

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1066261	Bracket, Armrest, Black	1	2
2		60275X000	Locknut (1/4-20)	1	4
3		1107805	H-Block	1	2
4		16720X004	Screw, Hex Head (1/4-20 x 1-1/2")	1	4
5		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	3
6		1109938	H-Block with Socket Cup	1	2
7		1023450-NLA	NLA - Rivet, Ratchet	1	10
8		1001843	Cap, Head Tube, Black	1	2
9		1121213-NLA	NLA = Bracket, Controller Mounting, Black	1	1
10		1113439-NLA	NLA - Screw, Phillips Pan Head Tap (#10-32 x 1")	1	2
11		1025736	Locknut (1/4-20)	1	3
12	A	1121226-NLA	NLA - Hardware, Frame (Pair)	2	1
13		1121206-NLA	NLA - Decal, Frame - P9000XDT	1	2
14		1065530	Bearing, Flange (7/16")	1	2
15		1074930	Sleeve w/ Bearing, Black, Adult/Hemi	1	2
16	B	1121207-NLA	NLA - Frame Assembly with Bearings, Black - Right	1	1
16	B	1121208-NLA	NLA - Frame Assembly with Bearings, Black - Left	1	1
17		15680X013	Anti-Rattle (1")	1	2
18	C	1061112	Anti-Tipper, For Chairs with 6" Fork & Caster - 1360A (Pair)	2	1

NOTE: A - Includes items 1-11
B - Includes items 14 and 15
C - Includes 2 Anti-Tippers and Instruction Sheet