


Crossbraces and Hardware

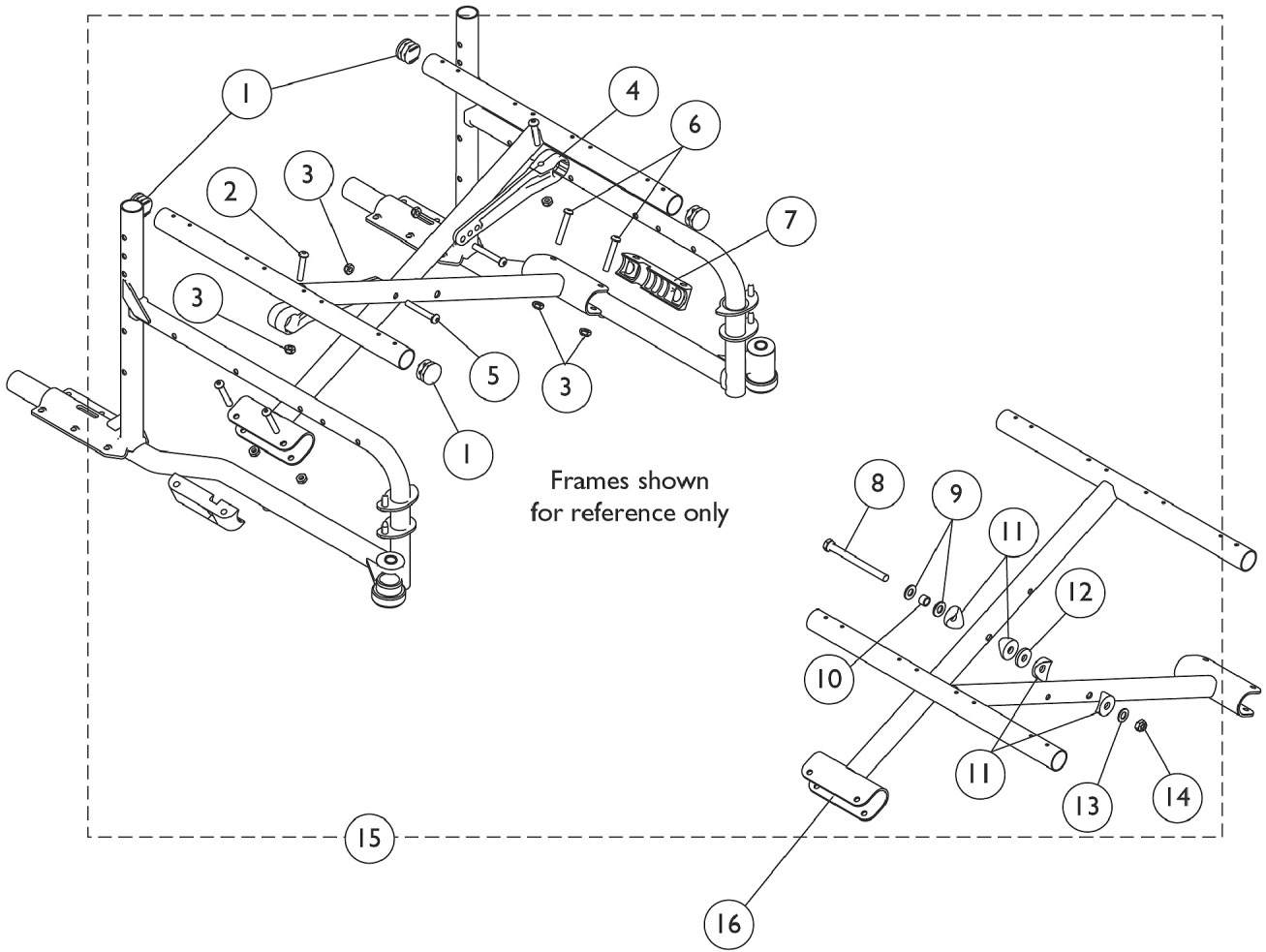

Crossbraces and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1029501	Plug Button, Black (1")	1	4
2		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	2
3		60275X000	Locknut (1/4-20)	1	8
4		1110001	Link, Pivot, Black (16 - 18" W)	1	2
5		1001762	Screw, Socket Button Head (1/4-20 x 1-3/4")	1	2
6		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	4
7		1012172-NLA	NLA - Clamp, Saddle	1	2
8		1098625	Screw, Hex Head (5/16-24 x 3-1/2")	1	1
9		16308X000	Washer, Flat (5/16 x 11/16 x 1/16")	1	2
10		1037913-NLA	NLA - Bushing (5/16 x 7/16 x 9/32")	1	1
11		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	4
12		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
13		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	1
14		1025730	Locknut (5/16-24)	1	1
15	A	1121209-NLA	NLA - Hardware, Crossbrace (Pair)	2	1
16		1109961	Crossbrace Assembly, Black (18" W)	1	2

NOTE: A - Includes items 1-14