

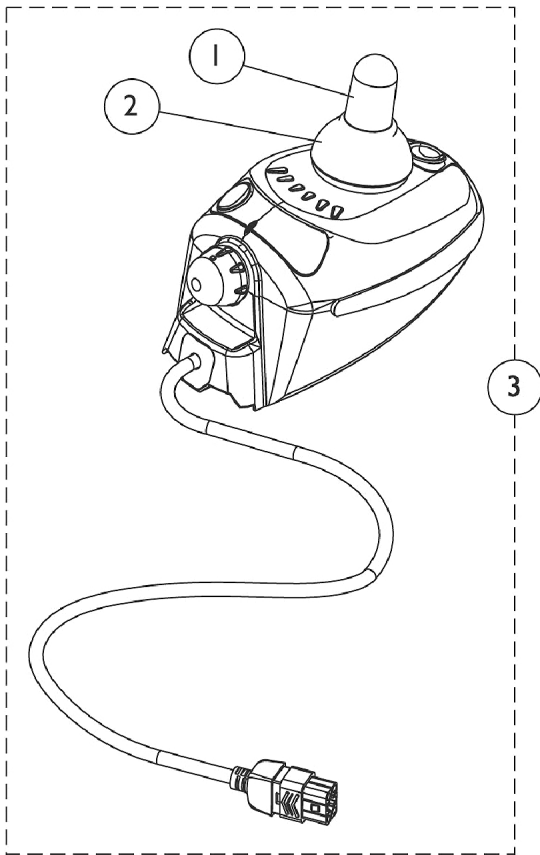


MK5 CSPJ Joystick

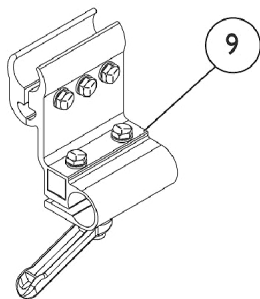

Joystick shown
for reference only

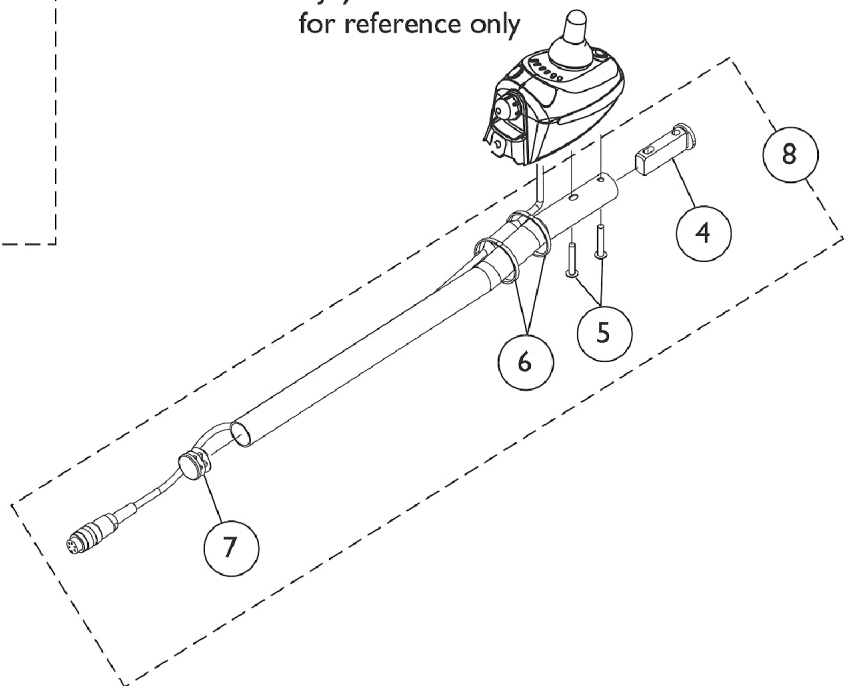

MK5 CSPJ Joystick

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1040217	Knob, Joystick Rubber, Black	1	1
2		1040218	Skirt, Joystick Rubber, Black	1	1
3		1121239	Joystick, MK5 CSPJ	1	1
4		1093063	Spacer, Joystick Tube, Black	1	1
5		1025714	Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2
6		1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	2
7		1027095	Plug Button, Black (7/8")	1	1
8	A	1093069	Kit, 1/2" Offset Joystick Tube with Hardware	1	1
9		1093071	Clamp, Fixed Height Joystick Tube	1	1

NOTE: A - Includes items 4-7