


Footrest Support Assembly T93H

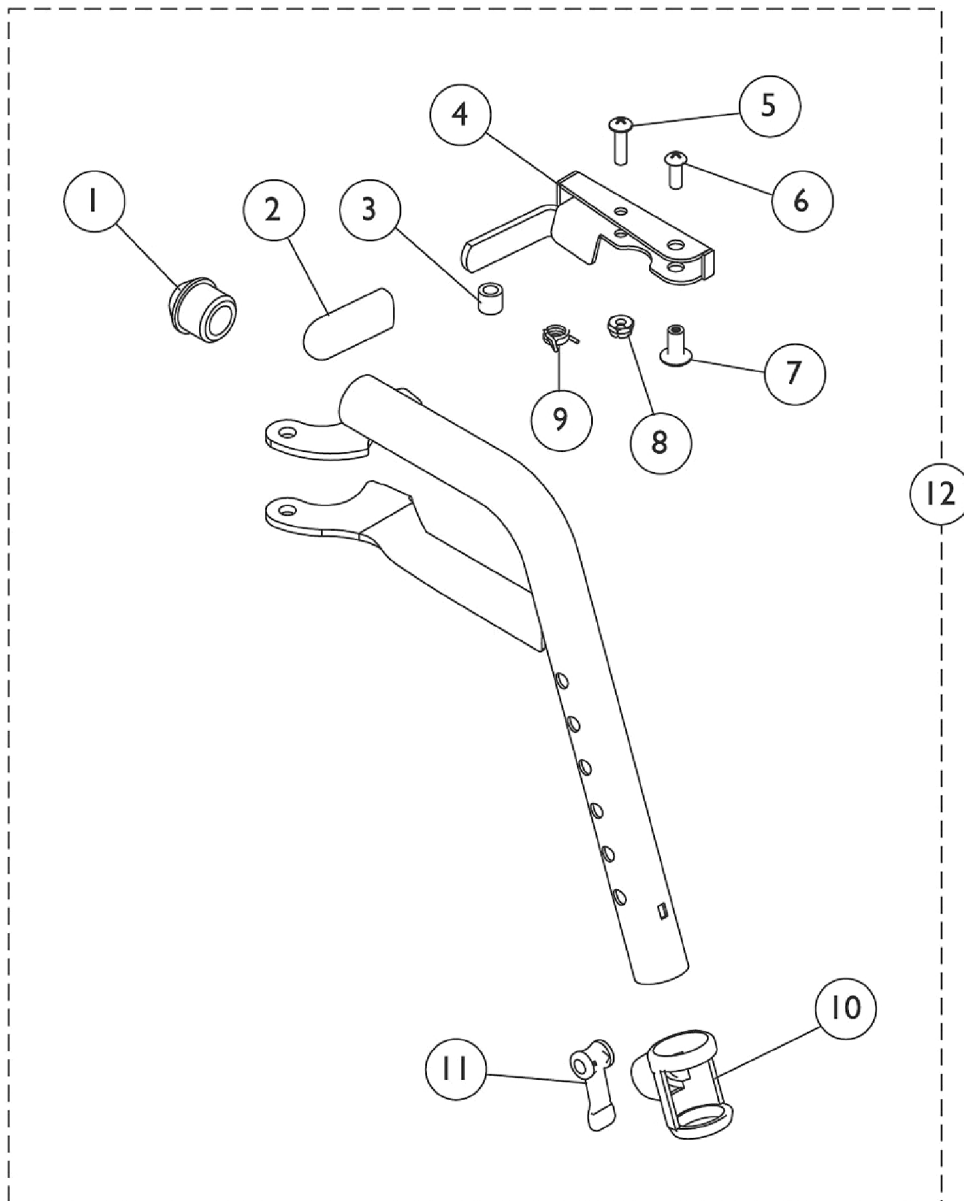

Footrest Support Assembly T93H

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1020719	Anti-Rattle, Black	1	1
2		1006490	Tip, Plastic, Black	1	1
3		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
4		1110178	Handle, Release, Black - Right	1	1
4		1110179	Handle, Release, Black - Left	1	1
5		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
6		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
7		1028846-NLA	NLA - Rivnut (#10-32)	1	1
8		1025735	Locknut (#10-32)	1	1
9		1028707	Spring, Release - Right	1	1
9		1028708	Spring, Release - Left	1	1
10		1048085	Housing, Cam	1	1
11		1048086	Lever, Cam, Black	1	1
12	A	8881118241	Footrest Support Assembly, Hemi, Color - Right	1	1
12	A	8881118242	Footrest Support Assembly, Hemi, Color - Left	1	1

NOTE: A - Includes items 1-11