
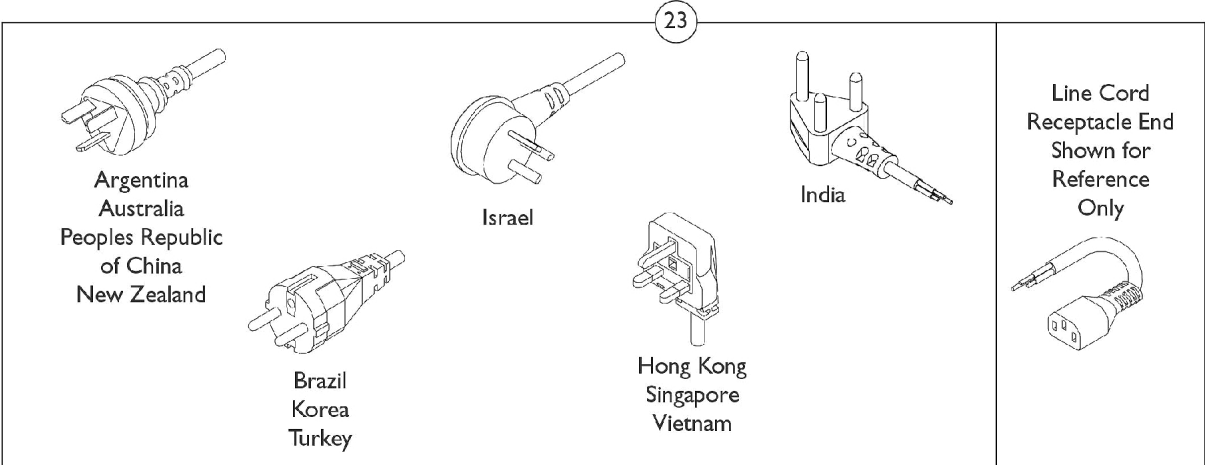
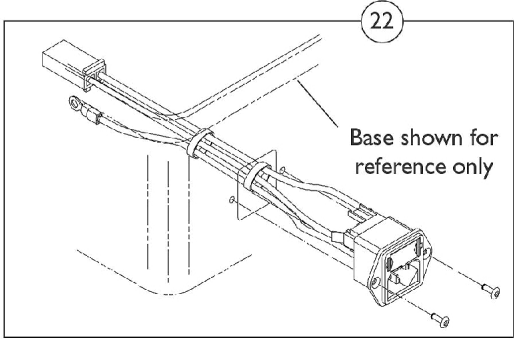


Cabinet and Base

Non Platinum 5 Models

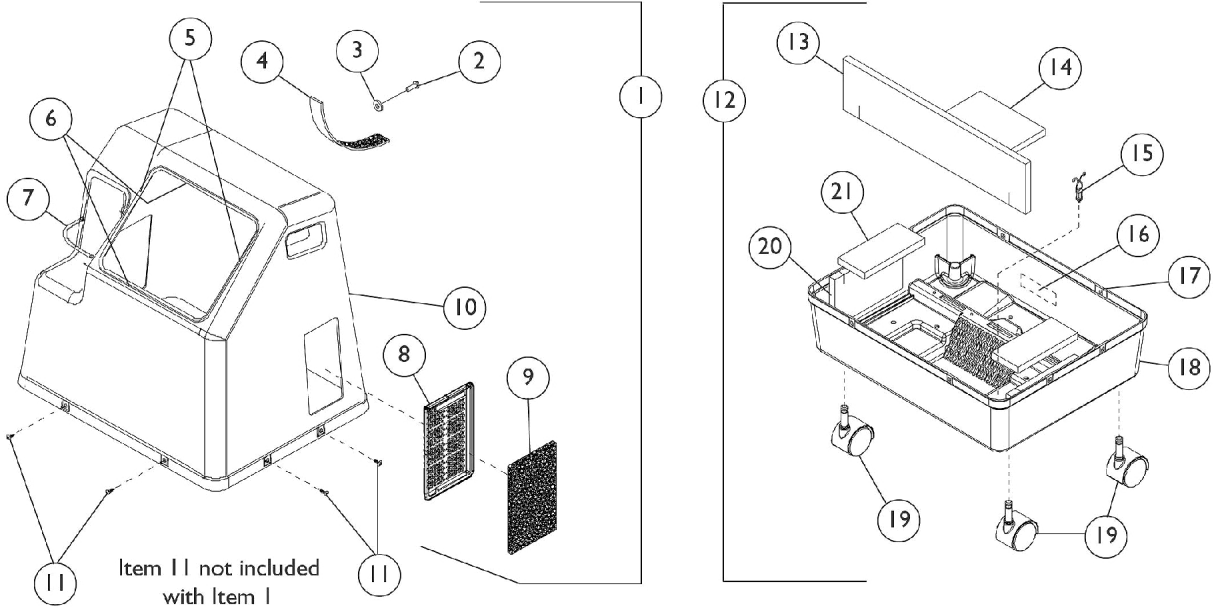

Cabinet and Base Non Platinum 5 Models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	2002295-NLA	NLA - Cabinet Assembly	1	1
2		2002540	NLA - Rivet, Plastic	1	1
3		1022500-NLA	NLA - Retainer, Push On	1	1
4		P0875X071-NLA	Strap, Hook and Loop	1	1
5		2001838-NLA	NLA - Gasket, Foam (9-13/16")	1	2
6		2001839	Gasket, Foam (8")	1	2
7		60125500	Cord, Elastic	1	1
8		2002107	Grid, Filter	1	1
9		2000489	Filter, Cabinet	1	1
10		2002150	Top, Cabinet	1	1
11		2000508	Screw, Phillips Truss Head Self Tap (#8-32 x 5/8")	1	8
12	B	1083454-NLA	NLA - Base Assembly	1	1
13		1089270-NLA	NLA - Foam, Base - Front	1	1
14		1086642-NLA	NLA - Foam, Base (3/4")	1	1
15		2000576	Stand-Off, Twist-Lok (1/2")	1	1
16		1083465-NLA	NLA - Label, Specification	1	1
17		2000360	Grommet, Plastic	1	8
18		1075658-NLA	NLA - Base, Finished	1	1
19	G	1076830-NLA	NLA - Caster (2")	1	4
19		1110426	Caster Assembly (2")	1	4
20		2002000-NLA	NLA - Foam, Base - Side	1	2
21		2001997-NLA	NLA - Foam, Base	1	2
22	C	2002574-NLA	NLA - Mounting, Inlet Module	1	1
23		1062794-NLA	NLA - Cord, Power - Argentina, Australia, Peoples Republic of China and New Zealand	1	1
23	H	1062791-NLA	NLA - Cord, Line - Brazil, Korea and Turkey	1	1
23		1131308-NLA	NLA - Tech Only - Cord, Power, Continental Europe	1	1
23		1102634	Cord, Power - India	1	1
23		1071398	Cord, Power - Israel	1	1
23		1062793-NLA	NLA - Cord, Power - Hong Kong, Singapore and Vietnam	1	1
	D,E,F	1071428	Label, Spanish - IRC5LX	1	1
	D,E,F	1071430	Label, Spanish - IRC5LXO2	1	1
	D,E,F	1079322	Label, Brazilian Portuguese - IRC5LX	1	1
	D,E,F	1079323	Label, Brazilian Portuguese - IRC5LXO2	1	1

NOTE: A - Includes: (1) each Cable Tie, Cabinet Front Foam, (2) each Cabinet Side Foam and Items 2-10.
 B - Includes Items 13-21.
 C - Includes (2) each Briv Rivet.
 D - Not Shown.
 E - Labels are not available for the following: Australia, Peoples Republic of China, Hong Kong, India, Israel, Korea, New Zealand, Singapore and Vietnam.
 F - Standard equipment manufactured with full English label applied. Language specific Label Overlays used to customize equipment.
 G - Item no longer available, refer to Part #1110426.
 H - Item no longer available, refer to Part #1131308.