


Pivot Slide Tube w/ Footplate - Legrest PH904A

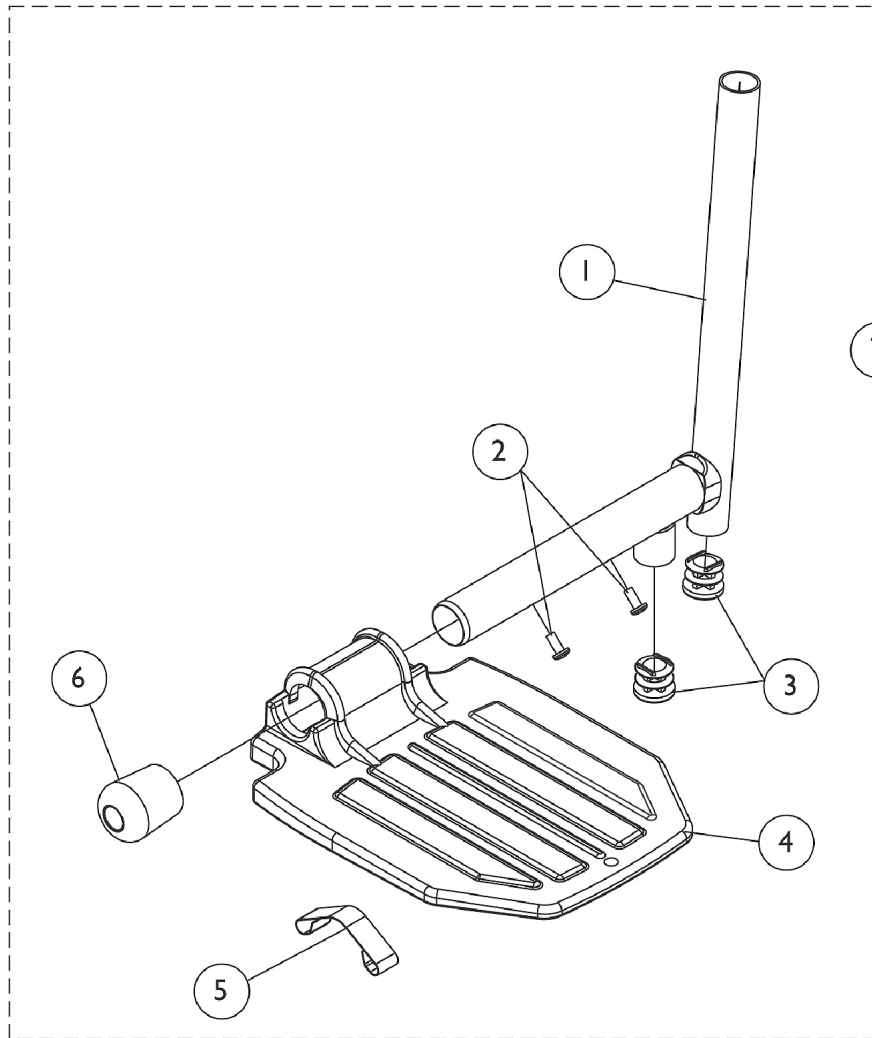

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1100568-NLA	NLA - Pivot/Slide Tube w/ Plug Button, Black	1	1
2		40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
3		1027097	Plug Button, Black (3/4")	1	2
4		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
5		1006981	Spring, Tension	1	1
6		1016346	Cap, End, Tube, Black (7/8")	1	1
7		1058604	Pivot/Slide Tube w/ Aluminum Footplate, Black - Right	1	1
7		1058605	Pivot/Slide Tube Assembly w/ Aluminum Footplate, Black - Left	1	1