


Wheel Lock - Push to Lock

12" and 20" Rear Wheels

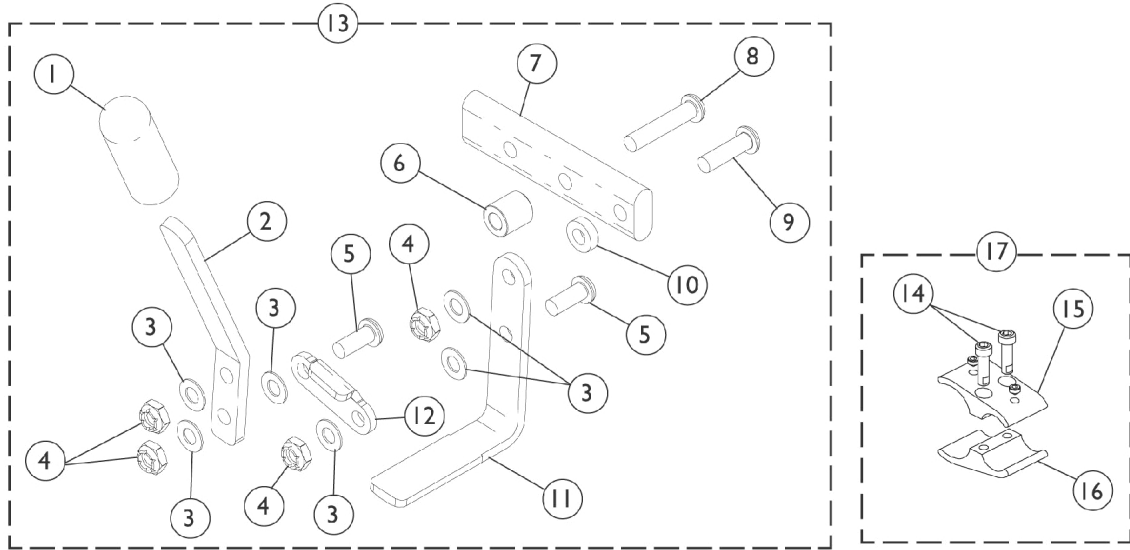

Wheel Lock - Push to Lock 12" and 20" Rear Wheels

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
2		1028414	Handle, Wheel Lock, Black - Right/ Left	1	1
3		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	6
4		1025765	Locknut (1/4-28)	1	4
5		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	2
6		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
7		1028399-NLA	NLA - Strap, Wheel Lock	1	1
8		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
9		1055306-NLA	NLA - Screw, Button Head Cap (1/4-28 x 7/8")	1	1
10		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
11		1021405-NLA	NLA - Shoe, Wheel Lock	1	1
12		1018733-NLA	NLA - Link, Stop, Black	1	1
13	A	1118610	Wheel Lock, Push to Lock High Mount -Right	1	1
13	A	1118611	Wheel Lock, Push to Lock High Mount -Left	1	1
14		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
15		1026097-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	1
16		1026096-NLA	NLA - Clamp, Wheel Lock, Threaded (1")	1	1
17	B	1039344	Kit, Wheel Lock Attaching Hardware (1" tubing)	1	1
14			Screw, Allen Head (1/4-20 x 1")	1	4
15			NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	2
16			NLA - Clamp, Wheel Lock, Threaded (1")	1	2

NOTE: A - Includes items 1-12
B - Includes items 14-16