
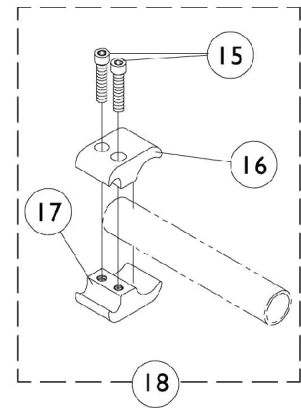


Wheel Lock - Pull to Lock

12" and 20" Rear Wheels

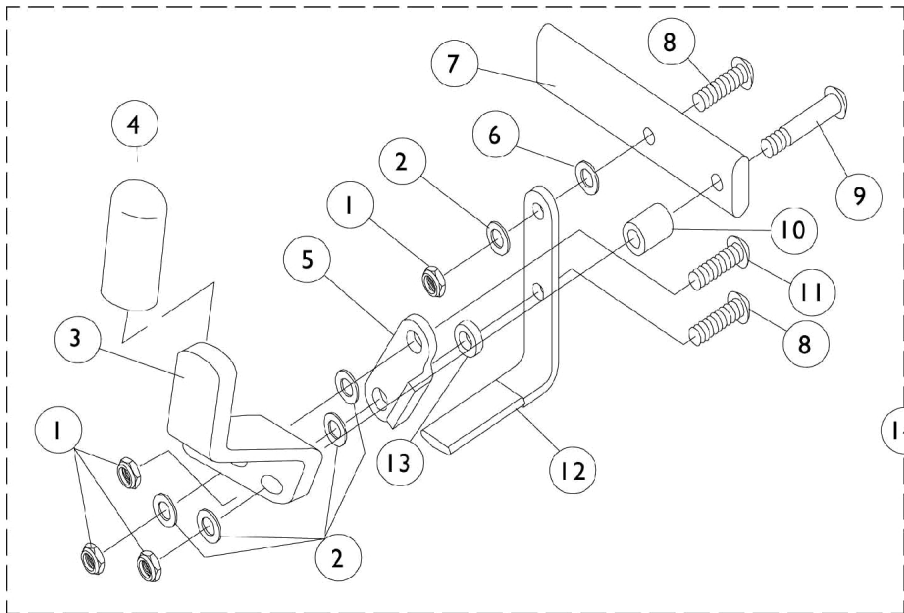

Wheel Lock - Pull to Lock 12" and 20" Rear Wheels

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1025765	Locknut (1/4-28)	1	4
2		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	5
3		1031780-NLA	NLA - Handle, Pull/ Lock, Black - Right	1	1
3		1031781-NLA	NLA - Handle, Pull/ Lock, Black - Left	1	1
4		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
5		1018733-NLA	NLA - Link, Stop, Black	1	1
6		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
7		1028399-NLA	NLA - Strap, Wheel Lock	1	1
8		1055305-NLA	NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	2
9		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
10		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
11		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	1
12		1021405-NLA	NLA - Shoe, Wheel Lock	1	1
13		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
14	A	1049539	Wheel Lock, Clamp On, Pull to Lock - Right	1	1
14	A	1049540	Wheel Lock, Clamp On, Pull to Lock - Left	1	1
15		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
16		1026097-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	1
17		1026096-NLA	NLA - Clamp, Wheel Lock, Threaded (1")	1	1
18	B	1039344	Kit, Wheel Lock Attaching Hardware (1" tubing)	1	1
			NLA - Clamp, Wheel Lock, Threaded (1")	1	2
			NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	2
			Screw, Allen Head (1/4-20 x 1")	1	4

NOTE: A - Includes items 1 - 13.
B - Includes items 15 - 17.