
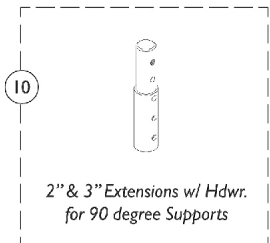
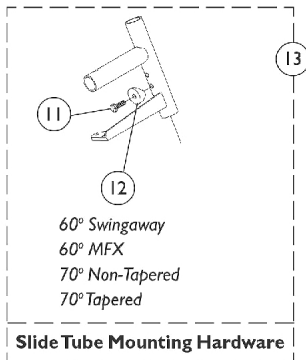
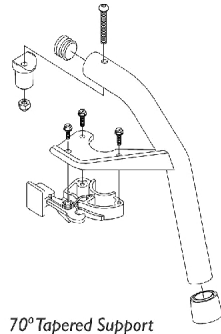
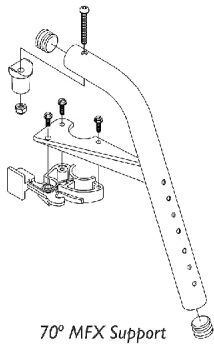
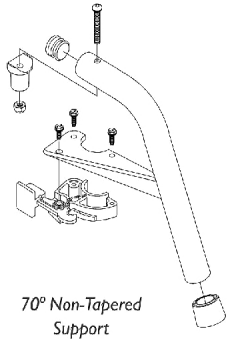
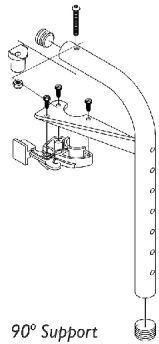
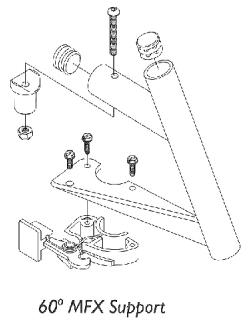


Footrest Support Assemblies

Swingaway 60 Degree, 70 Degree, and 90 Degree

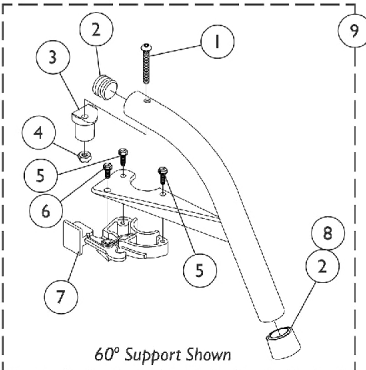

Footrest Support Assemblies

Swingaway 60 Degree, 70 Degree, and 90 Degree

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1022831	Screw, Socket Button Head (1/4-20 x 1-7/8")	1	1
2		1029501	Plug Button, Black (1")	1	1
3		1017148	Pivot, Footrest	1	1
4		1025736	Locknut (1/4-20)	1	1
5		1025451	Screw, Slotted Hex Washer Head (#10-32 x 5/8")	1	2
6		1017423	Screw, Slotted Hex Washer Head w/ Patch (#10-32 x 1/2")	1	1
7		1025454	Housing, Latch - Right	1	1
7		1025455	Housing, Latch - Left	1	1
8		1026124	Collar, Anti-Rattle	1	1
9	A	See Chart	#9 Footrest Support Assemblies	1	1
10		1118544	Kit, Extension w/ Hardware (2")	1	1
10		8881038231	Kit, 3" Extension w/ Hardware (Pair)	2	1
11		1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	1
12		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
13		1028004	Kit, Pivot Slide Tube Mounting Hardware (Pair)	2	1
	B	1026987	Guard, Impact (8" L)	1	1

NOTE: A - Specify frame finish when ordering.
B - Not Shown. For 70 deg. Tapered Footrest.

#9 Footrest Support Assemblies

Footrest Support Style	60° Swingaway	60° MFX	70° Non Tapered	70° MFX	70° Tapered	90° Swingaway
Right	8881026992	8881026998	8881026994	8881047481	8881026996	8881062596
Left	8881026993	8881026999	8881026995	8881047480	8881026997	8881062597