


# Fold Down Rigidizer Bar

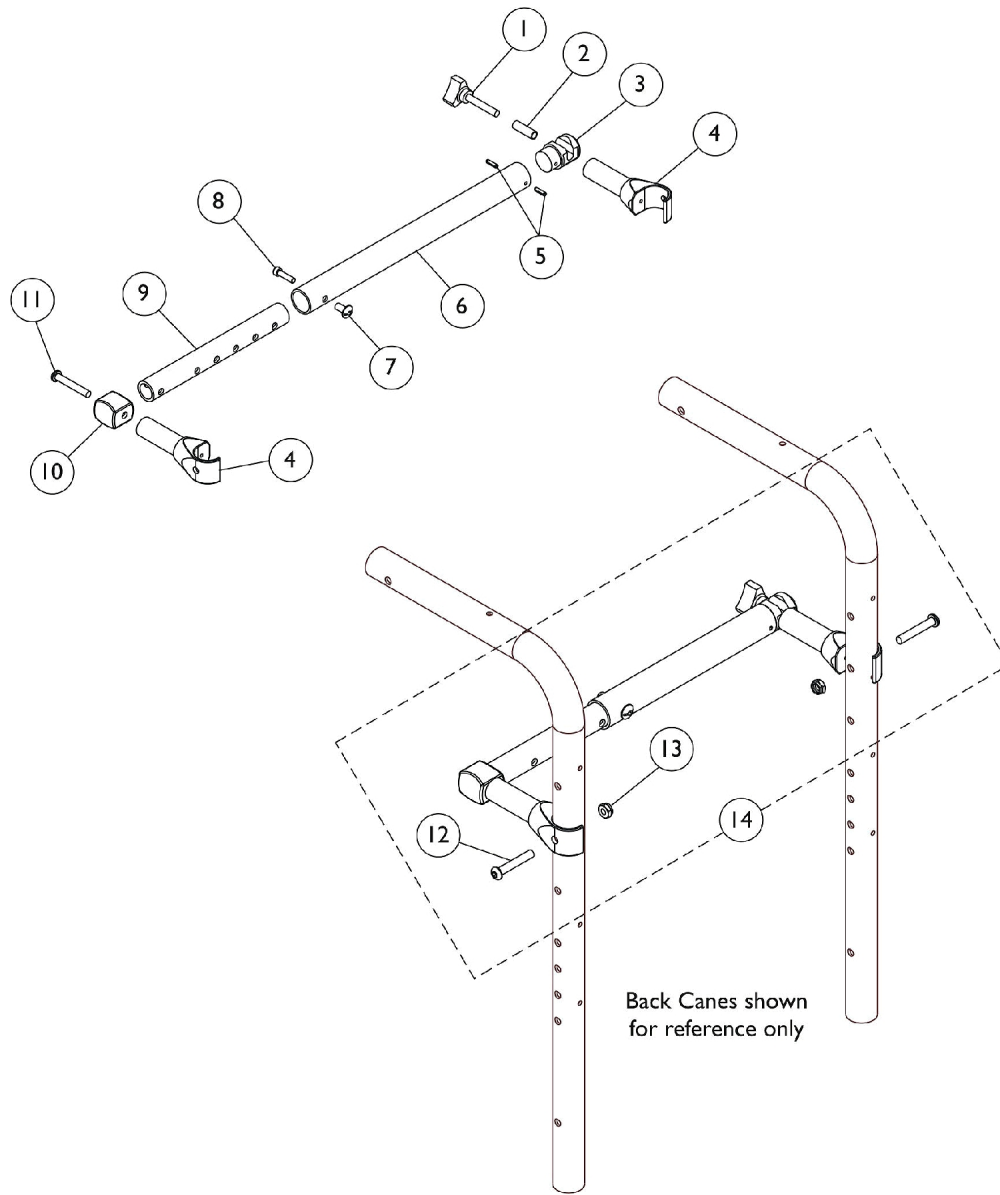

## Fold Down Rigidizer Bar

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027684	Knob, Clamping, Black	1	1
2		1118534	Bushing, Steel (5/16 x 1/4 x 1")	1	1
3		1118603-NLA	NLA - Plug, Slotted End	1	1
4	E	1118605-NLA	NLA - Clamp, Rigidizer Bar	1	2
5		51000M705	Pin, Roll (1/8 x 1/2")	1	2
6	A	1118600-NLA	NLA - Tube, Outside Rigidizer Bar, Black	1	1
6	B	1118601-NLA	NLA - Tube, Outside Rigidizer Bar, Black	1	1
7		1028846-NLA	NLA - Rivnut (#10-32)	1	1
8		0224600-NLA	NLA - Screw, Socket Head, Cap (#10-32 x 3/4")	1	1
9		1118599-NLA	NLA - Tube, Inside Rigidizer Bar	1	1
10	D	1118502-NLA	NLA - Cap, End, Lower, Black	1	1
11		1019590	Screw, Socket Button Head (1/4-20 x 1-5/8")	1	1
12		1083099	Screw, Socket Button Head (1/4-28 x 1-1/2")	1	2
13		60320X000	Locknut (1/4-28)	1	2
14	A,C	1118608-NLA	NLA - Kit, Spreader Bar with Hardware	1	1
14	B,C	1118609-NLA	NLA - Kit, Spreader Bar with Hardware	1	1

NOTE: A - 10-14" Wide Chairs  
 B - 14-18" Wide Chairs  
 C - Includes Rigidizer Bar (items 1-11) and items 12-13.  
 D - Part #1118502 is no longer available as a service part individually. See item 14 as a replacement.  
 E - Part #1118605 is no longer available as a service part individually. See item 14 as a replacement.