


# One Piece Tubular Footrest

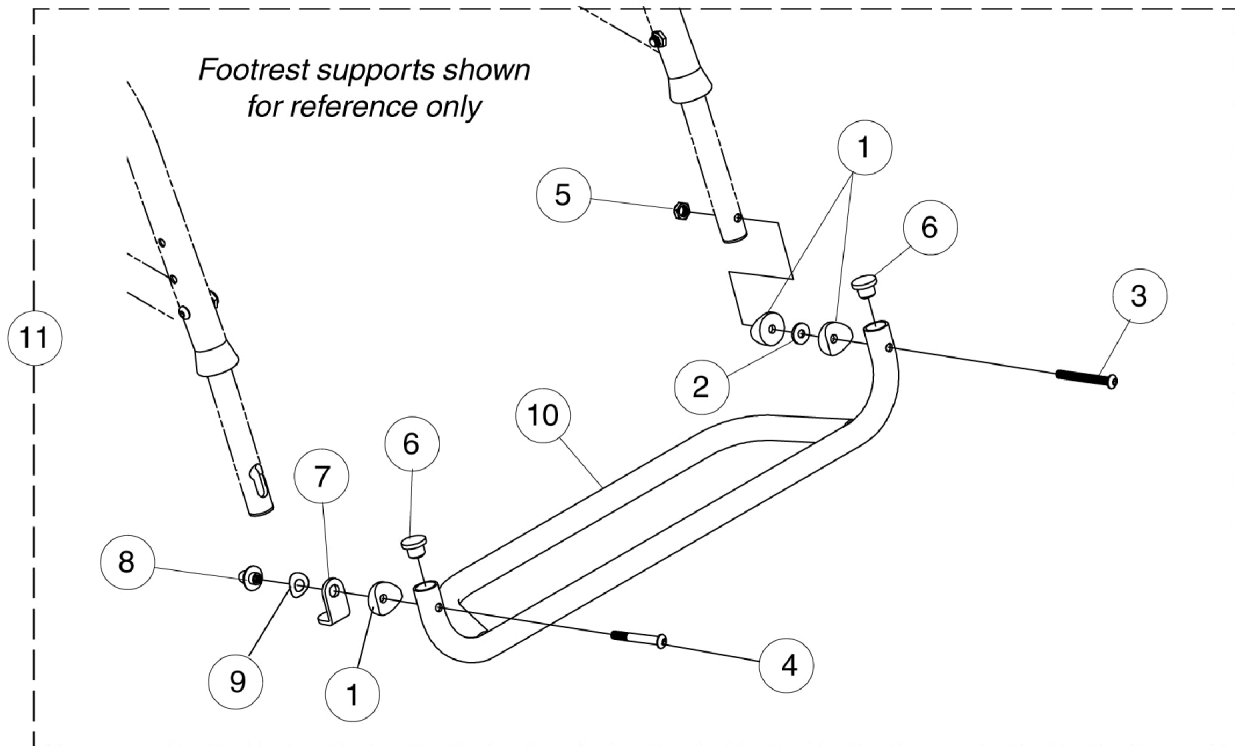

## One Piece Tubular Footrest

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1069059	Spacer, Coved (1/4 x 1/2 x 9/16")	1	3
2		1025456	Washer, Nylon (3/4 x 1/4 x 1/32")	1	1
3		1068505-NLA	NLA - Screw, Socket Button Head (1/4-28 x 2")	1	1
4		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
5		1025735	Locknut (#10-32)	1	1
6		1027097	Plug Button, Black (3/4")	1	2
7		1069041	Latch, Black	1	1
8		1068503	Knob	1	1
9		1068506	Washer, Wave (3/8 x 3/4 x 1/8")	1	1
10		See Chart	#10 Footrest Tube	1	1
11		See Chart	#11 Tubular Footrest Assembly	1	1

### #10 Footrest Tube

Footrest Finish	14" Wide	15" Wide	16" Wide	17" Wide	18" Wide	19" Wide	20" Wide
Black Anodized	1068881-NLA	1068882-NLA	1068883-NLA	1068884-NLA	1068885-NLA	1068886-NLA	1068887-NLA
Black Powder Coat	1068888-NLA	1068889-NLA	1068890-NLA	1068891-NLA	1068892-NLA	1068893-NLA	1068894-NLA

### #11 Tubular Footrest Assembly

Footrest Finish	14" Wide	15" Wide	16" Wide	17" Wide	18" Wide	19" Wide	20" Wide
Black Anodized	1068895-NLA	1068896-NLA	1068897-NLA	1068898-NLA	1068899-NLA	1068900-NLA	1068901-NLA
Black Powder Coat	1068922-NLA	1068923-NLA	1068924-NLA	1068925-NLA	1068926-NLA	1068927-NLA	1068928-NLA