


# Back Canes and Backrest Hardware

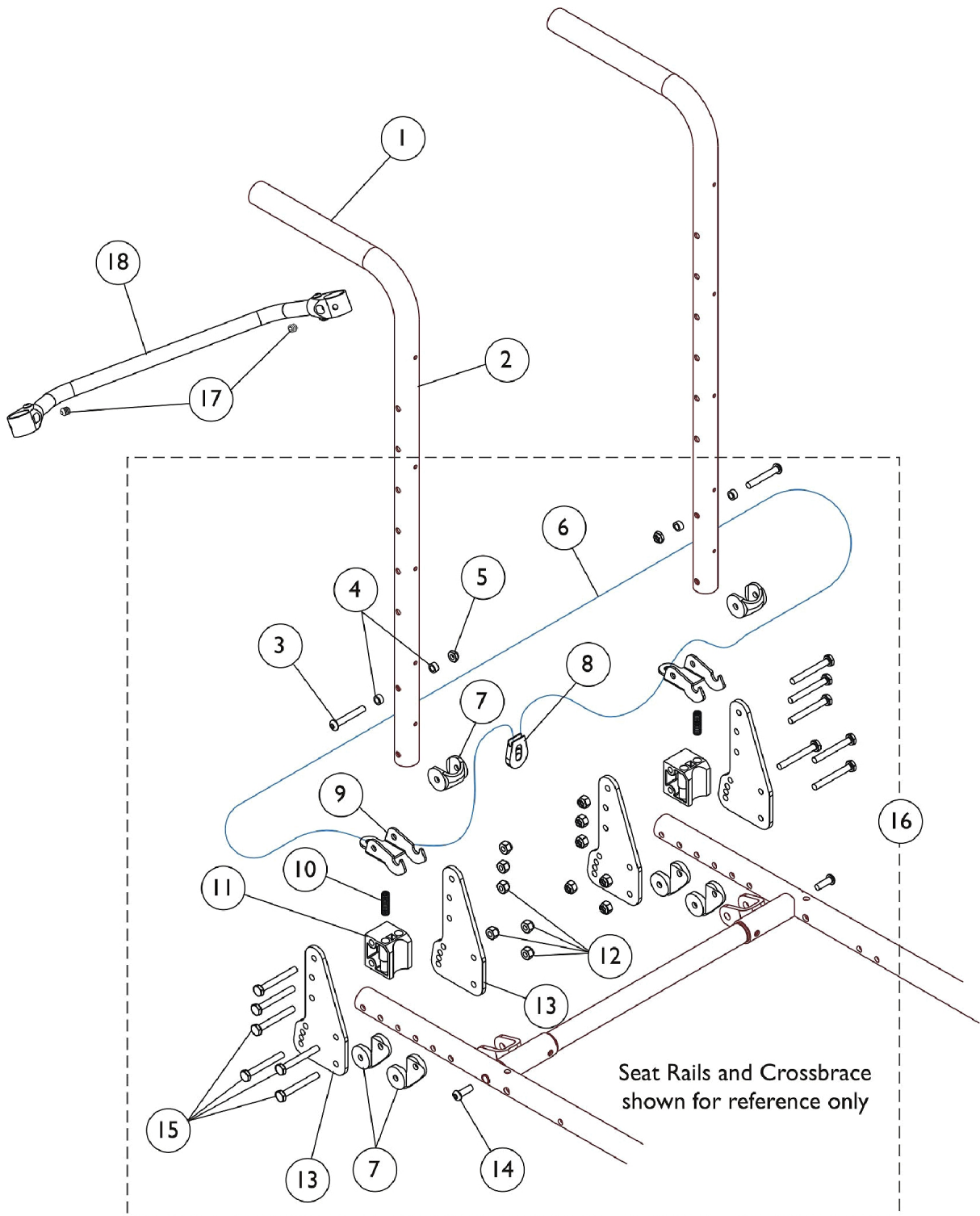

## Back Canes and Backrest Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1055398	Grip, Hand, Foam (4")	1	2
2		1121600	Back Cane Assembly, Black - Right/Left (20" H)	1	2
2		1121601	Back Cane Assembly, Black - Right/Left (24" H)	1	2
16	A	1044808	Kit, Backrest Hardware	1	1
3			Screw, Socket Button (1/4-20 x 3/4")	1	2
4			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
5			Locknut (1/4-20)	1	4
6			NLA - Cord (5-1/2')	1	1
7			Uni-Washer, Coved	1	6
8			Lock, Cord, Black	1	1
9			NLA - Latch, Seat, Black	1	2
10			NLA - Spring, Compression (5/16 x 1-3/8 x 1/32")	1	2
11			NLA - Block, Stop	1	2
12			Locknut (1/4-28)	1	12
13			NLA - Plate, Seat Adjustment	1	4
14			Screw, Socket Button Head (1/4-20 x 1-5/8")	1	2
15			Screw, Hex Head (1/4-28 x 2")	1	12
17		1045495-NLA	NLA - Screw, Set Socket (1/4-20 x 1/4")	1	2
18	B	See Chart	#18 Backrest Crossmember	1	1

NOTE: A - Includes items 3-15  
B - See Backrest Crossmember Chart

### #18 Backrest Crossmember

Chair Width	14"	15"	16"	17"	18"
	1042510	1042511	1042512	1050083	1050080
Chair Width	19"	20"	21"	22"	
	1050077	1050074	1085292-NLA	1085293-NLA	