


# Footrest Support Assembly - T93H Series

Invacare Transport Only

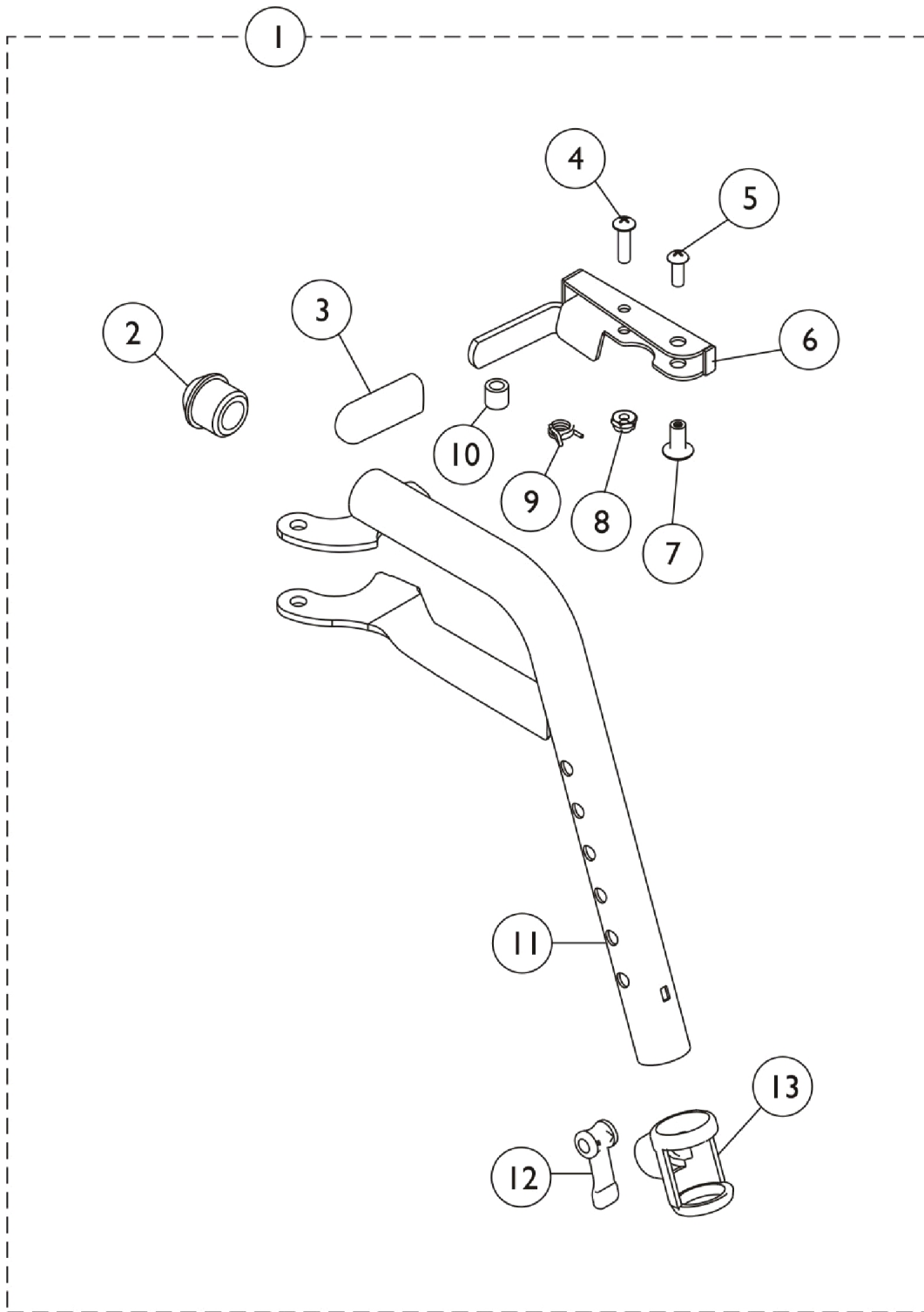

## Footrest Support Assembly - T93H Series Invacare Transport Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1111274	Footrest Support Assembly, Hemi, Black - Right	1	1
1	A	1111275	Footrest Support Assembly, Hemi, Black - Left	1	1
2		1020719	Anti-Rattle, Black	1	1
3		1006490	Tip, Plastic, Black	1	1
4		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
5		40163X002	Screw, Phillips Round Head (#10-32 x 1/2")	1	1
6	B	1110176-NLA	NLA - Handle, Release, Chrome - Right	1	1
6	B	1110177-NLA	NLA - Handle, Release, Chrome - Left	1	1
6	C	1110178	Handle, Release, Black - Right	1	1
6	C	1110179	Handle, Release, Black - Left	1	1
7		67005X000	Rivnut (#10-32)	1	1
8		60279X003	Locknut (#10-32)	1	1
9		1008935	Spring, Release - Right	1	1
9		1008936	Spring, Release - Left	1	1
10		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
11	B	1111270-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Right	1	1
11	B	1111271-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Left	1	1
11	C	1111268	Tube, Footrest, Support, Hemi, Black - Right	1	1
11	C	1111269	NLA - Tube, Footrest, Support, Hemi, Black - Left	1	1
12		1048086	Lever, Cam, Black	1	1
13		1048085	Housing, Cam	1	1

NOTE: A - Includes Items 2-13. After 10/14/14 color changed from Chrome to Black. It is recommended to order both the right and left side to match.  
B - Before 10/15/14 color was Chrome  
C - After 10/14/14 color is Black