



# Motor and Gearbox Assembly

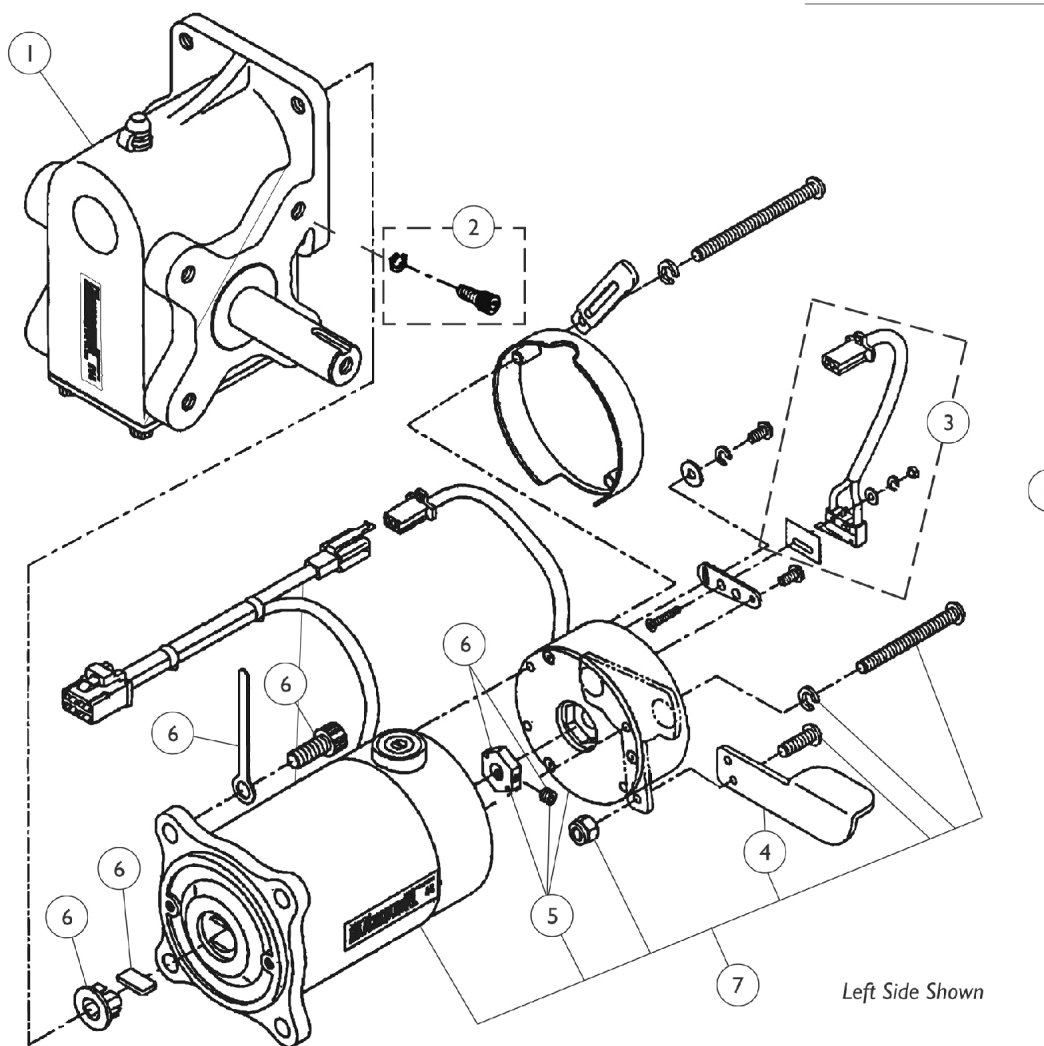

Motor Brush Cap  
shown for ref. only

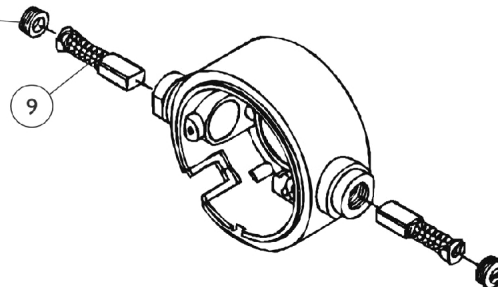

## Motor and Gearbox Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1099727-NLA	NLA - Gearbox - Right	1	1
1		1099726-NLA	NLA - Gearbox - Left	1	1
2		1099728-NLA	NLA - Hardware, Gearbox Mounting Includes: 8 ea. Washer, Spring (M8 x 13) 8 ea. Screw, Socket Head (M8 x 1.25 x 20)	8	8
3		1099733-NLA	NLA - Switch Assembly with Hardware Includes: 1 ea. Switch, 1 ea. Paper, Switch, 2 ea. Washer (2.2 x 5 x 0.5), 2 ea. Washer, Spring (2.1 x 4.3 x 0.7) 2 ea Nut (M2 x 0.4 x 6)	1	1
4		1099732-NLA	NLA - Handle, Free Wheel, Right	1	1
4		1099731-NLA	NLA - Handle, Free Wheel, Left	1	1
5		1099730-NLA	NLA - Brake, Motor with Axle	1	1
6		1099725-NLA	NLA - Package, Hardware, Motor/ Gearbox Includes: 1 ea. Plate, Motor Cable Tie Down 4 ea. Screw, Socket Head (M8 x 1.25 x 20) 1 ea. Plate, Bushing 1 ea. Bushing 1 ea. Driving Axle, Disc Brake with Set Screw	1	1
7		1099741-NLA	NLA - Motor/ Brake Assembly - Right	1	1
7		1099729-NLA	NLA - Motor/ Brake Assembly - Left	1	1
8		1099735-NLA	NLA - Motor/ Gearbox Assembly, Right	1	1
8		1099734-NLA	NLA - Motor/ Gearbox Assembly, Left	1	1
9		1099736-NLA	NLA - Brush, Motor (Pair)	2	2