


Shroud - Bottom

IRC5LXAW and IRC5LXO2AW

Shroud - Bottom

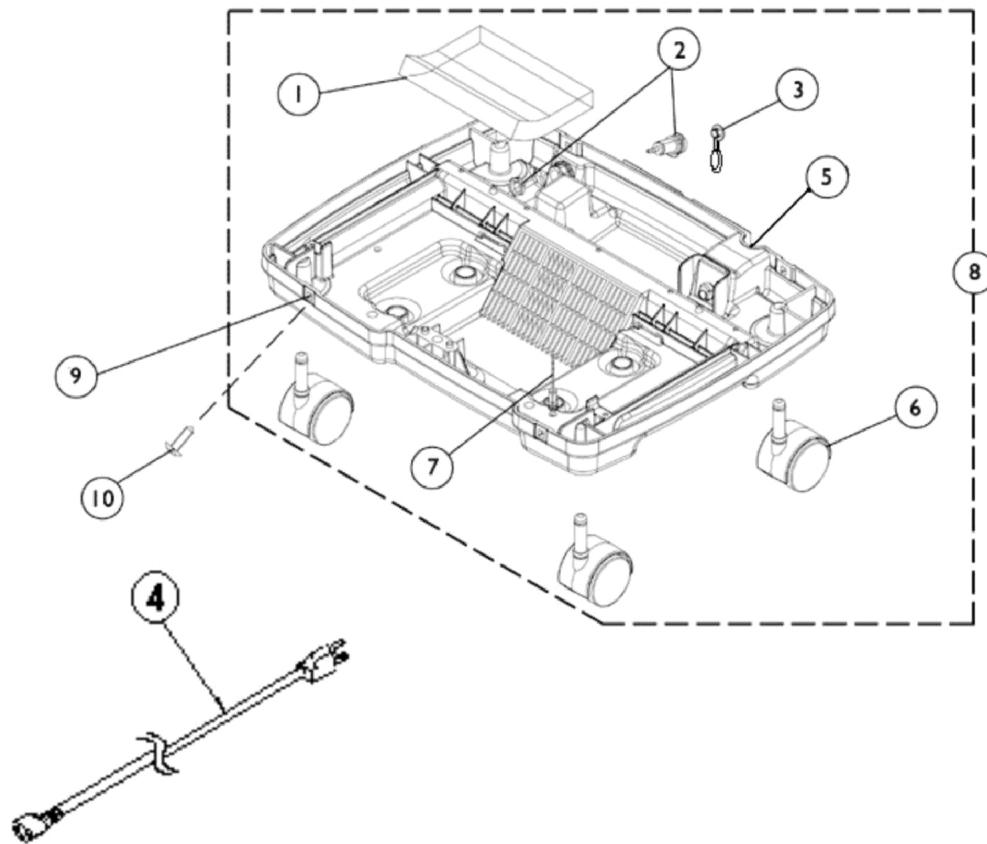

Shroud - Bottom

IRC5LXAW and IRC5LXO2AW

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1108816	Foam, Base	1	1
2		1104151	Fitting, Hose Barb with Nut (1/8")	1	1
3		1108211	Cap, End, Dust, Green	1	1
4	A,D	1062791-NLA	NLA - Cord, Line - Brazil, Korea and Turkey	1	1
4	A	1131308-NLA	NLA - Tech Only - Cord, Power, Continental Europe	1	1
4	A	1062794-NLA	NLA - Cord, Power - Argentina, Australia, Peoples Republic of China and New Zealand	1	1
4	A	1062793-NLA	NLA - Cord, Power - Hong Kong, Singapore and Vietnam	1	1
4	A	1071398	Cord, Power - Israel	1	1
4	A	1102634	Cord, Power - India	1	1
5		1105515-NLA	NLA - Tech Only - Module, Inlet (15A/ 250 VAC)	1	1
6	C	1076830-NLA	NLA - Caster (2")	1	4
6		1110426	Caster Assembly (2")	1	4
7		2000576	Stand-Off, Twist-Lok (1/2")	1	1
8	B,E	1134174-NLA	NLA - Tech Only - Base, Compressor, Black	1	1
9		2002360-NLA	NLA - Grommet, Plastic	1	4
9		2000360	Grommet, Plastic	1	4
10		2000508	Screw, Phillips Truss Head Self Tap (#8-32 x 5/8")	1	4

NOTE: A - Select line cord desired.
B - Includes Base Only.
C - Item no longer available, refer to Part #1110426.
D - Item no longer available, refer to Part #1131308.
E - As of 5/8/07 the base color has changed to black.