
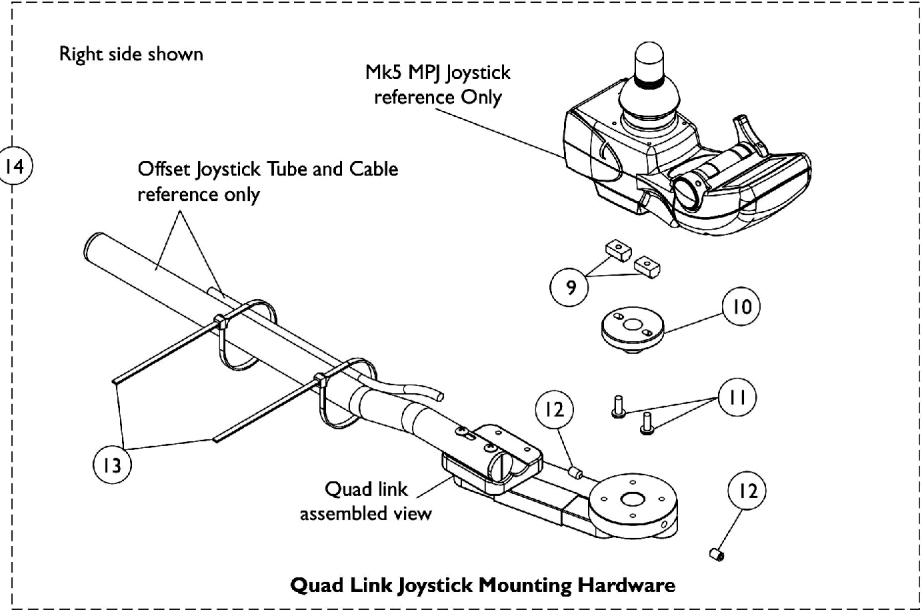


Quad Link Assembly For MK5 MPJ Joystick

Before 1/13/04

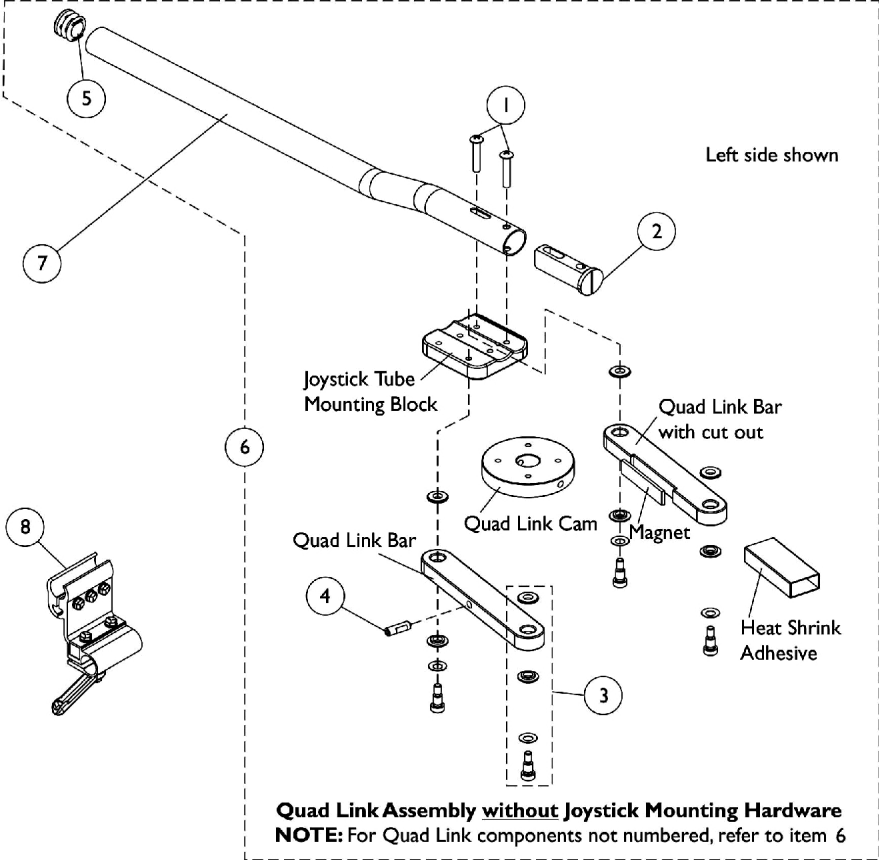

Quad Link Assembly For MK5 MPJ Joystick Before 1/13/04

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1025714	Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	2
2		1093063	Spacer, Joystick Tube, Black	1	1
3		1123005-NLA	NLA - Quad Link Hardware Consists Of: 4 each Screw, Socket Head Shoulder (1/4 x 1/4") 4 each Washer, Spring (1/4 x 1/2 x 1/32") 8 each Spacer, Flanged (1/4 x 9/16 x 1/8")	1	1
4		1123010-NLA	NLA - Screw, Socket Head Set w/ Patch (1/4-20 x 3/4")	1	1
5		1027095	Plug Button, Black (7/8")	1	1
6	A,C	1123003-NLA	NLA - Quad Link Assembly - Right	1	1
6	A,C	1123004-NLA	NLA - Quad Link Assembly - Left	1	1
7		1093506-NLA	NLA - Tube, Joystick Offset (1/2")	1	1
8		1093071	Clamp, Fixed Height Joystick Tube	1	1
9		1117216-NLA	NLA - T-Nut (#10-32)	1	2
10		1121101-NLA	NLA - Mount, Joystick Spin	1	1
11		1134086-NLA	NLA - Screw, Socket Button Head w/ Patch (#10-32 x 1/2")	1	2
12		1123010-NLA	NLA - Screw, Socket Head Set w/ Patch (1/4-20 x 3/4")	1	2
13		1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	2
14	B,C	1121114-NLA	NLA - Quad Link Joystick Mounting Hardware	1	1
NOTE: A - Includes items 1-5. When ordering a Quad Link Assembly for the first time, item 14 must be ordered as well. B - Includes items 9-13 and used for MK5 MPJ joystick. C - No Longer Available. See Quad Link dated after 1/12/04					