




# Front, Top, and Rear Shrouds w/ Attaching Hardware

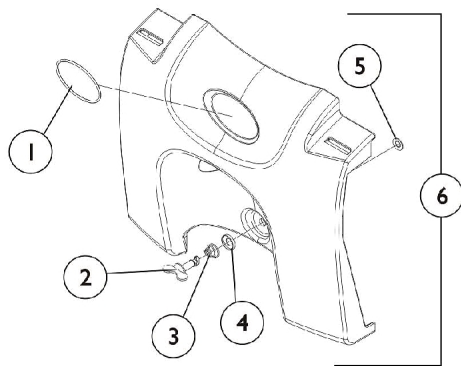


Front Shroud

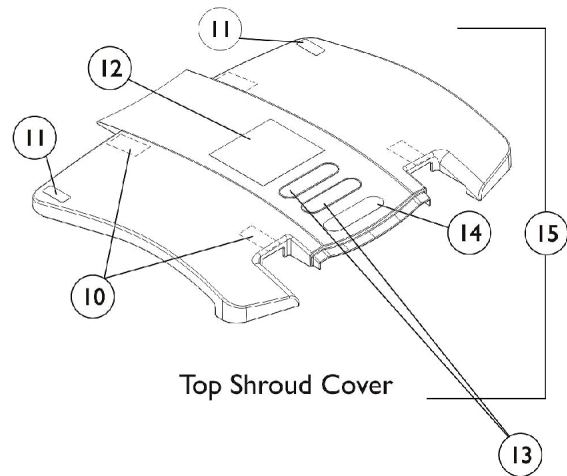


Top Shroud Cover

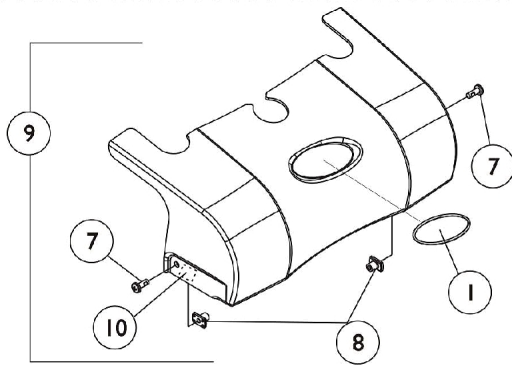

Rear Shroud Cover

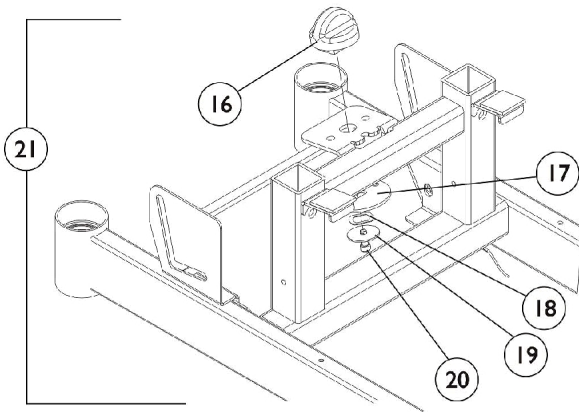

Rear Shroud Knob Assembly

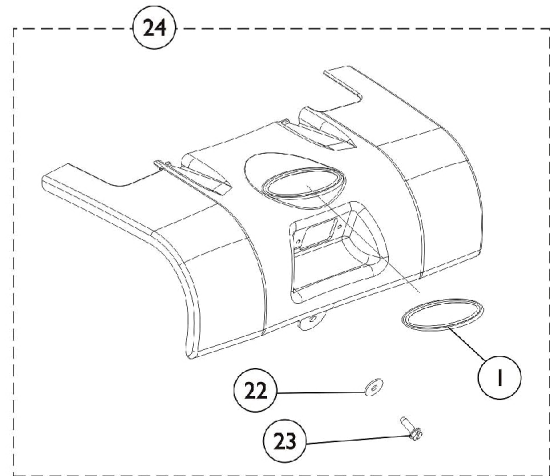

Rear Shroud and Hardware **After 12/18/03**

Rear Shroud and Hardware **Before 12/19/03**

# Front, Top, and Rear Shrouds w/ Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1095489	Decal, Invacare 3-D Logo (2-1/2" L)	1	1
2		1070574-NLA	NLA - Stud, Wing Head (1" Long)	1	1
3		1093312-NLA	NLA - Spring, Torsion Ejector	1	1
4		1070581-NLA	NLA - Washer, Nylon Cupped (9/32 x 19/32 x 3/32")	1	1
5		1070580-NLA	NLA - Washer, Nylon Retainer (1/4 x 1/2 x 1/32")	1	1
6	A	1112571	Shroud with Hardware - Front	1	1
7		1134086-NLA	NLA - Screw, Socket Button Head w/ Patch (#10-32 x 1/2")	1	2
8		1112371-NLA	NLA - Nut, Weld (10-32)	1	2
9	B,G	1114487-NLA	NLA - Shroud, Rear w/ Hardware (Candy Red Pearl)	1	1
9	B,G,J	See Note	See Footnote	1	1
9	B,G	1114489-NLA	NLA - Shroud, Rear w/ Hardware (Dark Blue Pearl)	1	1
10	E	1070509-NLA	NLA - Fastener, Loop with Adhesive Back (1" W x 2" L)	1	4
11		1079203	Decal, Warning Pinch Point	1	2
12	G	1114804-NLA	NLA - Label, Warning Wiring Dual, 22NF Battery	1	1
12	H,K	1118397-NLA	NLA - Label, Warning Wiring Dual, 22NF Battery	1	1
13	I	1119281-NLA	NLA - Plate, Name, Patent Numbers	1	1
14		1035900	Decal, Transportation	1	1
15	C,G,I	1116423-NLA	NLA - Shroud, Top w/ Decals	1	1
15	C,H,I	1122816	Shroud, Top w/ Decals	1	1
16		1109033-NLA	NLA - Knob, Shroud, Black	1	1
17		1109034-NLA	NLA - Plate, Latch Shroud	1	1
18		1108475-NLA	NLA - Washer, Wave (17/32 x 3/4 x 1/64")	1	1
19		1109031-NLA	NLA - Washer, Steel (13/64 x 1 x 3/64")	1	1
20		1113438-NLA	NLA - 3Screw, Socket Head (#10-32 x 1/2")	1	1
21	D	1112284-NLA	NLA - Hardware, Shroud Mounting - Rear	1	1
22		1070587	Washer, Flat (3/16 x 5/8 x 1/16")	1	1
23		1117240	Screw, Thumb (#10-32 x 1/2")	1	1
24	F,H	1123225	Kit, Shroud w/ Hardware, Rear, Candy Red Pearl	1	1
24	F,H	1123226	Kit, Shroud w/ Hardware, Rear, Magic Black	1	1
24	F,H	1123227	Kit, Shroud w/ Hardware, Rear, Dark Blue Pearl	1	1

NOTE: A - Includes Shroud plus items 1-5.  
B - Includes Shroud plus items 1, 7, 8 and 10.  
C - Includes Shroud plus items 10-12, & 14.  
D - Includes items 16-20.  
E - Rear shroud uses 1 ea. of item 10 cut in half to fit on the inside right and left side of the cover  
F - Includes Shroud plus items 1,22 and 23.  
G - Chairs built Before 12/19/03  
H - Chairs built After 12/18/03  
I - As of 3/10/04 the Patent Name Plates are no longer included in the Top Shroud Assembly. Two name plates have been replaced by one.  
J - No longer available as a service part.  
K - Part #1118397 is no longer available as a service part individually. Order item 15 as a replacement.