
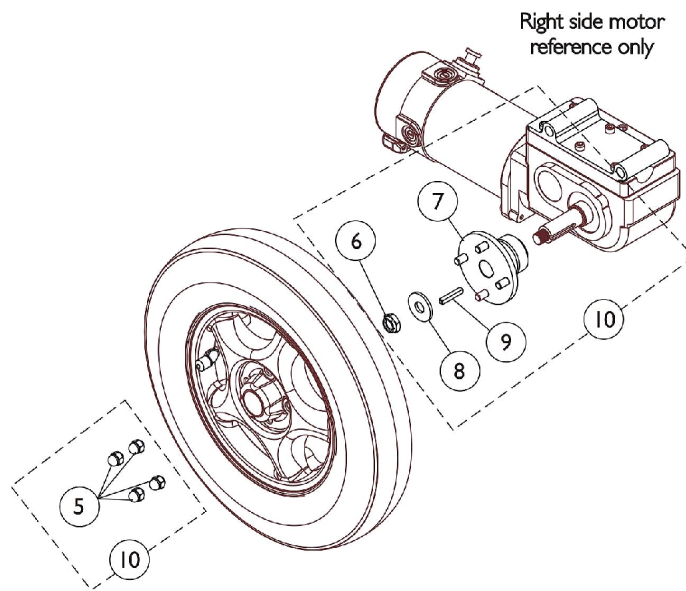


# Drive Wheel Assembly

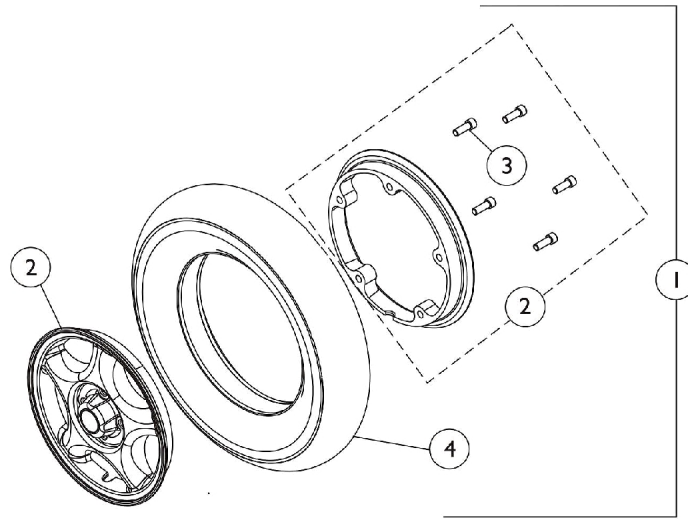

## Drive Wheel Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1104437	Drive Wheel Assembly, Flat Free, Grey Tire/Silver Rim (14 x 3")	1	1
1	A	1122793	Drive Wheel Assembly, Flat Free, Grey Tire/Silver Rim (14" x 3") (Pair)	2	1
2	B	1104882-NLA	NLA - Kit, 8" Rim w/ Hardware (14 x 3" Wheel)	1	1
4		1101754	Tire w/ Foam Fill Insert, Grey (14" x 3")	1	1
4		1122795	Tire w/ Foam Fill Insert, Grey (14" x 3") (Pair)	2	1
5		1100580	Nut, Beveled Hex (5/16-24)	1	4
6		1025737	Locknut (1/2-20)	1	1
7		1106174	Hub, Wheel	1	1
8		1036933	Washer (1/2 x 1-9/32 x 1/8")	1	1
9		1000180	Key (3/16 x 1-1/4")	1	1
10	C	1123400	Kit, Drive Wheel Mounting Hardware (14" x 3")	1	1

NOTE: A - Includes items 2-4  
 B - Sold complete. Sold complete. Includes the Rims and (5) ea. Socket Head Screws w/ Patch (5/16-18 x 7/8").  
 C - Includes items 5-9