


Footrest Support Assembly

TH93A, TH93C, TH93S

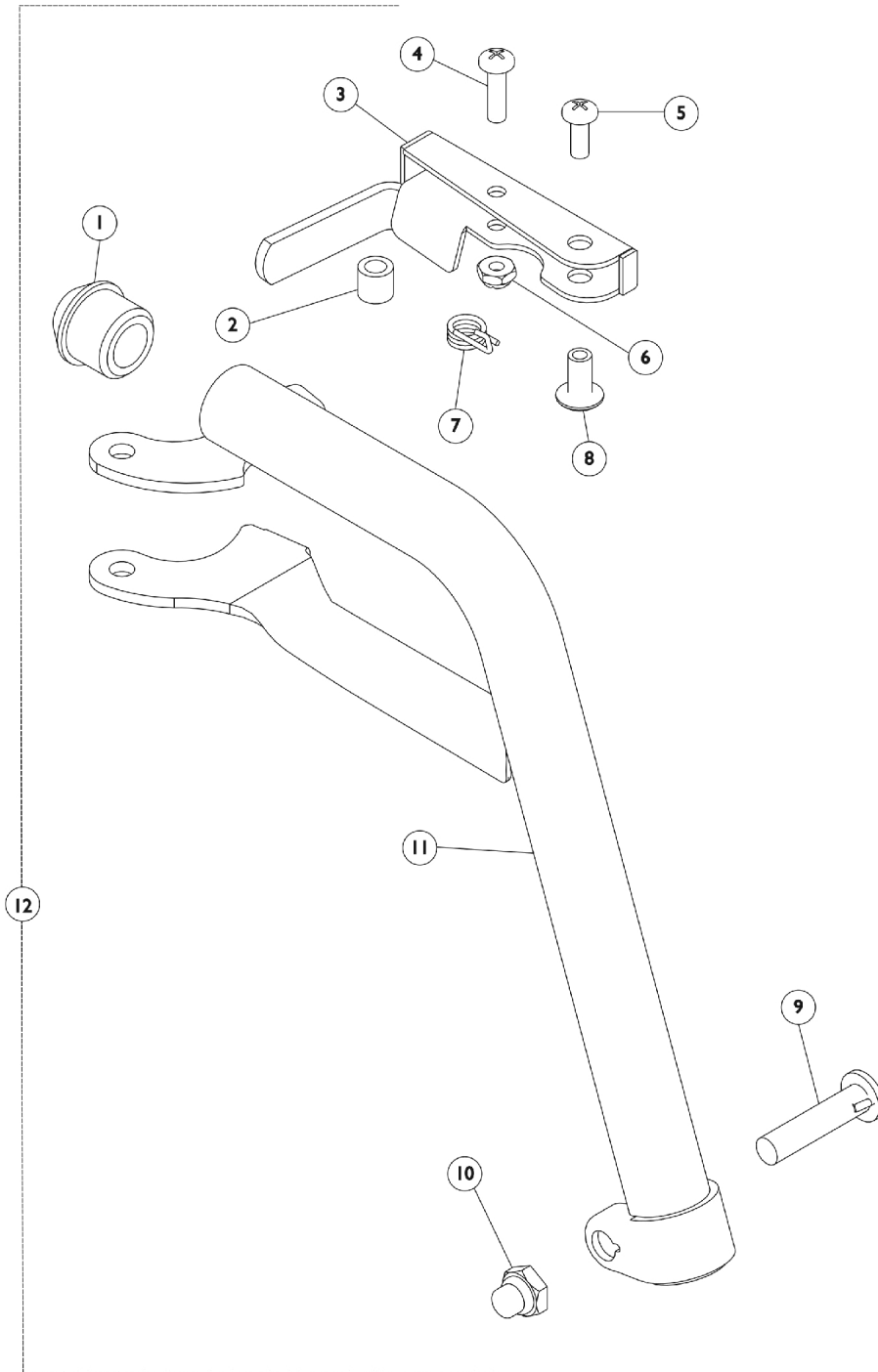

Footrest Support Assembly TH93A, TH93C, TH93S

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1020719	Anti-Rattle, Black	1	1
2		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
3	D	1110176-NLA	NLA - Handle, Release, Chrome - Right	1	1
3	D	1110177-NLA	NLA - Handle, Release, Chrome - Left	1	1
3	A	1110178	Handle, Release, Black - Right	1	1
3	A	1110179	Handle, Release, Black - Left	1	1
	E	1087698-NLA	NLA - Handle, Release, Chrome - Right	1	1
	E	1087699-NLA	NLA - Handle, Release, Chrome - Left	1	1
4		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
5		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
6		1025735	Locknut (#10-32)	1	1
7		1028707	Spring, Release - Right	1	1
7		1028708	Spring, Release - Left	1	1
8		1028846-NLA	NLA - Rivnut (#10-32)	1	1
9		1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	1
10		1107321	Nut, Low Cap (5/16"-24)	1	1
11		1124223-NLA	NLA - Tube, Footrest Support, Chrome - Right	1	1
11		1124224-NLA	NLA - Tube, Footrest Support, Chrome - Left	1	1
11		1124221	Tube, Footrest Support, Black - Right	1	1
11		1124222	Tube, Footrest Support, Black - Left	1	1
11	B	8881119154	Tube, Footrest Support, Color - Right	1	1
11	B	8881119155	Tube, Footrest Support, Color - Left	1	1
12	C	1124227	Footrest Support Assembly, Hemi, Black- Right	1	1
12	C	1124228	Footrest Support Assembly, Hemi, Black - Left	1	1
12	C	1124225	Footrest Support Assembly, Hemi, Black - Right	1	1
12	C	1124226	Footrest Support Assembly, Hemi, Black - Left	1	1
12	B,C	8881124229	Footrest Support Assembly, Hemi, Color - Right	1	1
12	B,C	8881124230	Footrest Support Assembly, Hemi, Color - Left	1	1

NOTE: A - Used on Black and Colored Front Riggings.
 B - Specify frame color when ordering, see category titled, "Frame Finishes, Upholstery Colors, Decals and Packaging."
 C - Includes items 1 thru 11.
 D - For use on chrome chairs.
 E - Not Shown. Order this item when ordering chair with Cyclical Drive only.