


Legrest Support Assembly

TH94

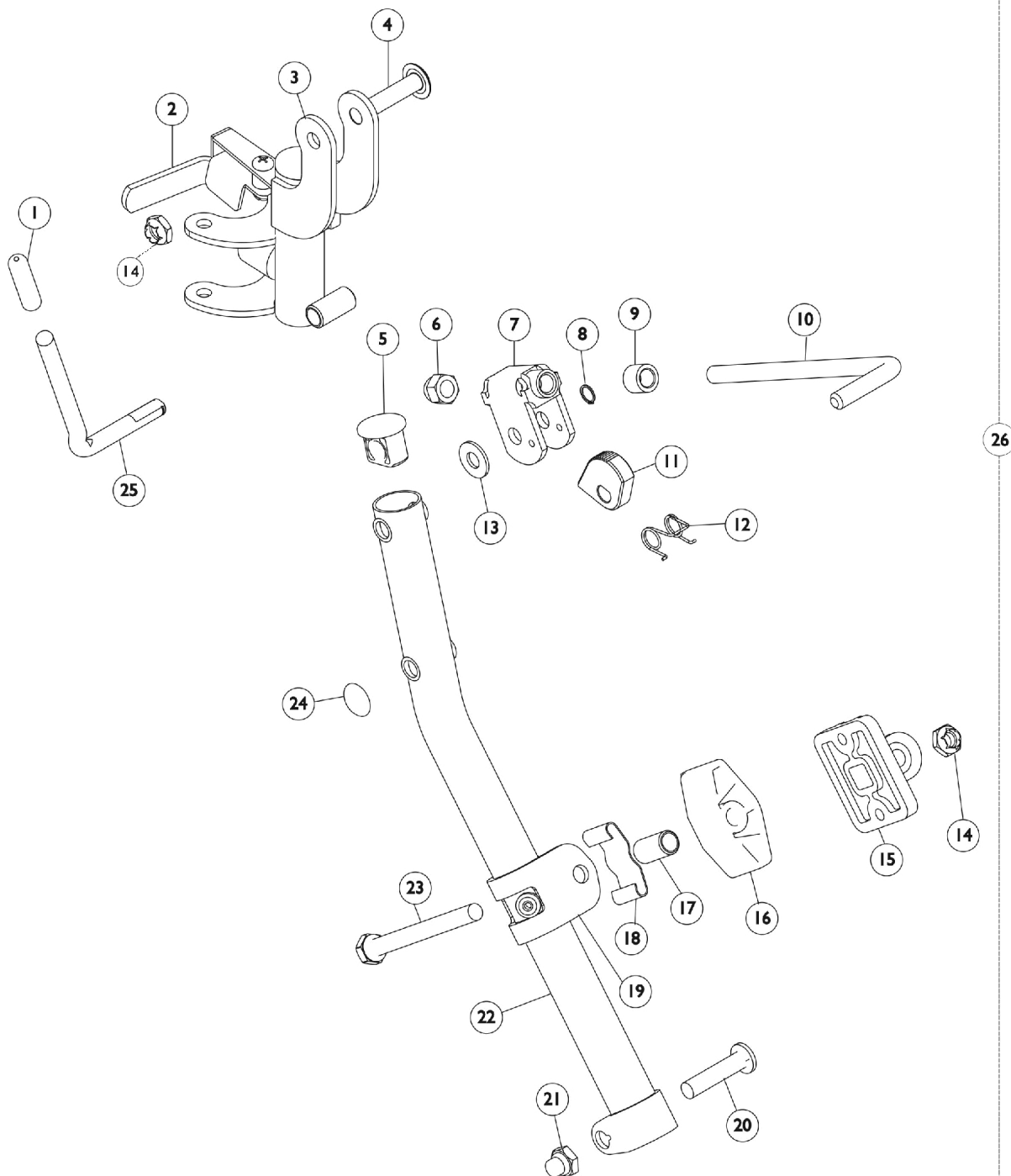

Legrest Support Assembly

TH94

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1040060	Tip, Plastic, Black	1	1
2	A	1110178	Handle, Release, Black - Right	1	1
2	A	1110179	Handle, Release, Black - Left	1	1
2	B	1031235-NLA	NLA - Handle, Release, Chrome - Right	1	1
2	B	1031236-NLA	NLA - Handle, Release, Chrome - Left	1	1
	D	1087698-NLA	NLA - Handle, Release, Chrome - Right	1	1
	D	1087699-NLA	NLA - Handle, Release, Chrome - Left	1	1
3	A	1124215	Legrest Support Tube Assembly, Hemi, Black - Right	1	1
3	A	1124216	Legrest Support Tube Assembly, Hemi, Black - Left	1	1
3	B	1031356-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Right	1	1
3	B	1031357-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Left	1	1
4		1030948	Screw, Slotted Truss Head (5/16-24 x 1-1/2")	1	1
5		1095433	Plug Button, Black (7/8")	1	1
6		10261X000	Nut, Acorn (5/16-24)	1	1
7		1110308	Bracket, Release, Black	1	1
8		1040831	Ring, Snap	1	1
9		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
10		42326X001	Bar, Ratchet	1	1
11		1037125	Cam, Legrest	1	1
12		1037485	Spring, Release	1	1
13		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
14		60264X001	Locknut (5/16-24)	1	2
15		04526X019	Plate, Legrest Pad	1	1
16		1091220	Spacer, Legrest	1	1
17		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
18		48823X008	Spring, Leaf	1	1
19	A	1110207	Bracket, Calf Pad Mounting, Black	1	1
19	B	1110208-NLA	NLA - Bracket, Calf Pad Mounting, Chrome	1	1
20	A	1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	1
20	B	11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
21		1107321	Nut, Low Cap (5/16"-24)	1	1
22	A,E	1119206-NLA	NLA - Tube, Elevating Support, Black - Right	1	1
22	A,E	1119207-NLA	NLA - Tube, Elevating Support, Black - Left	1	1
22	B	1095434-NLA	NLA - Tube, Support, Chrome - Right	1	1
22	B	1095435-NLA	NLA - Tube, Support, Chrome - Left	1	1
23		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
24		1096773	Nut, Push Dome (5/16")	1	1
25		1037126	Handle, Release	1	1
26	A,C	1124217	Legrest Support Assembly, Hemi, Black - Right	1	1
26	A,C	1124218	Legrest Support Assembly, Hemi, Black - Left	1	1
26	A,C	1124219	Legrest Support Assembly, Hemi, Black - Right	1	1
26	A,C	1124220	Legrest Support Assembly, Hemi, Black - Left	1	1

Legrest Support Assembly

TH94

NOTE: A - Black
B - Chrome
C - Includes items 1 thru 24.
D - Not Shown. Order this item when ordering chair with Cyclical Drive only.
E - No longer available. See item 26 part #1124217 or 1124218 as replacements.