


Frames - Back Canes & Hardware

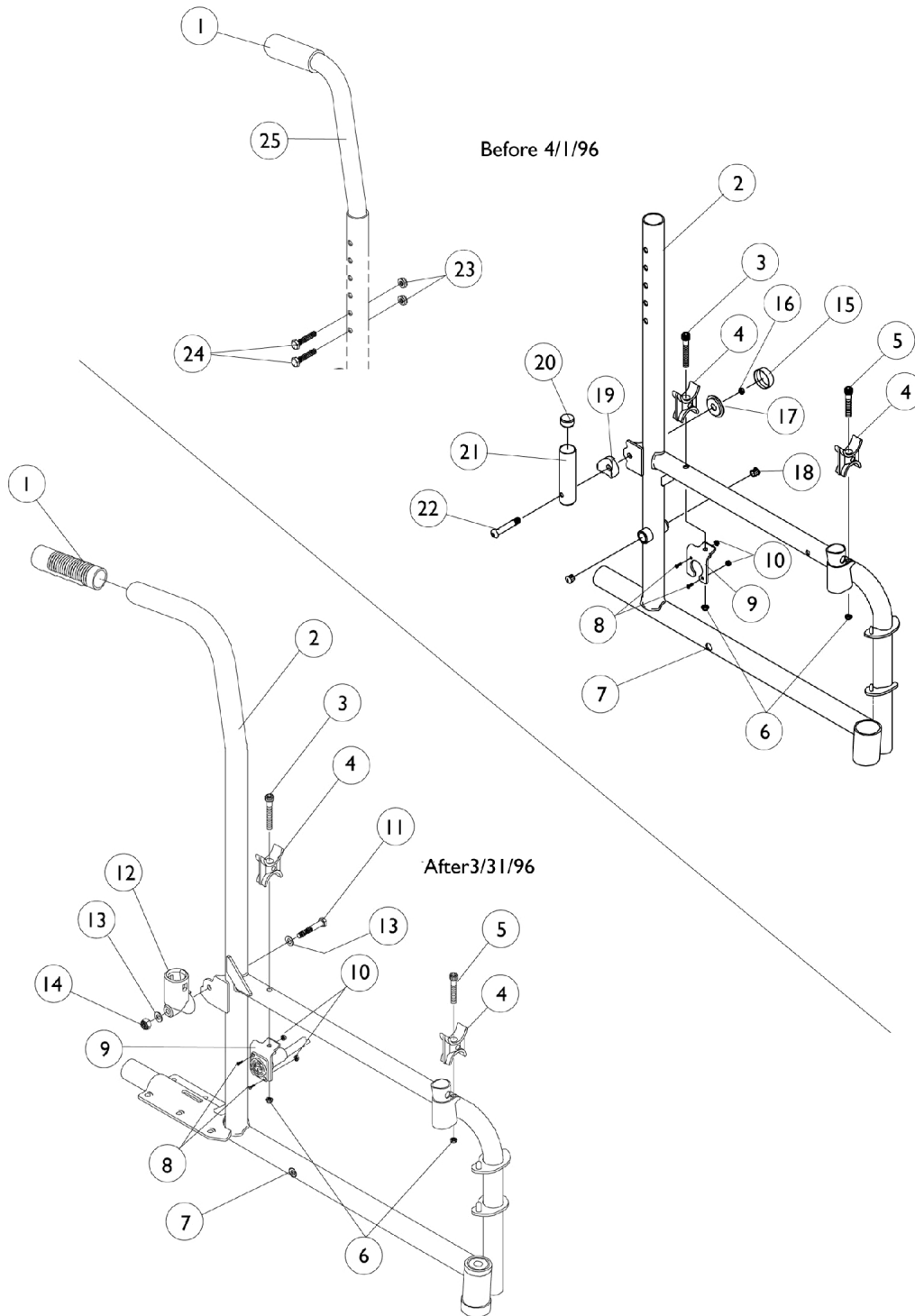

Frames - Back Canes & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	1006229	Grip, Foam (8")	1	1
1	D	1021261	Grip, Hand (6")	1	1
2	A,B,C	8881039395-NL A	NLA - Frame, Color - Right	1	1
2	A,B,C	8881039396-NL A	NLA - Frame, Color - Left	1	1
2	A,B,D	8881061512-NL A	NLA - Frame, Side, Color - Right	1	1
2	A,B,D	8881061513-NL A	NLA - Frame, Side, Color - Left	1	1
3		1003424	Screw, Socket Head (1/4-20 x 1-3/4")	1	1
4		1015390	Guide, Seat	1	1
5		1004170	Screw, Socket Head (1/4-20 x 1-1/2")	1	1
6		7930333	Locknut (1/4-20)	25	1
7		1058520-NLA	NLA - Fastener, Panel Snap-In	1	1
8		7930318-NLA	NLA - Package, Screw, Phillips Pan Head (#4-40 x 1/2")	25	1
9		1053837-NLA	NLA - Bracket, Charger Connector, Black	1	1
10		P0870X000	Nut, Hex Keps (#4-40)	1	2
11		1055298	Screw, Hex Head (1/4-28 x 1-5/8")	1	1
12		1056406	Cup, Socket Pivot	1	1
13		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
14		7930334-NLA	NLA - Locknut (1/4-28)	25	1
15		1016291	Cap, Hub, Black	1	1
16		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	1
17		1016290-NLA	NLA - Washer, Caplugs (9/32 x 1 x 3/32")	1	1
18		1027097	Plug Button, Black (3/4")	1	2
19		1015426	Spacer, Coved Cam - Right	1	1
19		1015427	Spacer, Coved Cam - Left	1	1
20		1023975-NLA	NLA - Bushing (3/8 x 3/4 x 3/8")	1	1
21		1015856	Socket, Armrest	1	1
22		1023968-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	1
23		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
24		1057955	Screw, Hex Head (1/4-20 x 1-1/4")	1	4
25		1028372	Back Cane Assembly, 10 Degree Bend	1	2

NOTE: A - Specify frame color.
 B - Decals not included with frame.
 C - Before 4/1/96
 D - After 3/31/96