
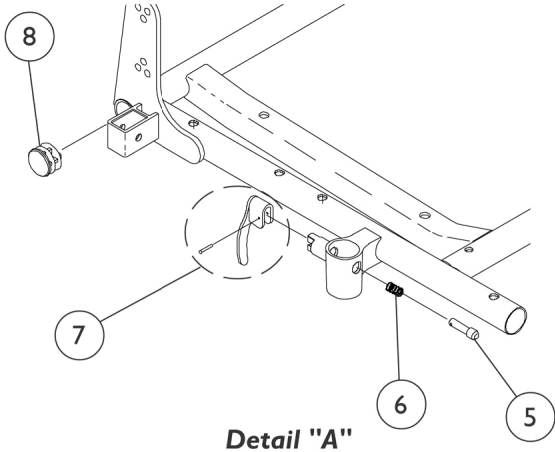


# Seat Frame, Adapter and Hardware - A.S.B.A.

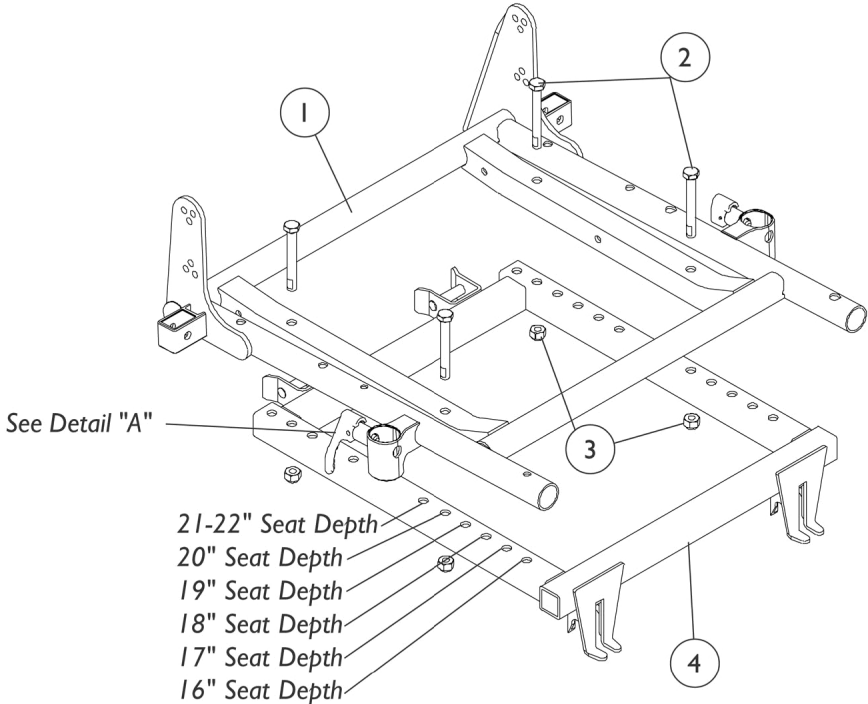

# Seat Frame, Adapter and Hardware - A.S.B.A.

## #1 Seat Frame Assembly

Chair Width	16"-17" Deep	18"-19" Deep	20"-21" Deep	22" Deep
18" Wide	1070674-NLA	1070683-NLA	1070692-NLA	1070701-NLA
19" Wide	1070675-NLA	1070684-NLA	1070693-NLA	1070702-NLA
20" Wide	1070676-NLA	1070685-NLA	1070694-NLA	1070703-NLA
21" Wide	1070677-NLA	1070686-NLA	1070695-NLA	1070704-NLA
22" Wide	1070678-NLA	1070687-NLA	1070696-NLA	1070705-NLA
23" Wide	1070679-NLA	1070688-NLA	1070697-NLA	1070706-NLA
24" Wide	1070680-NLA	1070689-NLA	1070698-NLA	1070707-NLA

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Seat Frame Assembly	1	1
2		1043628-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 2-1/2")	1	4
3		1026158	Locknut (5/16-18)	1	4
4		1106480	Adapter, Seat Frame	1	1
5		1028594	Pin, Arm Lock, Black	1	2
6		47471X074	Spring, Arm Lock	1	2
7		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
8		1029501	Plug Button, Black (1")	1	2