
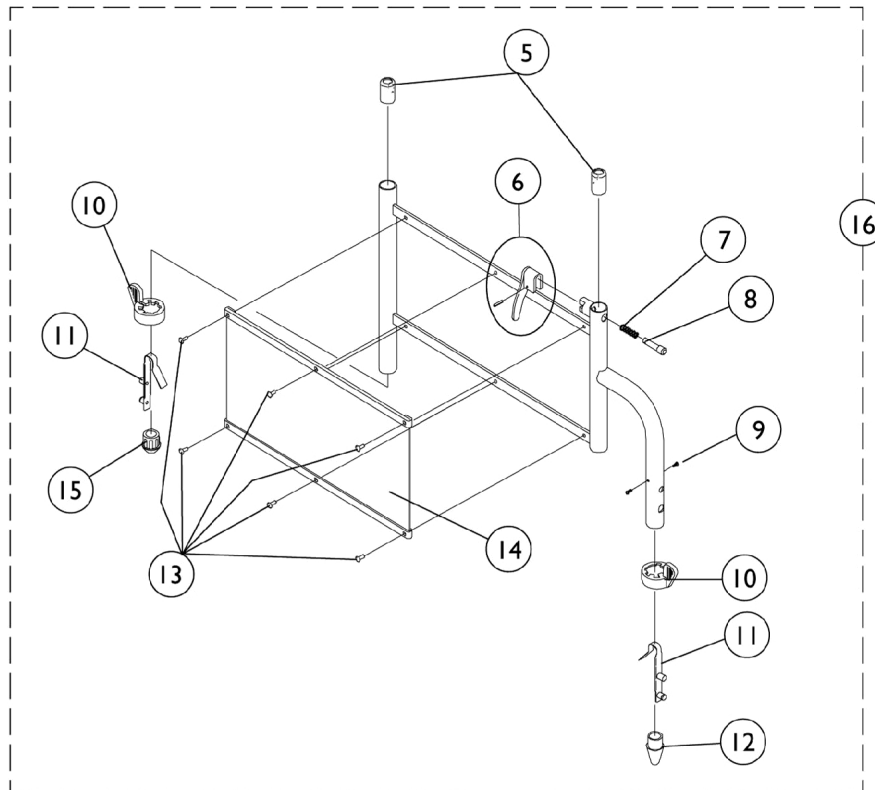
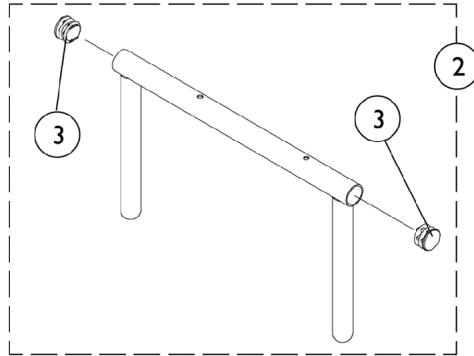


Adjustable Height Arms & Hardware

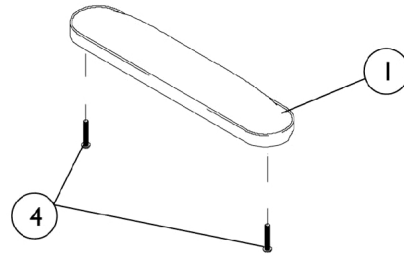

Adjustable Height Arms & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881011813-NL A	NLA - Pad, Arm, Full Length w/ Law Label	1	1
1	A	8881053569	Kit, Arm Pad w/ Screws, Full Length	2	1
1	A	8881019747-NL A	NLA - Pad, Arm - Desk Length	1	1
1	A	8881053568	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
2		1038470	Arm Tube Assembly, Conventional w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
2		1038472	Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right/Left	1	1
3		1027095	Plug Button, Black (7/8")	1	2
4		7930361-NLA	NLA - Package, Screw, Phillips Round Head (#10-32 x 1-1/8")	25	1
5		1036150	Anti-Rattle	1	2
6		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
7		47471X074	Spring, Arm Lock	1	1
8		47471X068	Pin, Locking	1	1
9		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	1
10		1015450	Handle, Quad Release	1	2
11		1018225	Spring, Torsion with Button Lock	1	2
12		1020060	Plug, Arm	1	1
13		1028645	Rivet	1	6
14	C	1064129	Guard, Clothing	1	1
15		1036484	Plug, Armrest, Black (3/4") (18GA)	1	1
16	B,C	8881061526-NL A	NLA - Arm Base, Color - Right	1	1
16	B,C	8881061527	Arm Base, Color - Left	1	1
NOTE: A - Specify upholstery color when ordering from Frame Finishes, Touch-Up Paints and Upholstery Color page. B - Specify frame color when ordering from Frame Finishes, Touch-Up Paints and Upholstery Color page. C - Stainless steel clothing guards were replaced w/ABS plastic. Chairs manufactured before 7/30/97 should be ordered in pairs to match.					