


Rollite Rollator


Model 65100, 65100R, 65100-JR, 65100R-JR, 68100 and 68100-TA

68100-TA (Midnight Blue) Shown

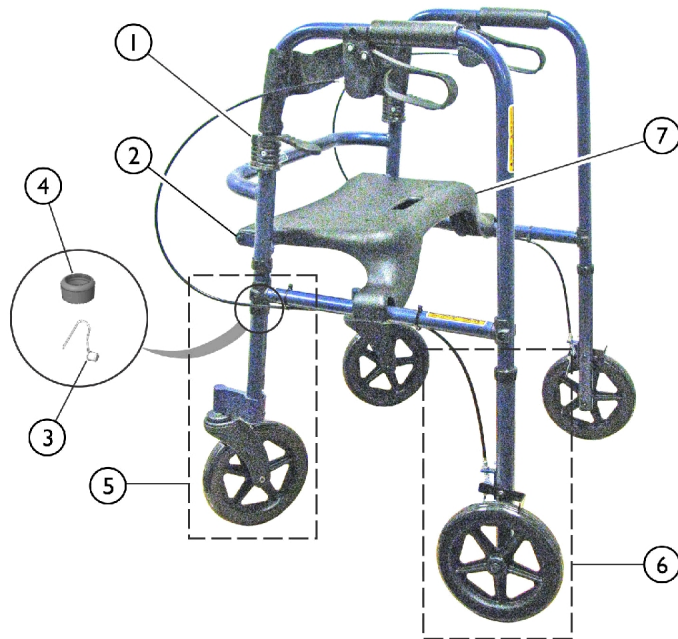

Rear Leg
Assemblies
Include
Brakes

65100R (Red) Shown

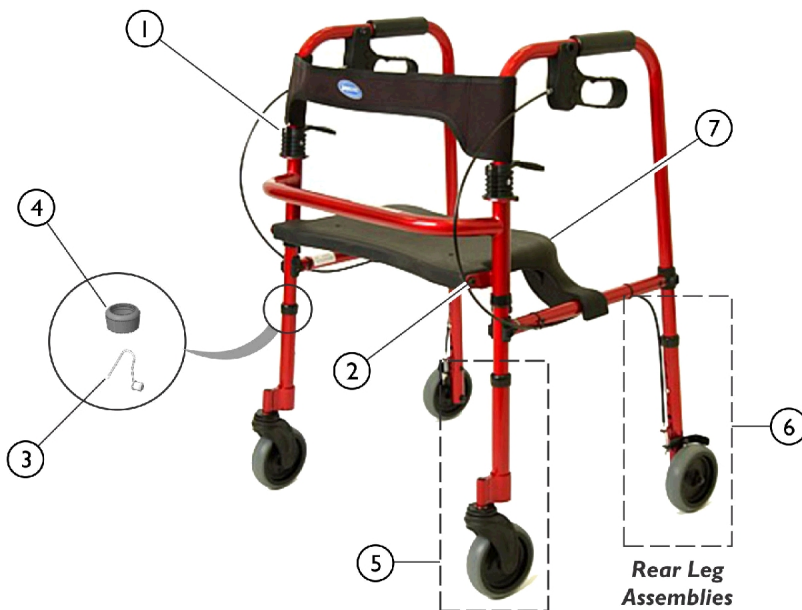

Rear Leg
Assemblies
Include
Brakes

Rollite Rollator

Model 65100, 65100R, 65100-JR, 65100R-JR, 68100 and 68100-TA

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	G	1043538-NLA	NLA - Nut, Push	1	4
1		1130784-NLA	NLA - Kit, Push Nut	10	1
	B	1109367-NLA	NLA - Seat Hardware	1	1
2	B	1021421-NLA	NLA - Screw, Phillips Pan Head (1/4-20 x 2-1/4")	1	2
2	B	1025195	Locknut (1/4-20)	1	2
2	B	1108477-NLA	NLA - Washer, Nylon (1/4 x 3/4 x 5/8")	1	2
2	B	1025728	Washer (1/4 x 1/2 x 1/32")	1	2
3		1094945	Button, Snap	1	4
4		1063124	Anti-Rattle, Grey (1-1/8")	1	4
5	C	1124256-NLA	NLA - Kit, Front Leg Extension Assembly, Adult - Midnight Blue, Black Wheel (65100)	2	1
5	C	1124258-NLA	NLA - Kit, Front Leg Extension Assembly, Adult - Electric Red, Black Wheel (65100R)	2	1
5	C	1124260-NLA	NLA - Kit, Front Leg Extension Assembly, Junior - Midnight Blue, Black Wheel (65100-JR)	2	1
5	C	1124262	Kit, Front Leg Extension Assembly, Junior - Electric Red, Black Wheel (65100R-JR)	2	1
5	C	1124264	Kit, Front Leg Extension Assembly, Adult - Midnight Blue, Black Wheel (68100)	2	1
5	C	1124266-NLA	NLA - Kit, Front Leg Extension Assembly, Tall Adult - Midnight Blue (68100-TA)	2	1
6	D,H	1124257-NLA	NLA - Kit, Rear Leg Extension and Brake Assembly, Adult - Midnight Blue, Black Wheel (65100)	2	1
6	D,H	1124261-NLA	NLA - Kit, Rear Leg Extension and Brake Assembly, Junior - Midnight Blue, Black Wheel (65100-JR)	2	1
6	D,H	1124259	Kit, Rear Leg Extension and Brake Assembly, Adult - Electric Red, Black Wheel (65100R)	2	1
6	D,H	1124263	Kit, Rear Leg Extension and Brake Assembly, Junior - Electric Red, Black Wheel (65100R-JR)	2	1
6	D,H	1124265-NLA	NLA - Kit, Rear Leg Extension and Brake Assembly, Adult - Midnight Blue, Black Wheel (68100)	2	1
6	D	1124267-NLA	NLA - Kit, Rear Leg Extension and Brake Assembly, Tall Adult - Midnight Blue (68100-TA)	2	1
7		1107737-NLA	NLA - Seat	1	1
	E	1046321-NLA	NLA - Paint, Touch-Up, 61P Electric Red	1	1
	E	1101326-NLA	NLA - Paint, Touch-Up, 105P Midnight Blue	1	1
	F	See Note	See Footnote	--	--
		65110	Basket, 65100, 65100R and 68100-TA	1	1
		65109	Basket, 65100-JR, 65100R-JR and 68100	1	1
		65408	Tray	1	1
	A	1107793-NLA	NLA - Backrest, Nylon Upholstery	1	1

Rollite Rollator

Model 65100, 65100R, 65100-JR, 65100R-JR, 68100 and 68100-TA

NOTE: A - Not available on Models 65100-JR, 65100R-JR and 68100.
B - Kit no longer available. Order each of item #2 in order to receive a complete kit.
C - Includes Front Caster and Leg Extension Tube.
D - Includes Hand Brake, Brake Cable, Brake Shoe, Leg Extension Tube and Hardware.
E - Not Shown.
F - For Swivel Wheel Attachment, see category titled, "Swivel Wheel Attachments."
G - Individual item no longer available, refer to Kit Part #1130784.
H - Wheels changed from Gray to Black on 8/18/2015.

NOTE: Front and Rear Leg Extensions are now sold in pairs only. Rear Leg Assembly includes the brake. Wheels (Item 6) changed from Gray to Black on 8/18/2015.