


# Rear Wheels & Motor Hardware

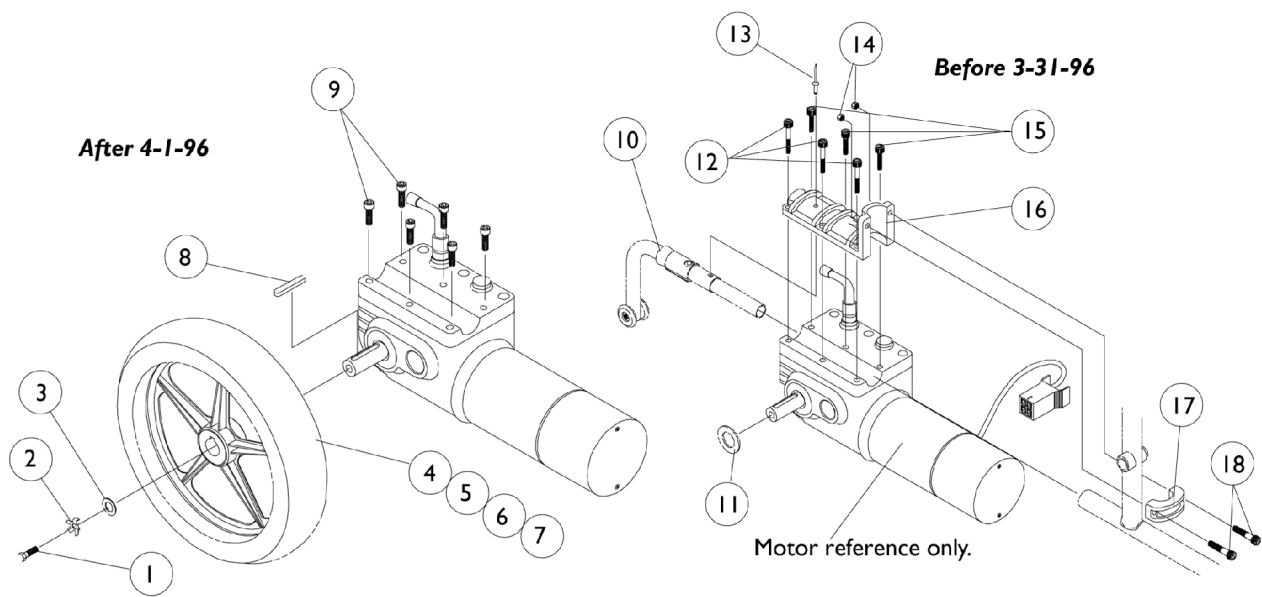

**NOTE:** Items 11 thru 18 only used on chairs before 3-31-96

## Rear Wheels & Motor Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930322-NLA	NLA - Package, Screw, Hex Head (M8 x 1.25 x 20mm)	25	1
2		1064179	Kit, Locking Tab Washer w/ Instruction Sheet (Pair)	2	1
3		1079247	Washer, Flat (21/64 x 53/64 x 7/64")	1	1
4		1064178-NLA	NLA - Drive Wheel Assembly, 5 Spoke, Pneumatic (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet	1	1
4		1064177	Drive Wheel Assembly, 5 Spoke w/ Foam Fill Insert (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet	1	1
5		1011524	Tire, Pneumatic, Grey (12-1/2" x 2-1/4")	1	1
6		1011525	Tube, Inner, Black (12-1/2 x 2-1/4")	1	1
6	A	1073936-NLA	NLA - Insert, Flat Free	1	1
7		1064176-NLA	NLA - Rim with Locking Tab	1	1
8		1023264	Key (6mm x 6mm x 40mm)	1	1
9		1057824-NLA	NLA - Screw, Socket Head (M6 x 16)	1	6
10	B	1034131-NLA	NLA - Anti-Tipper	1	1
10	C	1061684-NLA	NLA - Anti-Tipper	1	1
11		1023261-NLA	NLA - Washer, Steel {11/16 x 1-3/16 x 1/8")	1	1
12	D	1023257-NLA	NLA - Screw, Socket Head (M6 x 25MM)	1	3
13		1007094-NLA	NLA - Rivet, Pop (3/16") (UI Battery Boxes Only)	1	1
14		7930307-NLA	NLA - Locknut (1/4-20")	25	1
15		1023258-NLA	NLA - Screw, Socket Head Cap (M6 x 1 x 40mm)	1	3
16		1035012-NLA	NLA - Half Clamp, Motor	1	1
17		1035013-NLA	NLA - Half Clamp, Tube	1	1
18		7930304-NLA	NLA - Screw, Socket Head (1/4-20 x 1-1/4")	25	1

NOTE: A - As of 10/20/99 the "solid" flat free insert was replaced by a "foam fill" process. When a flat free insert is requested, a complete wheel must be ordered.  
 B - Before 3/31/96  
 C - After 4/1/96  
 D - No Longer Available.