


# Wheel Locks and Hardware

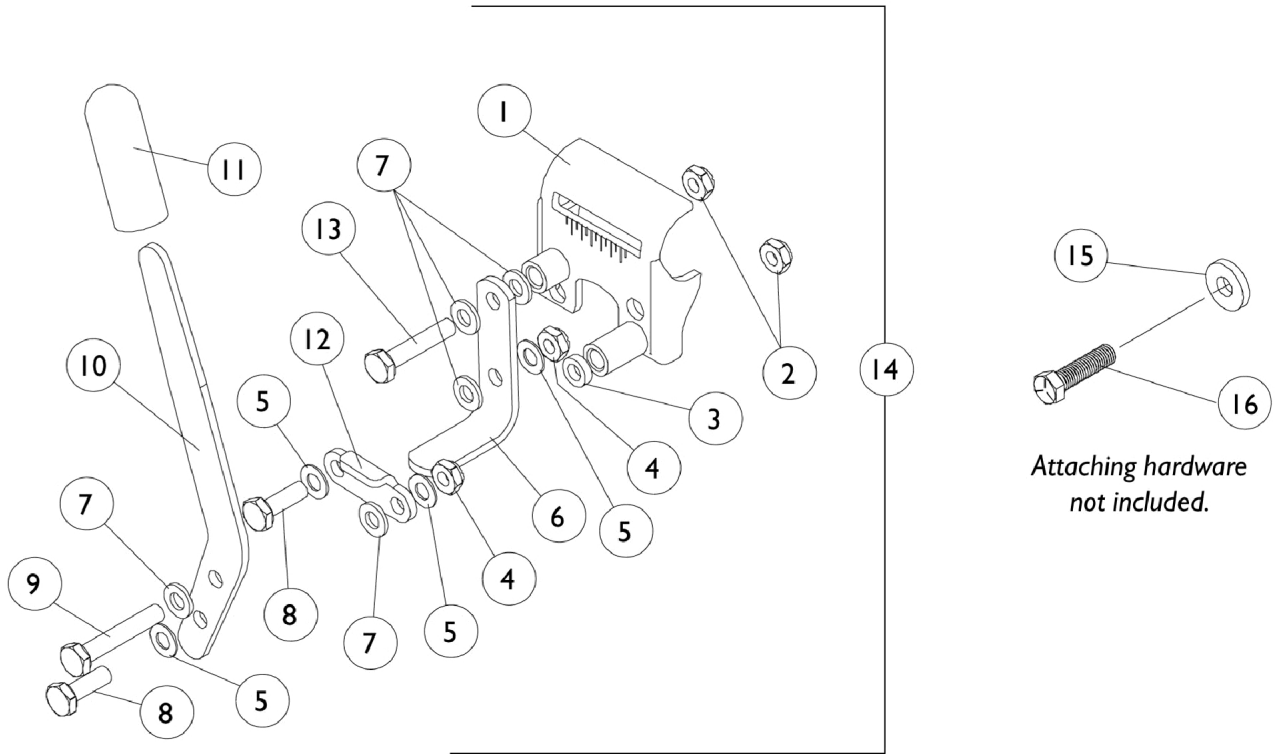

## Wheel Locks and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1031837-NLA	NLA - Bracket, Mounting, Black - Right	1	1
1		1031838-NLA	NLA - Bracket, Mounting, Black - Left	1	1
2		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	2
3		1026105-NLA	NLA - Spacer, Nylon (1/4 x 7/16 x 1/8")	1	1
4		7930334-NLA	NLA - Locknut (1/4-28)	25	2
5		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	4
6		1031833-NLA	NLA - Shoe, Wheel Lock	1	1
7		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	5
8		1028687	Screw, Hex Head (1/4-28 x 3/4")	1	2
9		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	1
10		1061504-NLA	NLA - Handle, Wheel Lock, Black	1	1
11		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
12		1018733-NLA	NLA - Link, Stop, Black	1	1
13		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
14		1061612-NLA	NLA - Wheel Lock Assembly - Right	1	1
14		1061613-NLA	NLA - Wheel Lock Assembly - Left	1	1
15		1018739	Washer (1/4 x 5/8 x 3/32")	1	1
16		7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	1