


Battery Box, Chargers, and Cables

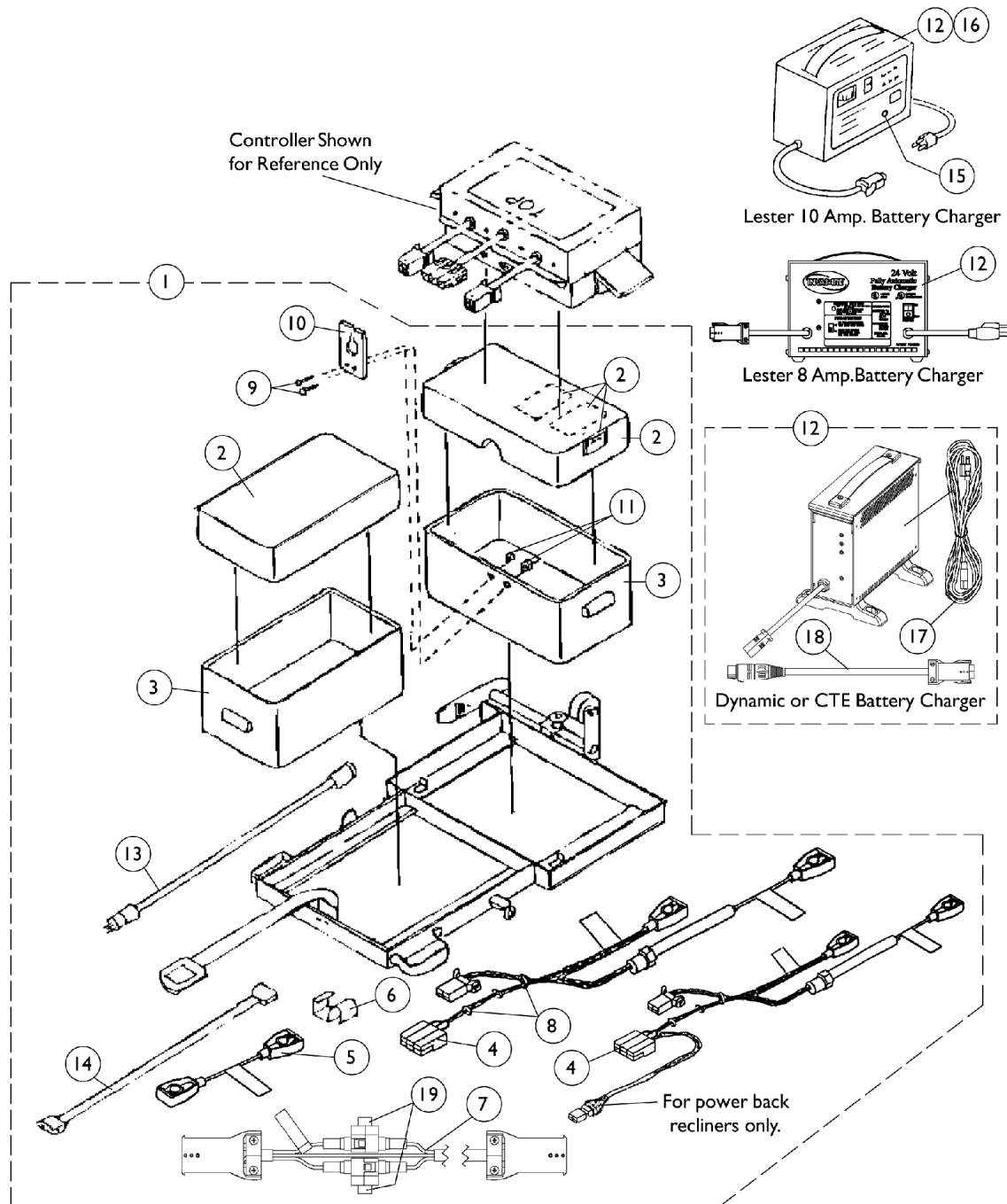

Battery Box, Chargers, and Cables Multi-Function Frame

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1033953-NLA	NLA - Battery Box with Carriage and Cables, Manual Back Recliner	1	1
1		1014798-NLA	NLA - Battery Box with Carriage and Cables, Power Back Recliner	1	1
2		1029685-NLA	NLA - Lid, Battery Box	1	1
2		1030443-NLA	NLA - Lid, Battery Box	1	1
2		1013033-NLA	NLA - Lid, Battery Box	1	1
2	A,D	1007406-NLA	NLA - Keepers	1	3
2	A,D	16348X010-NLA	NLA - Rivet (1/8")	1	4
2	A,D	16348X007	Rivet Washer (1/8 x 3/8 x 1/16")	1	4
2	A,D	47375X277-NLA	NLA - Fastener, Loop with Adhesive Back (1" W x 4" L)	1	2
3		1013309-NLA	NLA - Bottom, Box	1	1
4		1023702-NLA	NLA - Harness, Battery with Fuse	1	1
4		1104225-NLA	NLA - Harness, Battery with Fuse, Power Back Recliner	1	1
5		50000X123	Cable, Battery Jumper	1	1
6		1010681-NLA	NLA - Plug, Clamp	1	1
7		1013399-NLA	NLA - Cable, Charger Extension	1	1
8		1095503-NLA	NLA - Package, Tie Wrap (5-5/8" L)	10	1
9		7930335-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	2
10		50000X060-NLA	NLA - Guide, Cable	1	1
11		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	2
12		1014758-NLA	NLA - Charger, Battery, 10 Amp. 115vac, Off Board, Dual Mode Square Plug, Lester	1	1
12		1085685-NLA	NLA - Charger, Battery, 8 Amp. 115vac, Off Board, Dual Mode, Square Plug w/ AC Power Line Cord, Dynamic	1	1
12		1053163-NLA	NLA - Charger, Battery, 8 Amp, 115vac, Off Board, Dual Mode, Square Plug, Lester	1	1
12	D,D1	1062556-NLA	NLA - Battery Charger, 8 Amp. 115vac (Off Board) Single Mode Round Plug w/ AC Power Line Cord and Square Plug Adapter Cable, CTE Corp.	1	1
13		1013582-NLA	NLA - Cable, Remote Extension	1	1
14		1014284-NLA	NLA - Cable, Actuator Extension	1	1
15		1029291-NLA	Holder, Fuse, 1559 Dual Mode Charger	1	1
16		1028809-NLA	NLA - Timer, 1559 Dual Mode Charger	1	1
17	E	1089627-NLA	NLA - Cord, AC Power, 115VAC, 2 Prong Off-Board (11' 6" L)	1	1
17		1121086	Cord, AC Power, 115VAC, 3 Prong On-Board (6' 6-1/2" L)	1	1
18		1053831-NLA	NLA - Cable, Adapter	1	1
19	B	1098892-NLA	NLA - Fuse - Buss ATC-15 (15 AMP)	1	2
19	C	1090117-NLA	NLA - Fuse - Buss Mini ATM-15 (15 AMP)	1	2

Battery Box, Chargers, and Cables

Multi-Function Frame

NOTE: A - Part numbers 1007406, 16348X010, 16348X007, and 47375X277 are available for individual sale or as part of Miscellaneous Lid Hardware.
B - Before 3/11/03
C - After 3/10/03
D - CTE charger introduced 4/1/04. Used on Gel Cell or AGM (Absorbed Glass Mat) batteries. For wet cell batteries refer to the Lester charger.
D1 - CTE charger looks exactly the same as Dynamic's charger except for the company name that is silk screened on the narrow side panel, where the charger plug is (hard wired) attached. Serial number starts with 04A00000001.
E - No longer available as a service part. Please use 1121086 as its replacement.