





Battery Charger and Harnesses

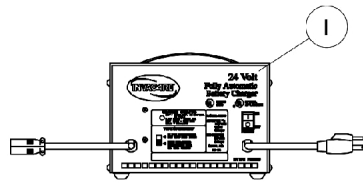


Lester Battery Charger

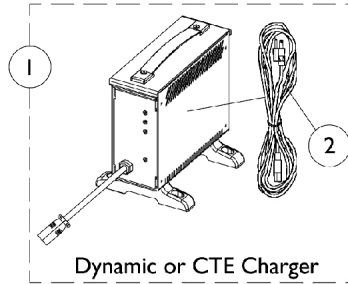



Dynamic or CTE Charger

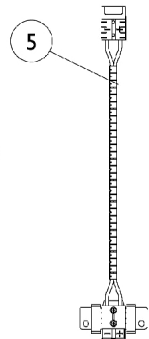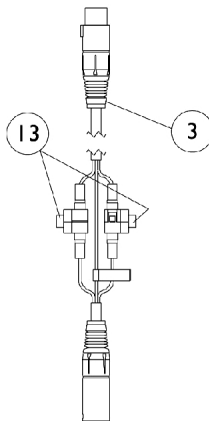

GP 24 battery harness for
R11 80 AMP. controller.
Available service part
11/1/01

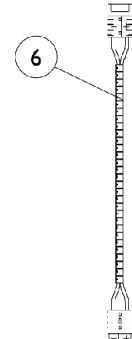

22NF battery harness for
R11 80 AMP. controller.
Available service part
11/1/01

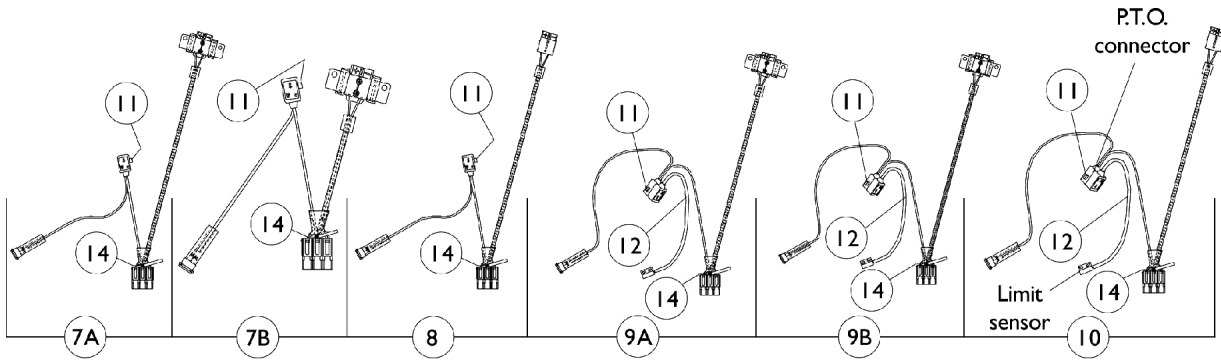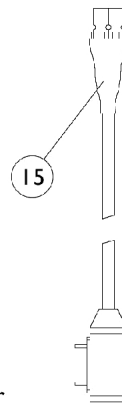

Battery Charger and Harnesses

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,P	1053161-NLA	NLA - Charger, Battery, 8 Amp, 115vac, Off Board Dual Mode Round Plug, Lester	1	1
	E	1053162-NLA	NLA - Charger, Battery, 12 Amp, 115vac, Off Board, Dual Mode, Round Plug, Lester	1	1
1		1085678-NLA	NLA - Charger, Battery, 8 Amp. 115vac, Off Board, Dual Mode, Round Plug w/ AC Power Line Cord, Dynamic	1	1
1	N,N1	1123249	Charger, Battery, 8 Amp. 115vac (Off Board) Single Mode Round Plug w/ AC Power Line Cord	1	1
1	N,N1	1123249E-NLA	NLA - WARR RPLMT - Charger, Battery, 8 Amp, 115VAC (Off Board) Single Mode Round Plug w/ AC Power Line Cord	1	1
2	O	1089627-NLA	NLA - Cord, AC Power, 115VAC, 2 Prong Off-Board (11' 6" L)	1	1
2		1121086	Cord, AC Power, 115VAC, 3 Prong On-Board (6' 6-1/2" L)	1	1
3		1083929-NLA	NLA - Cable, Charger Extension, Round Plug (24" L)	1	1
4	B	1092912-NLA	NLA - Programmer Extension Cable Assembly (18" Long)	1	1
5	D	1109509-NLA	NLA - Harness, Battery (GP24)	1	1
6	D	1107296-NLA	NLA - Harness, Battery (22NF)	1	1
7A	F,H	1095263-NLA	NLA - Harness, Wiring, Round Plug, GP24 Dual Battery Box	1	1
7B	G,H	1092314	Harness, Wiring, Black Battery Connector, Round Plug, GP24 Dual Battery Box	1	1
8	J	1092306-NLA	NLA - Harness, Wiring, Round Plug, 22NF Single Battery Box	1	1
9A	F,I	1095264-NLA	NLA - Harness, Wiring, Round Plug, GP24 Dual Battery Box	1	1
9B	G,I	1095265-NLA	NLA - Harness, Wiring, Round Plug, GP24 Dual Battery Box	1	1
10	K	1095266-NLA	NLA - Harness, Wiring, Round Plug, 22NF Single Battery Box	1	1
11		1090117-NLA	NLA - Fuse - Buss Mini ATM-15 (15 AMP)	1	1
12	C	1092913-NLA	NLA - Cable, Drive Lockout Sensor	1	1
13	L	1098892-NLA	NLA - Fuse - Buss ATC-15 (15 AMP)	1	1
13	M	1090117-NLA	NLA - Fuse - Buss Mini ATM-15 (15 AMP)	1	2
14	L	1101099-NLA	NLA - Fuse, Buss ATC-20 (20 AMP)	1	1
14	M	1119230-NLA	NLA - Fuse - Buss Mini ATM-20 (20 AMP)	1	1
15		1098905-NLA	NLA - Cable Assembly, 24 Volt, Anderson/P.T.O.	1	1
	E	1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	8

Battery Charger and Harnesses

- NOTE: A - On 7/18/01 the Lester 8 amp. battery charger was reinstated.
B - Used only when a 2nd Generation Seating System is ordered.
C - used for manual recliners and high back captain seats.
D - Battery harness used with the MKIV RII 80 AMP. controller for Torque and Torque SP models. Available service part as of 11/1/01.
E - Not Shown.
F - Before 9/24/99
G - After 9/23/99
H - For A.S.B.A. Non-Recliner, All Powered Seating Systems and H.B. Semi Recline Captain Seats - GP24 Dual Battery Boxes
I - For A.S.B.A. Manual Recliner and H.B. Full Recline Captain Seats - GP24 Dual Battery Boxes
J - For A.S.B.A. Non-Recliner, All Powered Seating Systems and H.B. Semi Recline Captain Seats - 22NF Single Battery Box
K - For A.S.B.A. Manual Recliner and H.B. Full Recline Captain Seats - 22NF Single Battery Box
L - Before 4/1/03
M - After 3/31/03
N - CTE charger introduced 4/1/04. Used on Gel Cell or AGM (Absorbed Glass Mat) batteries. For wet cell batteries refer to the Lester charger.
N1 - CTE charger looks exactly the same as Dynamic's charger except for the company name that is silk screened on the narrow side panel, where the charger plug is (hard wired) attached. Serial number starts with 04A0000001.
O - No longer available as a service part. Please use 1121086 as its replacement.
P - Use Battery Charger part number 1123249 as a replacement.

NOTE: A fused power take off (P.T.O.) connector was added to all harnesses 3/21/00.