


# Battery Charger, Cables and Wire Harness

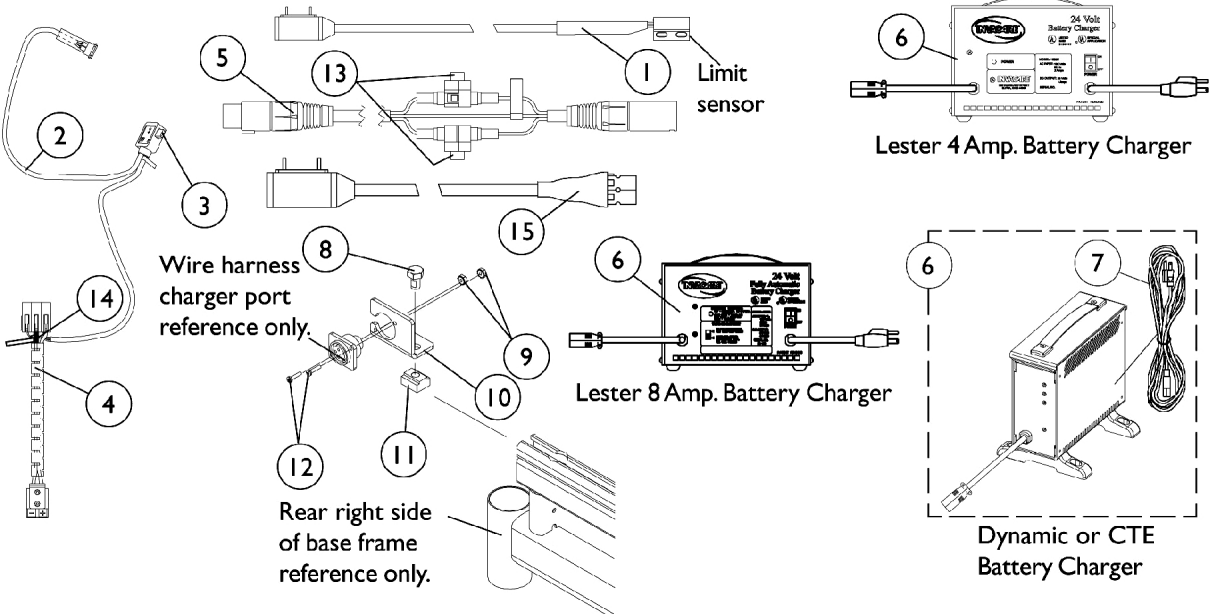

## Battery Charger, Cables and Wire Harness

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	1092913-NLA	NLA - Cable, Drive Lockout Sensor	1	1
2	B	1095623-NLA	NLA - Harness, Wiring	1	1
3		1090117-NLA	NLA - Fuse - Buss Mini ATM-15 (15 AMP)	1	1
4		1090357-NLA	NLA - Wrap, Slit Wire Harness (1/2" O.D., 36" Long Cut To Length)	1	1
5		1083929-NLA	NLA - Cable, Charger Extension, Round Plug (24" L)	1	1
6		1085678-NLA	NLA - Charger, Battery, 8 Amp. 115vac, Off Board, Dual Mode, Round Plug w/ AC Power Line Cord, Dynamic	1	1
6	D,K	1053161-NLA	NLA - Charger, Battery, 8 Amp, 115vac, Off Board Dual Mode Round Plug, Lester	1	1
6	E,H	1061411-NLA	NLA - Charger, Battery, 4 Amp. 115vac, Off Board, Dual Mode, Round Plug, Lester	1	1
6	I,I1	1123249	Charger, Battery, 8 Amp. 115vac (Off Board) Single Mode Round Plug w/ AC Power Line Cord	1	1
6	I,I1	1123249E-NLA	NLA - WARR RPLMT - Charger, Battery, 8 Amp, 115VAC (Off Board) Single Mode Round Plug w/ AC Power Line Cord	1	1
7	J	1089627-NLA	NLA - Cord, AC Power, 115VAC, 2 Prong Off-Board (11' 6" L)	1	1
7		1121086	Cord, AC Power, 115VAC, 3 Prong On-Board (6' 6-1/2" L)	1	1
8		1061922-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1/2")	1	1
9		P0870X000	Nut, Hex Keps (#4-40)	1	2
10		1070430-NLA	NLA - Bracket, Charger Connector, Black	1	1
11		1069724	T-Nut (5/16-18)	1	1
12		7930318-NLA	NLA - Package, Screw, Phillips Pan Head (#4-40 x 1/2")	25	2
13	F	1098892-NLA	NLA - Fuse - Buss ATC-15 (15 AMP)	1	1
13	G	1090117-NLA	NLA - Fuse - Buss Mini ATM-15 (15 AMP)	1	2
14	F	1101099-NLA	NLA - Fuse, Buss ATC-20 (20 AMP)	1	1
14	G	1119230-NLA	NLA - Fuse - Buss Mini ATM-20 (20 AMP)	1	1
15		1098905-NLA	NLA - Cable Assembly, 24 Volt, Anderson/P.T.O.	1	1
	A	1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	1
	A	1109152-NLA	NLA - Wrap, Slit Wire Harness (3/8" O.D.) (36" Long, Cut To Length)	1	1

NOTE: A - Not Shown  
 B - Wiring harness is used when a MKIV Quad Pulse controller, or MKIV RII for Weight Shifting Tilt controller are used.  
 C - Sensor cable is used when a manual recliner or high back captain seat is ordered  
 D - On 7/18/01 Lest 8 amp. battery charger was reinstated.  
 E - 250-S Models Only  
 F - Before 4/1/03  
 G - After 3/31/03  
 H - CTE 8 Amp. charger replaced the obsoleted 4 Amp. charger as of 4/1/04.  
 I - CTE charger introduced 4/1/04. Used on Gel Cell or AGM (Absorbed Glass Mat) batteries. For wet cell batteries refer to the 8 amp. Lester charger.  
 I1 - CTE charger looks exactly the same as Dynamic's charger except for the company name that is silk screened on the narrow side panel, where the charger plug is (hard wired) attached. Serial number starts with 04A00000001.  
 J - No longer available as a service part. Please use 1121086 as its replacement.  
 K - Use Battery Charger part number 1123249 as a replacement.