
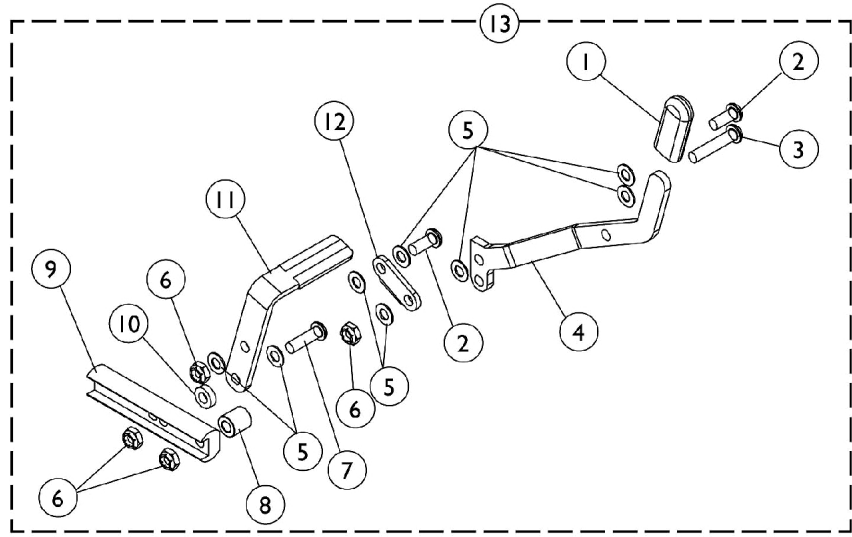
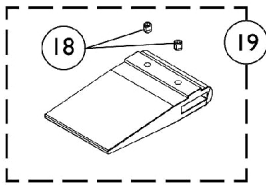


Wheel Lock - 12" Wheels

Compass Limited chairs (Pull to Lock)

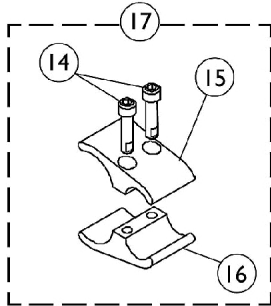

Wheel Lock - 12" Wheels

Compass Limited chairs (Pull to Lock)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1006490	Tip, Plastic, Black	1	1
2		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	2
3		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
4		1059345-NLA	NLA - Handle, Wheel Lock, Black - Right	1	1
4		1059346-NLA	NLA - Handle, Wheel Lock, Black - Left	1	8
5		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	4
6		1025765	Locknut (1/4-28)	1	1
7		1055306-NLA	NLA - Screw, Button Head Cap (1/4-28 x 7/8")	1	1
8		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
9		1080551-NLA	NLA - Strap, Wheel Lock	1	1
10		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
11		1021405-NLA	NLA - Shoe, Wheel Lock	1	1
12		1018727-NLA	NLA - Link, Stop, Black	1	1
13	A	1083090	Wheel Lock Assembly - Right	1	1
13	A	1083091	Wheel Lock Assembly - Left	1	1
14		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
15		1043711	Clamp, Half, Non-Threaded	1	1
16		1043712	Clamp, Half, Threaded	1	1
17	B	1068210-NLA	Kit, Wheel Lock Attaching Hardware (1" Tubing)	1	2
14			Screw, Allen Head (1/4-20 x 1")	1	2
15			Clamp, Half, Non-Threaded	1	1
16			Clamp, Half, Threaded	1	1
18		1008438	Screw, Set Socket Cup Point (#8-32 x 3/16")	1	2
19	C	1067279	Kit, Wheel Lock Shoe Spacer	1	1
			Spacer, Wheel Lock Shoe (2-7/16")	1	1
			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	2

NOTE: A - Includes items 1 thru 12.
B - Includes items 14 thru 16.
C - Includes Spacer and Set Screws (item 18)