


Articulating Legrest Support - ATL4A (Center Pivot)

Legrest Support does not include Legrest Pad. Available only on Rear Wheel Drive chairs

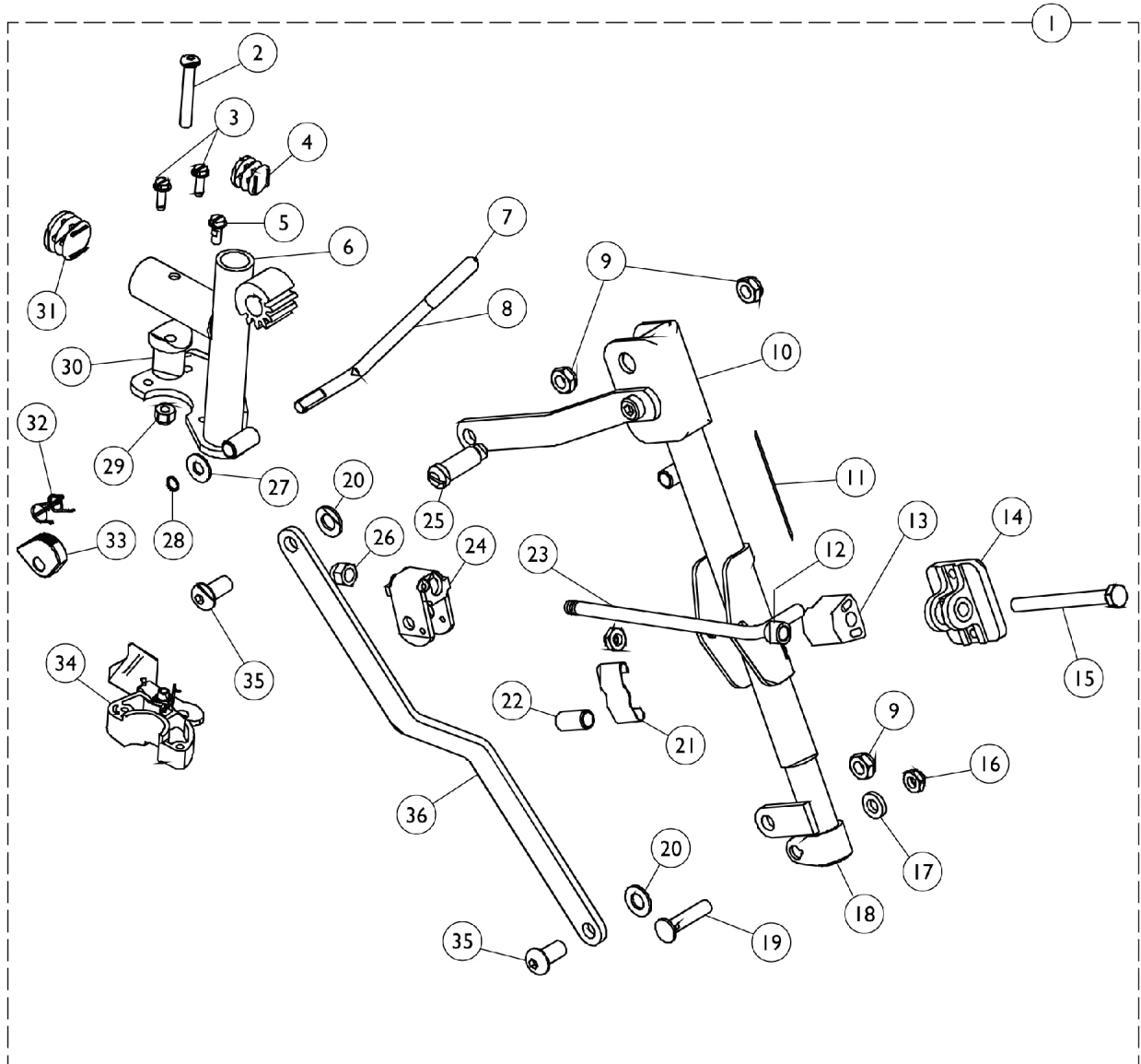

Articulating Legrest Support - ATL4A (Center Pivot)

Legrest Support does not include Legrest Pad. Available only on Rear Wheel Drive chairs

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1082540	Articulating Legrest Support Assembly, Sector, Chrome - Right	1	1
1		1082541	Articulating Legrest Support Assembly, Sector, Chrome - Left	1	1
2		7930301-NLA	NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4")	25	1
3		1025451	Screw, Slotted Hex Washer Head (#10-32 x 5/8")	1	2
4		1027097	Plug Button, Black (3/4")	1	1
5		7930314	Package, Screw, Slotted Hex Washer Head (#10-32 x 1/2")	25	1
6		1082538-NLA	NLA - Tube, Legrest Support, Articulating, Sector, Chrome - Right	1	1
6		1082539-NLA	NLA - Tube, Legrest Support, Articulating, Sector, Chrome - Left	1	1
7		1040060	Tip, Plastic, Black	1	1
8		1037126	Handle, Release	1	1
9		40363X000	Hex Nut (3/8"-16)	1	3
10		1022877-NLA	NLA - Tube, Support and Drive Link, Chrome - Right	1	1
10		1022878-NLA	NLA - Tube, Support and Drive Link, Chrome - Left	1	1
11		1024911-NLA	NLA - Decal, Smartleg	1	1
12		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
13		1025133	Spacer, Legrest Pad, Black	1	1
14		04526X019	Plate, Legrest Pad	1	1
15		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
16		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	2
17		1027377	Washer (#20)	1	1
18		1022848	Tube, Slide, Articulating, Chrome - Right	1	1
18		1022849	Tube, Slide, Articulating, Chrome - Left	1	1
19		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
20		1049527	Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
21		48823X008	Spring, Leaf	1	1
22		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
23		42326X004	Bar, Ratchet - Right/Left	1	1
24		04528X002	Bracket, Release, Black	1	1
25		1022854-NLA	NLA - Screw, Shoulder Slotted Head (1/2 x 1-1/8", 3/8-16)	1	1
26		7930336-NLA	NLA - Nut, Acorn (5/16-24)	25	1
27		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
28		1037485	Spring, Release	1	1
29		7930333	Locknut (1/4-20)	25	1
30		1017148	Pivot, Footrest	1	1
31		1029501	Plug Button, Black (1")	1	1
32		1037485	Spring, Release	1	1
33		1037125	Cam, Legrest	1	1
34		1025454	Housing, Latch - Right	1	1
34		1025455	Housing, Latch - Left	1	1

Articulating Legrest Support - ATL4A (Center Pivot)

Legrest Support does not include Legrest Pad. Available only on Rear Wheel Drive chairs

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
35		1022866-NLA	NLA - Screw, Socket Button Head (3/8-16 x 3/4")	1	2
36		1022846-NLA	NLA - Bar, Link, Black	1	1