


Tiller Assembly with Main Wiring Harness - Invacare 400 Scooter

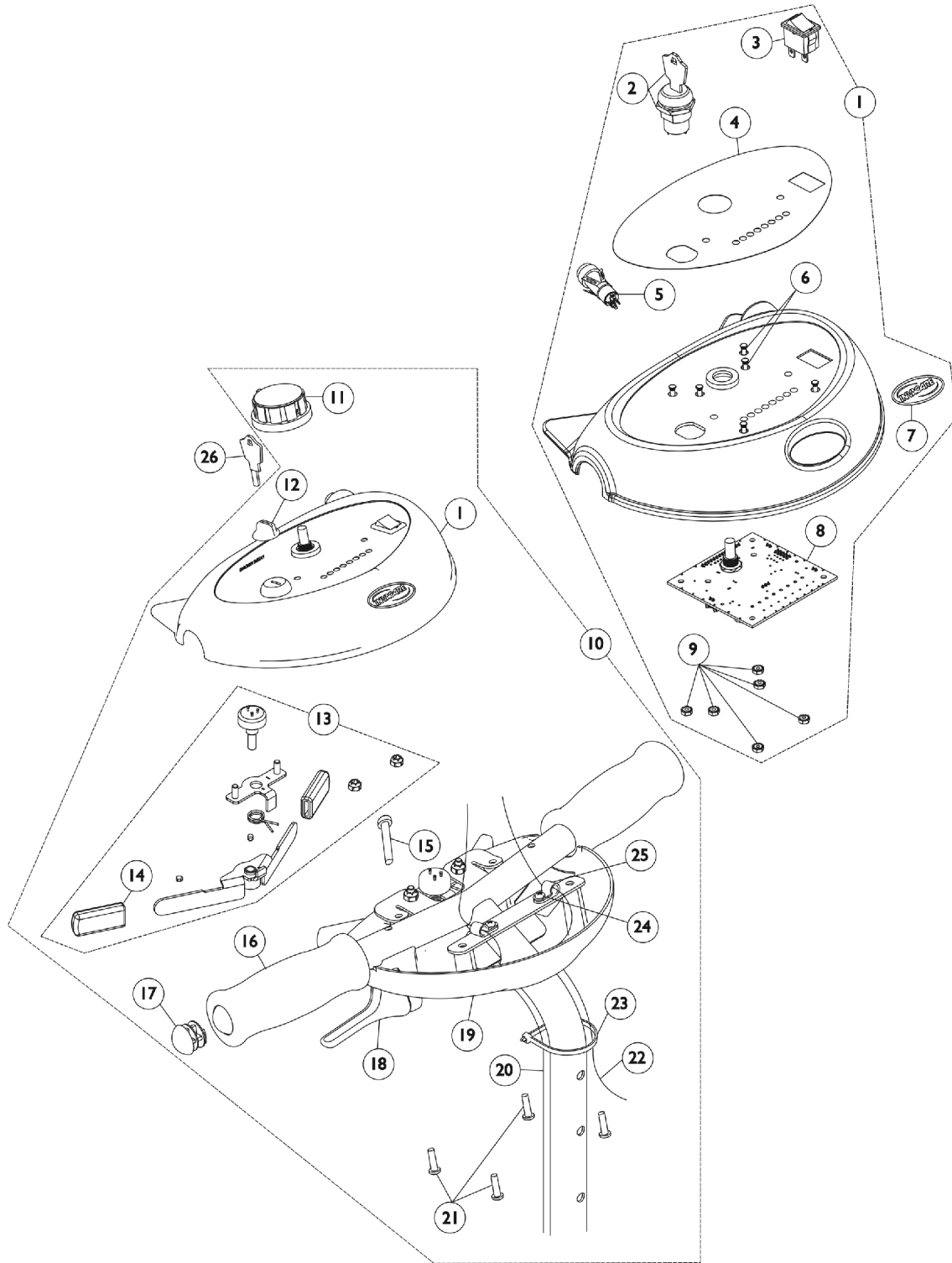

Tiller Assembly with Main Wiring Harness - Invacare 400 Scooter

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1125342-NLA	NLA - Shroud, Tiller with Circuit Board - Candy Red Pearl (Invacare 400 Scooter)	1	1
2			NLA - Key Switch with Key Assembly	1	1
3			NLA - Switch, Light	1	1
4			NLA - Decal, Faceplate	1	1
5			Switch, Horn	1	1
6			NLA - Stud (#4-40 x 3/8")	1	6
7			NLA - Decal, Invacare 3-D Logo (1-19/32" L)	1	1
8			NLA - Circuit Board with Speed Pot	1	1
9			Nut, Hex Keps (#4-40)	1	6
			NLA - Label, Patent Pending	1	1
			NLA - Shroud, Tiller, Upper, Candy Red Pearl	1	1
1	A	1125343-NLA	NLA - Shroud, Tiller with Circuit Board - Deep Blue Pearl (Invacare 400 Scooter)	1	1
			NLA - Key Switch with Key Assembly	1	1
			NLA - Decal, Faceplate	1	1
			NLA - Switch, Light	1	1
			Switch, Horn	1	1
			NLA - Stud (#4-40 x 3/8")	1	6
			NLA - Decal, Invacare 3-D Logo (1-19/32" L)	1	1
			NLA - Circuit Board with Speed Pot	1	1
			Nut, Hex Keps (#4-40)	1	6
			NLA - Label, Patent Pending	1	1
			NLA - Shroud, Tiller, Upper, Deep Blue Pearl	1	1
26	F	1126100-NLA	NLA - Key, Ignition	1	1
13	C,D	1119473-NLA	NLA - Kit, Throttle Lever with Instruction Sheet	1	1
			NLA - Throttle Lever Assembly	1	1
			Instruction Sheet, Throttle Lever	1	1
			NLA - Bag, Poly Zip-Lip (8-3/4" L x 6" W)	1	1
14		1114265-NLA	NLA - Cap, End, Throttle, Black	1	2
10	B	1124904-NLA	NLA - Tiller Assembly - Cand Red Pearl (Invacare 400 Scooter)	1	1
			NLA - Key Switch with Key Assembly	1	1
			NLA - Decal, Faceplate	1	1
			NLA - Switch, Light	1	1
			Switch, Horn	1	1
			NLA - Stud (#4-40 x 3/8")	1	6
			NLA - Circuit Board with Speed Pot	1	1
			Nut, Hex Keps (#4-40)	1	6
			NLA - Label, Patent Pending	1	1
			NLA - Shroud, Tiller, Upper, Candy Red Pearl	1	1
			NLA - Bag, Poly Zip-Lip (8-3/4" L x 6" W)	1	1
			Instruction Sheet, Throttle Lever	1	1
			NLA - Throttle Lever Assembly	1	1
			Knob, Control	1	1
			NLA - Boot, Key Switch	1	1

Tiller Assembly with Main Wiring Harness - Invacare 400 Scooter

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
10	B	1124905-NLA	NLA - Screw, Socket Head (#10-24 x 1-1/4")	1	1
			Plug Button, Black (5/8")	1	2
			NLA - Grip, Hand, Black Foam	1	2
			NLA - Handle, Tiller Angle Adjustable with Cable (Invacare 400 Scooter)	1	1
			NLA - Shroud, Tiller, Lower	1	1
			NLA - Clamp, Cable	1	2
			NLA - Screw, Phillips Pan Head (#6-20 x 3/8")	1	2
			Tie Wrap (5-5/8" L)	1	1
			NLA - Harness, Wiring, Light	1	1
			Screw, Phillips Pan Head (#8-16 x 5/8")	1	4
			NLA - Harness, Wiring, Main Tiller	1	1
			NLA - Tiller Weldment - Black (Invacare 400 Scooter)	1	1
			NLA - Label, Serial Number	1	1
			NLA - Decal, Invacare 3-D Logo (1-19/32" L)	1	1
			NLA - Tiller Assembly - Deep Blue Pearl (Invacare 400 Scooter)	1	1
			NLA - Key Switch with Key Assembly	1	1
			NLA - Decal, Faceplate	1	1
			NLA - Switch, Light	1	1
			Switch, Horn	1	1
			NLA - Stud (#4-40 x 3/8")	1	6
			NLA - Decal, Invacare 3-D Logo (1-19/32" L)	1	1
			NLA - Circuit Board with Speed Pot	1	1
			Nut, Hex Keps (#4-40)	1	6
			NLA - Label, Patent Pending	1	1
			NLA - Shroud, Tiller, Upper, Deep Blue Pearl	1	1
			NLA - Bag, Poly Zip-Lip (8-3/4" L x 6" W)	1	1
			Instruction Sheet, Throttle Lever	1	1
			NLA - Throttle Lever Assembly	1	1
			Knob, Control	1	1
			NLA - Boot, Key Switch	1	1
			NLA - Screw, Socket Head (#10-24 x 1-1/4")	1	1
			Plug Button, Black (5/8")	1	2
			NLA - Grip, Hand, Black Foam	1	2
			NLA - Handle, Tiller Angle Adjustable with Cable (Invacare 400 Scooter)	1	1
			NLA - Shroud, Tiller, Lower	1	1
			NLA - Clamp, Cable	1	2
			NLA - Screw, Phillips Pan Head (#6-20 x 3/8")	1	2
			Tie Wrap (5-5/8" L)	1	1
			NLA - Harness, Wiring, Light	1	1
			Screw, Phillips Pan Head (#8-16 x 5/8")	1	4
NLA - Harness, Wiring, Main Tiller	1	1			
NLA - Tiller Weldment - Black (Invacare 400 Scooter)	1	1			

Tiller Assembly with Main Wiring Harness - Invacare 400 Scooter

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
26	F	1126100-NLA	NLA - Label, Serial Number	1	1
			NLA - Key, Ignition	1	1
NOTE: A - Includes: Items 2-9 and Patent label and Upper Shroud (not shown). B - Includes: Items 2-25 and Patent Label, Upper Shroud, Tiller Weldment, and Serial Label (not shown). C - Includes: All items contained in box as shown in drawing with Instruction Sheet. D - Throttle Pot is included with this assembly. F - This Key is included with item 2.					